Almohadilla AirTouch™ N20

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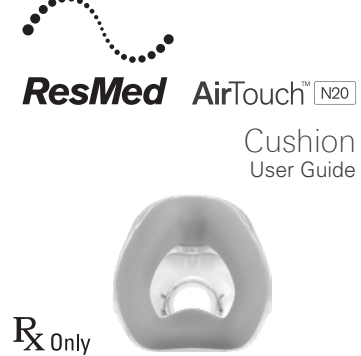
ENGLISH
The AirTouch™ N20 foam cushion is compatible with the AirFit™ N20 mask system. When assembling and disassembling the foam cushion to the AirFit N20 mask, follow the steps in the AirFit N20 user guide. For normal use and maintenance of components other than the foam cushion, refer to the AirFit N20 user guide. In addition, follow the steps below for maintenance, troubleshooting and specific warnings related to use of the AirTouch N20 foam cushion.
AirTouch™ N20 cushion
The cushion is intended for single-patient reuse in both home and hospital/institutional environments and recommended to be replaced monthly.

The cushion is intended for single-

patient reuse in both home and hospital/institutional environments and recommended to be replaced monthly.

⚠ CAUTION

- The cushion must not be submerged in liquid as this may damage the cushion and affect mask performance.**
- Discard the cushion if it is submerged in liquid or heavily soiled.**
- Do not expose to domestic cleaning products or cleaning devices.**

Cleaning your mask at home

Daily/After each use:

If required, wipe your cushion with a CPAP wipe or similar alcohol-free wipe and then allow to dry.

Monthly:

Replace the cushion by following relevant steps of the disassembly / reassembly instructions.

Troubleshooting

Mask leaks around the face.

Cushion is nearing the end of its life.

The cushion is designed to be replaced on a monthly basis. Cushion performance may deteriorate as the foam nears the end of its life. Adjust your headgear and if the problem persists, replace the cushion.

Cushioned is not positioned correctly.

Repostion mask on your face. If leak continues, turn off device, resettle your cushion and then turn device back on.

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