

# AirTouch™ F30i Comfort

Minimalist Freedom Versatile Fit

The Resmed AirTouch™ F30i Comfort features a top-of-head tube connection, fabric-wrapped frame, and ComfiSoft™ silicone-coated fabric cushion with AdaptiSeal™ technology, designed to adapt to the face and deliver softness at every touch for all-around comfort.

Full face mask



## How to fit the AirTouch F30i Comfort



Place the cushion under the nose and ensure it sits comfortably against the face. With the Resmed logo on the headgear facing up, pull the headgear and frame over the head.



Bring the lower headgear straps under the ears and attach the magnetic clips\* to the frame. If needed, feed longer hair through the back of the headgear.



Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.

## Ordering information

Product	Part number	Recommended replacement schedule**
<b>Complete mask system (includes frame with elbow, cushion and headgear)</b> <ul style="list-style-type: none"><li>Small frame—medium cushion</li><li>Standard frame—small wide/medium/large cushion</li><li>Large frame—medium cushion</li></ul>	62442 62443/62444/62445 62446	-
<b>Frame system (includes frame with elbow and cushion)</b> <ul style="list-style-type: none"><li>Small frame - small wide/medium cushion</li><li>Standard frame - small wide/medium/large cushion</li><li>Large frame - medium/large cushion</li></ul>	62457/62458 62460/62461/62462 62464/62465	Every 3 months
<b>Cushion</b> <ul style="list-style-type: none"><li>Small wide</li><li>Medium</li><li>Large</li></ul>	62489 62490 62491	Every month
<b>Headgear (includes magnetic clips*)—standard</b>	63372	Every 6 months

\*This mask contains magnets that may interfere with certain implants of medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

\*\*The recommended replacement schedule is a reflection of the [Medicare accessories replacement schedule](#). The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.