

How to fit the AirTouch F30i Comfort



With one hand holding the bottom of the cushion and the other on the frame, position the mask in front of your face.



Place the cushion under your nose and ensure it sits against your face. With the Resmed logo on the headgear facing up, pull the headgear and frame over your head.



Bring the lower headgear bands under your ears and attach the magnetic clips* to the frame connectors.




Undo the fastening tabs on the upper headgear bands and pull evenly.



Repeat step four with the lower headgear bands.



Connect the air tubing from the device to the mask elbow.

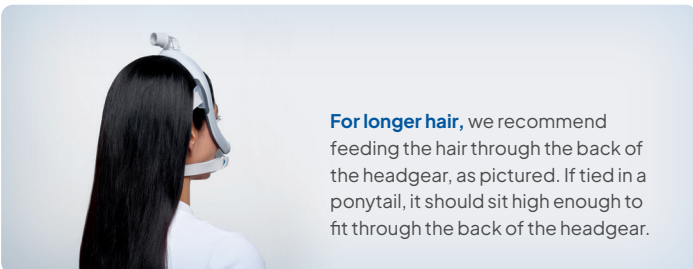


For longer hair, we recommend feeding the hair through the back of the headgear, as pictured. If tied in a ponytail, it should sit high enough to fit through the back of the headgear.

Crafted for softness and comfort to elevate the sleep experience

The AirTouch F30i Comfort offers patients soft, breathable comfort from the very first fit. Its adaptive seal helps maintain stability as patients move, while the top-of-head tube connection supports freedom to sleep in their preferred position. QuietAir™ venting helps create a more restful environment for patients and their bed partners, and the mask is designed to be easy to wear so it can naturally become part of a bedtime routine — helping patients feel confident staying on therapy.



Scan the QR code or contact your Resmed sales representative to get going.

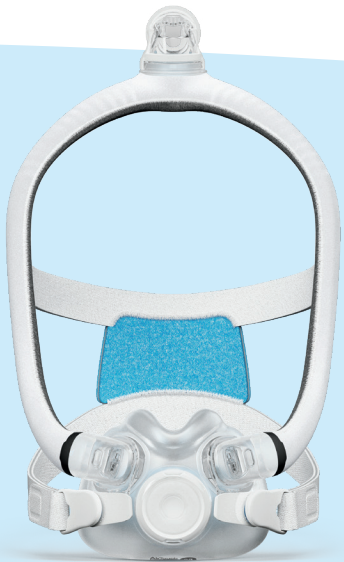

* The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

† Australian Good Design Award Winner 2025, awarded by Good Design Australia

Patients are encouraged to speak with their treating clinicians for any clinical symptoms or medical conditions.

¹ Resmed internal testing for the sound power of AirTouch F30i mask with QuietAir vent, ID: A5637990

² Resmed internal testing for the moisture-wicking properties of fabric, ID: A5498268.

Distributed by Resmed Corp., 9001 Spectrum Center Blvd., San Diego, CA 92123 USA. AdaptiSeal, AirFit, AirTouch, ComfSoft, and QuietAir are trademarks of the Resmed family of companies. For patent and other intellectual property information, see Resmed.com/ip. © 2026 Resmed. 10114460/2 2026-02

Softness with
every touch.
Confidence
in every fit.

The Resmed
AirTouch™ F30i Comfort
full face mask



Designed for all-around comfort

The AirTouch F30i Comfort is designed to surround patients in softness with a fully fabric-wrapped frame and a ComfiSoft™ cushion that fuses breathable fabric with flexible silicone. Together, they create a mask that feels gentle all around, while AdaptiSeal™ technology adapts to facial contours for a secure fit.



Fabric-wrapped flexible frame
Available in three sizes, the frame is designed to adjust easily to different facial profiles and head shapes for an accurate fit the first time, saving time in setup.


Magnetic clips*
Snap on and off for easy fit, adjustment and removal.

Flexible and UltraCompact full face cushion
The flexibility and under-the-nose design reduce facial coverage, creating a lighter feel on the face. Available in three sizes and interchangeable with the AirFit™ F30i and AirFit X30i.

A Resmed-first innovation
The combination of our innovative ComfiSoft™ silicone-coated fabric cushion with AdaptiSeal™ technology that contours to the face without any rigid plastic is designed to deliver maximized comfort and secure seal performance.



Top-of-the-head tube connection and quick-release elbow
Allows patients to sleep in the position they want via 360-degree elbow rotation, while also enabling them to easily connect and disconnect the device tubing.

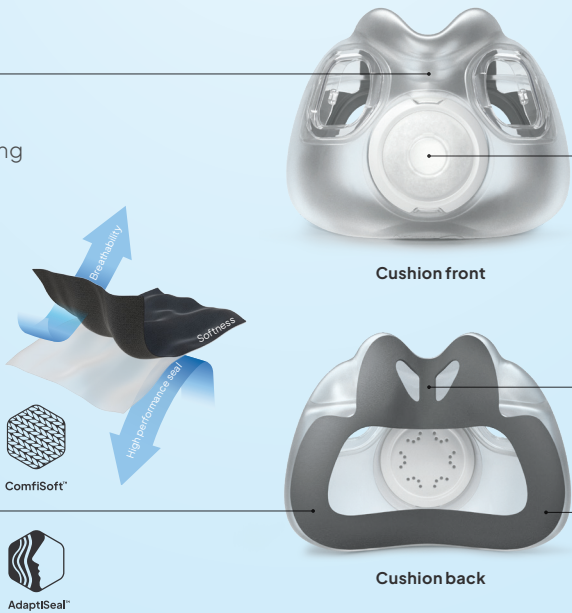

QuietAir™ venting
Measured 20 decibels, quiet and diffused venting minimizes disturbance to both patients and bed partners.¹

Nasal septum bridge
Designed to hold the shape of the cushion to create a comfortable and reliable seal.

Soft, breathable and moisture-wicking
Soft fabric helps to keep the nose and mouth area cooler and less sweaty for ultimate skin comfort.²



Complete mask system
62442
62443
62444
62445
62446

QuietAir vent

UltraCompact full face cushion
62489
62490
62491

Fabric-wrapped frame with elbow

F30i/X30i headgear
63372

Magnetic clips*

Frame system

62457, 62458, 62460, 62461, 62462, 62464, 62465

AirTouch™ F30i Comfort

Complete mask system
(includes mask frame with elbow, full face cushion and headgear)

Product name	Product code
AirTouch F30i Comfort mask system - M/SML	62442
AirTouch F30i Comfort mask system - SW/STD	62443
AirTouch F30i Comfort mask system - M/STD	62444
AirTouch F30i Comfort mask system - L/STD	62445
AirTouch F30i Comfort mask system - M/LGE	62446

UltraCompact full face cushion with QuietAir vent
(ComfiSoft™ cushion with AdaptiSeal™ technology)

Product name	Product code
AirTouch F30i cushion - SW	62489
AirTouch F30i cushion - M	62490
AirTouch F30i cushion - L	62491

Frame system
(includes fabric-wrapped frame with elbow and full face cushion)

Product name	Product code
AirTouch F30i Comfort frame system - SW/SML	62457
AirTouch F30i Comfort frame system - M/SML	62458
AirTouch F30i Comfort frame system - SW/STD	62460
AirTouch F30i Comfort frame system - M/STD	62461
AirTouch F30i Comfort frame system - L/STD	62462
AirTouch F30i Comfort frame system - M/LGE	62464
AirTouch F30i Comfort frame system - L/LGE	62465

Headgear
(includes magnetic clips*)

Product name	Product code
F30i/X30i headgear	63372

Cushion sizes: Small-wide (SW), Medium (M), Large (L)
Frame sizes: Small (SML), Standard (STD), Large (LGE)

Live life in high Res

