

## Fitting Guide



**Pixi Hint!** If fitting the mask for the first time, we recommend the child be sitting in an upright position to ensure correct fit.



- Hold the mask gently against the child's face, checking that the nose is fully enclosed by the cushion and the mouth is clear.



- Pull the headgear over the head and move the middle strap down until it sits across the middle of the back of the head.



- Keeping your hand on the frame to maintain correct headgear position, bring the lower headgear strap behind the back of the neck.



- Attach the lower headgear strap to the mask by holding the tab and hooking the quick release clip onto the lower headgear arm.



- If necessary, adjust the straps by removing the Velcro® and gently tightening the headgear straps until comfortable. Do not overtighten.
- For optimal results, adjust the straps starting at the bottom strap and work upwards.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

### Tip



- When fully fitted, ensure the top strap is not overtightened and vertically aligned with the top of the ears; the middle strap should sit across the back of the head horizontally aligned with the top of the ears.



For details, refer to the **Pixi Paediatric Mask User Guide.**

**Combine the Pixi mask** with the latest Stellar or VPAP devices for a therapy system that delivers greater comfort, reliability and effective care.



**Stellar™**



**VPAP™ S**



**VPAP™ ST**



**Pixi Mask**



**Pixi™**  
PAEDIATRIC MASK