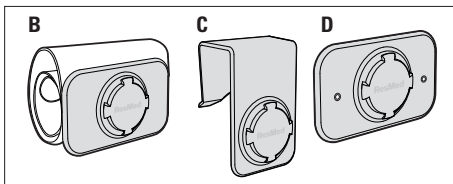
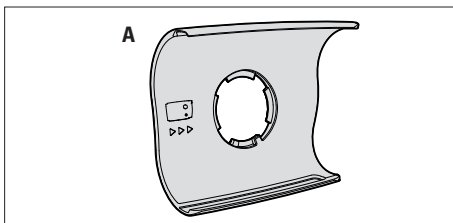
**ResMed**

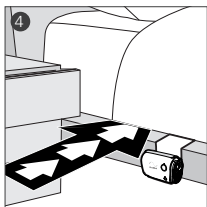
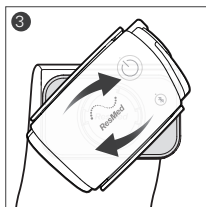
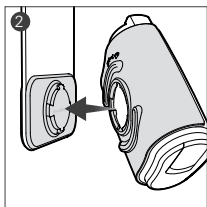
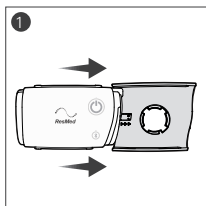
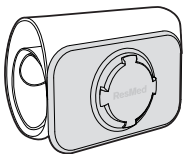
**AirMini™** MOUNT SYSTEM

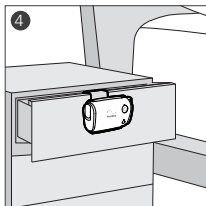
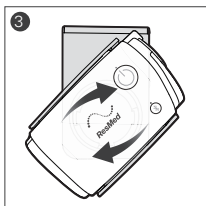
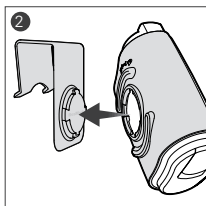
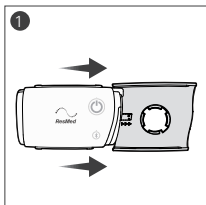
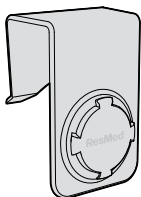
## User Guide

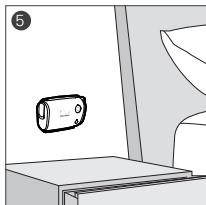
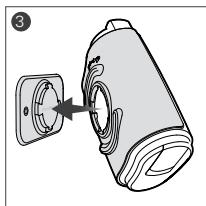
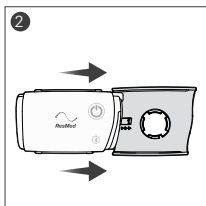
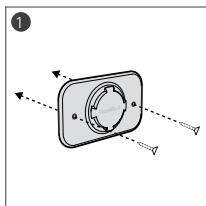
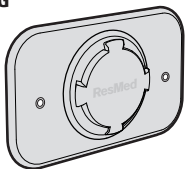
English | Deutsch | Français | Italiano |  
Español | Português | Nederlands | Svenska |  
Dansk | Norsk | Suomi | Český | Eesti |  
العربية | 简体中文 | 繁體中文 | 日本語

E

**F**

**G**

## Intended use

The AirMini mount system allows the AirMini device to be secured to points such as a bedframe, bedside table, wall or airplane seat pocket.

The AirMini mount system is not intended to be worn.

## Components

Refer to illustrations A, B, C and D on page 1.

| Part | Description                                     |
|------|---|
| A    | Mount   |
| B    | Mattress module                                 |
| C    | Hook module                                     |
| D    | Wall module (screws/adhesive tape not included) |

## Using the mount system

### Mattress module

The mattress module can be used to attach the AirMini to the side of a bed, between a bedframe and mattress. Refer to illustration E on page 2.

1. Slide the AirMini into the mount.
2. Hold the mount at an angle and push it onto the mattress module. Adjust the angle to fit the mount in place.
3. Turn the mount clockwise until it locks into place.
4. Lay the strap flat underneath the mattress.

## **Hook module**

The hook module can be used to hang the AirMini onto various locations such as a bed rail, bedside table or airplane seat pocket. Refer to illustration F on page 3.

1. Slide the AirMini into the mount.
2. Hold the mount at an angle and push it onto the hook module. Adjust the angle to fit the mount in place.
3. Turn the mount clockwise until it locks into place.
4. Hang the hook onto a bed rail, bedside table or airplane seat pocket.

## **Wall module**

The wall module can be used to mount the AirMini to a wall. Refer to illustration G on page 4.

1. Attach the wall module to the wall with screws and/or adhesive double-sided tape (not included).
2. Slide the AirMini into the mount.
3. Hold the mount at an angle and push it onto the wall module. Adjust the angle to fit the mount in place.
4. Turn the mount clockwise until it locks into place.

## WARNINGS

- Read instructions before use.
- Do not attempt to modify the mount system. There are no user-servicable parts in the mount system.

**Note:** For any serious incidents that occur in relation to this product, these should be reported to ResMed and the competent authority in your country.

## Technical specifications

---

### Gross Dimensions

Mount (excluding modules): 87 mm (H) x 107 mm (W) x 42mm (D) (3.4" (H) x 4.2" (W) x 1.7" (D))

---

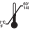
Wall module screw max diameter: 2.5mm

---

Design Life: 12 months.

---

# Symbols

**EC REP** European Authorized Representative (ISO 15223-1).  Manufacturer (ISO 15223-1). **LOT** Batch Code (ISO 15223-1). **REF** Catalogue Number (ISO 15223-1). **SN** Serial number (ISO 15223-1).  Humidity limitation (ISO 15223-1).  Temperature limitation (ISO 15223-1).  Atmospheric pressure limitation (ISO 15223-1).  Importer. **MD** Medical device.

See symbols glossary at [ResMed.com/symbols](https://www.resmed.com/symbols).

## Limited warranty

ResMed warrants that your AirMini mount system shall be free from defects in material and workmanship for a period of 90 days from the date of purchase by the initial customer. This warranty is not transferable.

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See [ResMed.com](http://ResMed.com) for other ResMed locations worldwide. AirMini is a trademark and/or registered trademark of the ResMed family of companies. For patent and other intellectual property information, see [ResMed.com/ip](http://ResMed.com/ip). © 2019 ResMed Pty Ltd. 388139/1 2019-11
**ResMed Pty Ltd**