ResMed

Astral DC/DC Adapter

User Guide

English | Deutsch | Français | Italiano | Español | Português | Nederlands | Svenska |
Dansk | Norsk | Suomi | Ελληνικά | Polski | Türkçe | Česky | 简体中文 | 繁體中文 | 日本語 |
Български | Hrvatski | Eesti | Magyar | Íslenska | Latviski | Lietuvių k. | Română | Русский |
Slovenčina | Slovenščina | فارسی | العربية | עברית | اردو

English

The ResMed DC/DC Adapter ("Adapter"), provides an additional method of operating an Astral™ 100 or Astral 150 Ventilation device where AC mains power is not available.

The Adapter plugs directly into a standard DC socket (eg, an automotive cigarette lighter socket) to power the Astral device and charge the internal battery.

The Adapter can be used with power sources ranging from 12V to 24V DC.

⚠️ WARNINGS

- Read instructions before use.
- The Adapter must only be used with the Astral series of ventilators.

- When using the Adapter, do not touch the connections at the back of the Astral device.
- Do not connect the car battery to an AC charger while the Adapter is connected to the Astral device.
- Do not attempt to modify the Adapter, there are no user-serviceable parts inside.
- If using supplemental oxygen with the Astral device, ensure oxygen is turned off prior to connecting or disconnecting the adapter. Only use oxygen in a well-ventilated environment.

⚠️ CAUTIONS

- Always start the car before plugging in the Astral.
- The Adapter is intended for use on negative ground systems between 12V and 24V (where the negative terminal of the battery is grounded). Using it on a positive ground system may damage the Astral device and the vehicle's electrical system.

If you are unsure whether you have a negative ground system, consult the car, boat or other vehicle manual.

- If the external DC power source drops below 11V, the Astral will switch to internal battery.

Notes:

- When powered from the Adapter, the internal battery of the Astral device will charge. The Adapter will not charge the Astral External Battery.
- The vehicle's electrical system must be in good condition. Have it checked by an auto electrician if in doubt.
- If you notice any unexpected changes in the device, signs of degradation that affect performance or if the enclosure is broken, discontinue use and contact your Health Care Provider.
- For any serious incidents that occur in relation to this device, these should be reported to ResMed and the competent authority in your country.

Using the Adapter

Refer to the illustration at the beginning.

1. Plug the connector end of the Adapter into the DC input socket of the Astral device.
2. Start the car.
3. Plug the adaptor plug end of the Adapter into the vehicle DC socket.
4. The light on the Adapter will glow green when the connection is made. The DC mains LED Indicator on the Astral device will also be illuminated.
5. Power on the Astral device by pressing the green power on/off switch at the back of the device.

Note: The Therapy on/off LED indicator on the Astral device will display a constant green when the Astral device is powered on but not ventilating.

Cleaning

If necessary, wipe the outside of the Adapter with a clean cloth.

Technical specifications

Length (connector to connector): 1.8 m

Fuse rating: 10A , 250 V (fast blow)

Input: 12/24 V --- 7.5/3.75 A

Output: 12/24 V --- 7.5/3.75 A

Typical power consumption: 70 W

Maximum power consumption: 90 W continuous / 120W peak

Environmental conditions:

Refer to the Astral 100/150 User Guide

Electromagnetic compatibility:

Refer to the Astral 100/150 User Guide

Storage and transport temperature:

-30°C to 70°C

Symbols

The following symbols may appear on your device or packaging:

Indicates a Warning or Caution;

Manufacturer; European

Authorized Representative; Serial

number; Catalogue number;

CE Labelling; Underwriters

Laboratories; Medical device;

Importer; Temperature limitation.

See symbols glossary at
ResMed.com/symbols.

Environmental Information

The Astral DC Adapter should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on

natural resources and prevent hazardous substances from damaging the environment. If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

Limited warranty

ResMed warrants that your ResMed DC/DC Adapter shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

During the warranty period, if the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover:
a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs;
c) any damage or contamination due to cigarette, pipe, cigar or other smoke;
d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase. For product purchased in a country in the European Union ("EU") or European Free Trade Association ("EFTA"), "region" means the EU and EFTA.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty replaces all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

Visit ResMed.com for the latest information on ResMed's Limited Warranty.

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See www.resmed.com for other ResMed locations worldwide. For patent and other intellectual property information, see www.resmed.com/ip. Astral is a trademark and/or registered trademark of the ResMed family of companies. ©2022 ResMed. 278653/1 2022-06
Manufacturer: ResMed Pty Ltd