

# RESMED


# RESMED

Small

Solutions made easy



Recommended  
replacement  
frequency is every  
30 days

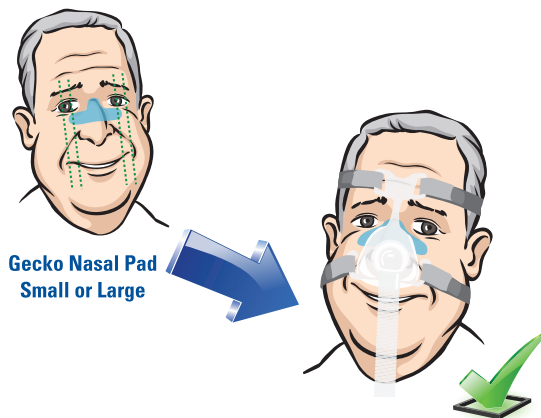

Gecko Nasal Pad  
Small or Large

Suitable for nasal and full face masks

LOT  
Batch Code

REF 61912  
Catalogue Number

-20C / -4F 60C / 140F



ResMed Pty Ltd  
1 Elizabeth Macarthur Drive,  
Bella Vista NSW 2153 Australia.  
2019-11-15



EU IMPORTER:  
ResMed European Operations BV  
Weena 327, Rotterdam, 3013 AL NL

EC REP ResMed SAS

Parc Technologique de Lyon  
292 Allée Jacques Monod  
69791 Saint Priest Cedex France

See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide. See symbols glossary at [www.resmed.com/symbols](http://www.resmed.com/symbols)  
For patent and other intellectual property information, see [ResMed.com/ip](http://ResMed.com/ip).  
© 2019 ResMed.

R619-308/4 2019-11



MEDICAL  
DEVICE

MADE IN THE USA

### To use:

1. Before each use, wash the Gecko Nasal Pad in warm water using mild soap, then rinse.
2. Place the Gecko Nasal Pad on bridge of nose as shown.
3. Fit your mask as usual.

### Intended use

Skin pressure protector device intended for single patient re-use only.

### Recommended cleaning

Wash in warm water using a mild soap. Rinse well with drinking quality water and allow to air-dry away from direct sunlight.

### Material

Polymer gel composed of USP grade mineral oil. Not made with natural rubber latex and silicone. This product and packaging do not contain any hazardous substances and may be disposed of with your normal household refuse.

### CAUTION

- Do not clean with shampoo
- Do not place in microwave
- Do not place in dishwasher
- Keep product away from open flames
- Keep product away from direct sunlight
- Do not ingest or place in mouth
- Discontinue using this product if you have ANY adverse reaction and consult your physician or sleep therapist.
- Replace this product if there is any visible deterioration.

**Note:** Any serious incident should be reported to ResMed and your country's competent authority.



Comfort is a key factor in successful therapy

Gecko™  
NASAL PAD

