Remote Alarm II

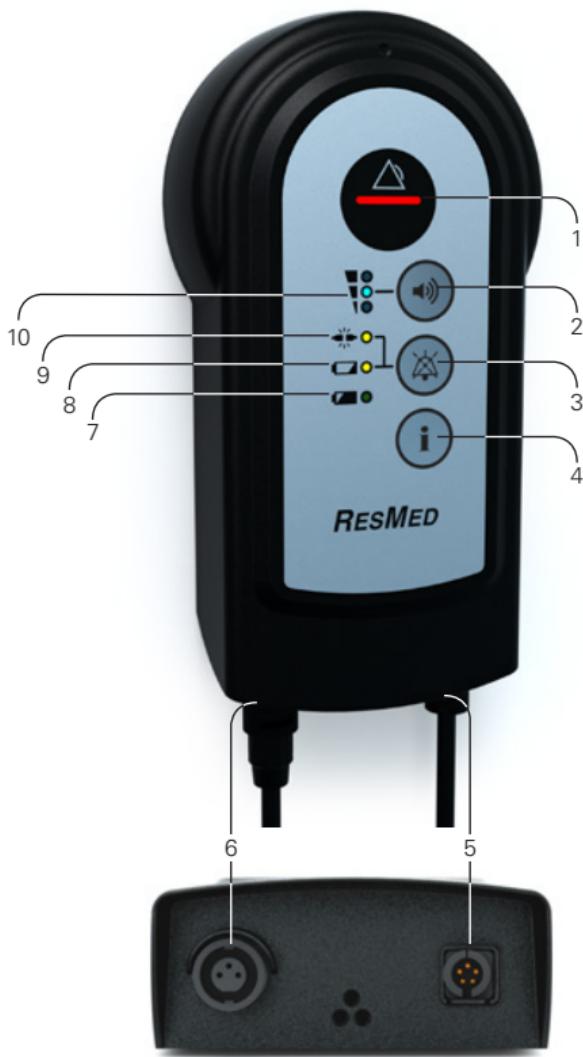
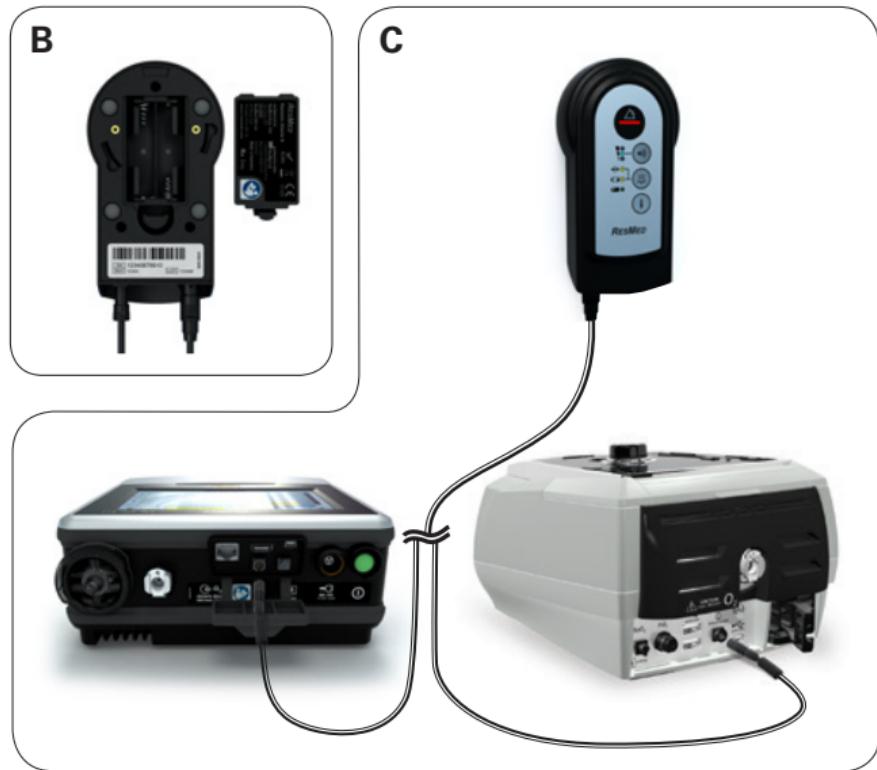
ACCESSORY

User Guide

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • Ελληνικά • Polski • Magyar • Русский • Türkçe • Česky • 简体中文 • 繁體中文 • Български • Eesti • Hrvatski • Íslenska • Lietuvių k. • Latviski • Română • Slovenčina • Slovenščina • اردو • عبرית • فارسی • العربية

Respiratory Care Solutions
Making quality of care easy

A**B****C**

D

E

RESMED

Remote Alarm II User Guide

English

The ResMed Remote Alarm II (Remote Alarm) has been designed for use with Astral and Stellar ventilators.

Read the entire manual before using the Remote Alarm.

Intended use

The Remote Alarm is intended for home and hospital use. Its primary purpose is to alert the caregiver of a ventilation patient to an alarm condition on the ventilator even when the caregiver is not at the bedside or in the same room with the patient and ventilator.

The Remote Alarm generates an audible and visual signal when an alarm is triggered on the ventilator. The Remote Alarm is powered by battery and connected to the ventilator via a cable.

⚠️ WARNINGS

- The Remote Alarm must only be used for its intended use as described in this manual.
- Do not disassemble the Remote Alarm, it has no user-serviceable parts inside; if damaged or defective, it should be returned to your ResMed representative.
- Explosion hazard – do not use in the vicinity of flammable anaesthetics.
- Ensure the remote alarm is placed in a clearly visible location on a flat surface or attached to a wall using the wall bracket provided.

Notes:

- *The Remote Alarm is not intended for mobile use, such as with a wheelchair.*
- *Refer to the ventilator's manual for its indications for use.*
- *If you notice any unexplained changes in the device (Remote Alarm II), signs of degradation that affect performance or if the enclosure is broken, discontinue use and contact ResMed or your authorised ResMed representative.*
- *For any serious incidents that occur in relation to this device, these should be reported to ResMed and the competent authority in your country.*

Remote alarm at a glance

Refer to illustration A

- | | |
|-----------------------------|-------------------------------|
| 1. Alarm indicator | 6. Input connector (3 pin) |
| 2. Volume control button | 7. Good battery indicator |
| 3. Remote Alarm mute button | 8. Low battery indicator |
| 4. Test button | 9. Alarm disconnect indicator |
| 5. Output connector (5 pin) | 10. Volume level indicator |

The ResMed Remote Alarm II comprises the following components:

- Remote Alarm • Wall bracket.

Available separately:

- 2 m, 5 m, 10 m and 20 m alarm cable.

Optional accessory:

- 30 m Hospital Alarm system cable

For a full list of compatible devices for the Remote Alarm II, see the Ventilation Accessories guide on the Products page under Ventilation Devices at ResMed.com. If you do not have Internet access, please contact your ResMed representative.

Set up

Installing batteries before use

Refer to illustration B

1. Open the battery cover at the rear of the Remote Alarm by pushing in the latch and levering the cover away.
2. Insert two AA 1.5V batteries (not included).
3. Replace the battery cover, ensuring that it clicks into place. A beep will sound once the batteries are inserted.
4. Press  to check the functionality of the alarm. Refer to the *Testing the Remote Alarm* instructions.

Note: Under normal usage conditions, new high quality type LR6 (AA size) batteries should power the Remote Alarm for approximately 12 weeks.

Connecting to the ventilator

Refer to illustration C

1. Connect one end of the alarm cable to the (3 pin) input connector on the Remote Alarm.
2. Connect the other end to the (5 pin) output connector located at the rear of the ventilator. Once connected, the alarm will beep and the Green Battery Good LED will blink once every 10 seconds.
3. Once set up is complete, press  to check the functionality of the Remote Alarm. Refer to the *Testing the Remote Alarm* instructions.

CAUTION

The Remote Alarm cable is equipped with a push-pull locking connector. To remove the cable from the ventilator, gently pull on the connector. Do not twist its outer housing.

When an alarm is activated

1. Check the patient's status.
2. View the alarm message in the Alarm display on the Ventilation device.
3. Follow the on-screen help (if available) or refer to the Alarm Troubleshooting section in the User Guide that came with your Ventilation device.

Connecting a second Remote Alarm

Refer to illustration D

If required, a second Remote Alarm can be attached to the first Remote Alarm to enable a Remote Alarm to be placed in two separate rooms or locations.

1. Connect the Remote Alarm as per the instructions in *Connecting to the ventilator*.
2. Connect one end of the alarm cable to the output connector (5 pin) of the first Remote Alarm.
3. Connect the other end into the input connector (3 pin) on the second Remote Alarm.
4. Once set up is complete, press  on the first alarm to check the connection is correct and the second alarm is receiving the signal.
5. Press  to check the functionality of the second Remote Alarm. Refer to the *Testing the Remote Alarm* instructions.

Connecting to a Hospital Alarm System

1. Connect the Remote Alarm as per the set up instructions in *Connecting to the Ventilator*.
2. Connect one end of the 30m Hospital Alarm system cable to the output connector (5 pin) of the remote alarm.
3. Connect the other end to the Hospital Alarm Management system. (Refer to the instructions provided with the cable).
4. Once set up is complete, press  to check the functionality of the Remote Alarm and confirm that the alarm is heard through the Hospital Alarm Management system.

Note: Ventilators can also be connected directly to a Hospital Alarm System using the 30 m Hospital alarm system cable.

Alarm types for the Remote Alarm

Different types of alarms can be generated on the Remote Alarm. All alarms are indicated by an audible and visual signal.

Ventilator alarms

The Remote Alarm's main purpose is to relay alarm conditions on the ventilator. The ventilator alarms on the Remote Alarm alert the caregiver that an alarm has been triggered by the Ventilator and therefore must be addressed on the Ventilator itself, not by interacting with the Remote Alarm unit.

When a ventilator alarm is activated:

1. Check the patient's status.
2. View the alarm message in the Alarm display on the Ventilation device.
3. Follow the on-screen help (if available) or refer to the Alarm Troubleshooting section in the User Guide that came with your Ventilation device.

Remote Alarm alarms

- **Low Battery Alarm**

This alarm alerts the caregiver that the batteries of the Remote Alarm are low. The batteries need to be replaced.

- **Disconnect alarm**

This alarm alerts the caregiver that the Remote Alarm has been disconnected from the Ventilator.

Check the patient and ventilator prior to looking for the source of the disconnect.

Note: For further information regarding alarm types and indicators refer to the Indicators and alerts section in this Guide.

Indicators and alerts

The Remote Alarm provides the following indicators and buttons.

LED indicators	Status
Alarm	 Audible alert and Red flashing alarm bar. An alarm has been activated on the ventilator.
Volume	 Blue solid LED Indicates current alarm volume level.
Disconnect	 Audible alert and Yellow blinking LED. <ul style="list-style-type: none">• The Remote Alarm has been disconnected from the ventilator.• The LED will continue to blink until the situation has been resolved.
Battery level	 Audible alert and Yellow LED – will blink once every 10 seconds. <ul style="list-style-type: none">• The Remote Alarm battery is low.• The LED will continue to blink until the situation has been resolved.• The audible alarm can be muted but will sound every 4 hours until the battery is replaced.
	 Green LED – will blink once every 10 seconds The Remote Alarm battery is good.
Alarm button	
	Volume level button Press until desired volume level is achieved. Refer to <i>Setting the volume</i> instructions.
	Remote Alarm mute button Press the Remote Alarm mute to silence the alarm generated for a Low Battery alert. Pressing the Remote Alarm mute during a disconnect will put the Remote Alarm into a power saving mode.
	Remote Alarm self-test. Press and hold to test the functionality of the Remote Alarm.

Testing the Remote Alarm

CAUTION

A test of the remote alarm should be performed prior to initial use and after every change of battery. Test the alarm periodically as per the facility policy. For dependent patients perform a test on a daily basis.

Press  on the Remote Alarm.

The following will occur:

- The alarm LED illuminates and the alarm sounds.
- The LED corresponding to the set volume illuminates.
- The Disconnect LED blinks if the alarm is not connected to the device and lights permanently if connected.
- The battery level LED corresponding to the battery level illuminates. Yellow LED if battery life is low, or green LED if battery life is good. (Replace the battery if the battery life is low).
- If a second Remote Alarm is connected, the second Remote Alarm will also sound.

Setting the volume

WARNING

When adjusting alarm volume, ensure that the alarm can be heard above ambient noise levels.

A choice of three volume levels for the alarm can be selected. The default setting is the highest level, intended for use in noisy environments. A short press will show the current volume level.

To adjust the volume level:

Press and hold the  for one second. The alarm will beep and the blue volume LEDs will blink to indicate the volume level can be changed.

Press  until you reach your desired volume level (low, medium or high).

After 10 seconds of inactivity the blue volume LEDs will stop blinking and the volume level will be saved.

Wall mounting the Remote Alarm

Refer to illustration E

Two types of screws can be used to affix the bracket to the wall. A minimum of three screws (not included) is required.

WARNINGS

- Ensure the wall bracket is securely fixed in place.
 - Ensure the wall bracket is located in an area that will not be affected by moisture.
 - When affixed to the wall bracket, ensure the Remote Alarm cord does not pose a tripping or choking hazard.
1. Align the rear of the Remote Alarm with the guide-line on the face plate of the wall bracket.
 2. Holding the Remote Alarm against the bracket, turn the Remote Alarm from  to 

English 5

Cleaning and maintenance

The Remote Alarm can be cleaned using an anti-bacterial cleaning solution such as Mikrozid AF® or CaviCide™ on a clean, non-dyed disposable cloth. For a full list of approved cleaning solutions, refer to the Technical Specifications section.

Troubleshooting

Problem/possible cause	Solution
The LED does not turn on when connected to the ventilator or when the Self-test button is pressed.	
Batteries may not be inserted or may be flat	Check that two AA batteries are inserted correctly in the rear of the alarm. If necessary, change the batteries. If the problem persists, contact your ResMed service agent.
The Remote Alarm detects a disconnect while the cable is connected.	
Poor connection	Change the cable. If the problem persists, change the Remote Alarm.
The Remote Alarm detects a disconnect while an alarm is raised on the ventilator.	
Poor connection	Change the cable. If the problem persists, change the Remote Alarm.
The second Remote Alarm is sounding an alarm with no alarm raised on the ventilator.	
Poor connection.	Check that the connecting cable is firmly plugged into the first Remote Alarm and the rear of the ventilator. If the problem persists, contact your ResMed service agent.
The Remote Alarm triggers a low battery alarm.	
Batteries are flat.	Change the batteries.

Technical specifications

Operating temperature	0°C to +40°C
Operating humidity	10% to 95% non-condensing
Operating altitude	Sea level to 3,050 m
Storage and transport temperature	-30°C to +70°C
Storage and transport humidity	10% to 95% non-condensing
Electromagnetic compatibility	This product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2:2020, for residential, commercial and light industry environments. The electromagnetic compatibility tables for these ResMed devices can be found on www.resmed.com , on the Products page under Service & Support. Click on the PDF file for your device.
Output volume	55–90 dBA (as per ISO 3744:2010)

Alarm Volume Range	The lower audio level will generate a volume: 65 dBA +/- 5 dBA The medium audio level will generate a volume: 75 dBA +/- 5 dBA The higher audio level will generate a volume: 85 dBA +/- 5 dBA
IEC 60601-1-11:2020 classification	Transit operable equipment.
Nominal dimensions (L x W x H)	145mm x 80mm x 31mm
Power supply	Two AA 1.5V Batteries (LR6 – Alkaline or FR6 – Lithium)
Weight	115g Remote Alarm only (without battery)
Housing construction	Flame retardant engineering thermoplastic
Cleaning solutions	<p>The following cleaners and disinfectants are compatible for use when cleaning the external surfaces of the Remote Alarm. They are suitable for weekly use.</p> <ul style="list-style-type: none"> • Cavicide™ * • Mikrozid® AF* • Isopropyl alcohol • Teepol Multipurpose detergent and Actichlor Plus • Bleach (1:10) (dilute Hypochlorite) <p>* Suitable for monthly use only. Always follow the manufacturers recommended cleaning instructions.</p>
Compatible devices	<p>The Remote Alarm II is compatible with the following ResMed devices:</p> <ul style="list-style-type: none"> • Stellar series of ventilators • Astral series of ventilators.
Maximum service life	<ul style="list-style-type: none"> • When the Remote Alarm is connected to the ventilator, and assuming that the ventilator does not alarm more than 1% of the time, the battery life of the Remote Alarm shall be more than 6 weeks (ambient temperature 21°C and ambient humidity not condensing) • When the Remote Alarm is connected to the ventilator in standby (not alarming), the battery life of the ventilator shall be more than 12 weeks (ambient temperature 21°C and ambient humidity not condensing)

Symbols

The following symbols may appear on your product or packaging.

 Indicates a Warning or Caution. A warning alerts you to possible injury or explains special measures for the safe and effective use of the product; a caution alerts you to possible damage to the product;  Read instructions before use;  Keep away from rain;  Drip proof;

 Temperature limitation;  Humidity limitation; Class II equipment;  Manufacturer;  European authorized representative;  European RoHS;  Canadian Standards Association;  Battery charge level;  Alarm mute;  Batch code;  Medical device;  Importer;  Unique device identifier;  Catalogue number;  Serial number;  Unlock;  Lock.

See symbols glossary at ResMed.com/symbols.

Environmental information

WEEE 2002/96/EC is a European Directive that requires the proper disposal of electrical and electronic equipment. These devices should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

Storage

Ensure that the Remote Alarm is clean and dry before storing for any length of time. Remove the batteries from the Remote Alarm before storing. Store your Remote Alarm in a dry place out of direct sunlight.

Disposal

The Remote Alarm should be disposed of in accordance with local environmental regulations.

Servicing

This device is intended to provide safe and reliable operation when operated in accordance with the instructions provided by ResMed. ResMed recommends that the device be inspected by an authorized ResMed Service Center if there is any sign of wear or concern with device function.

Limited warranty

ResMed Pty Ltd (hereafter 'ResMed') warrants that your ResMed product shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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© 2022 ResMed.
Astral and Stellar are trademarks and/or registered trademarks of the ResMed family of companies. Mikrozid is a trademark of Schülke&Mayr. Cavicide is a trademark of Metrex. For patent and other intellectual property information, see ResMed.com/ip