

## ResMed Connectivity Module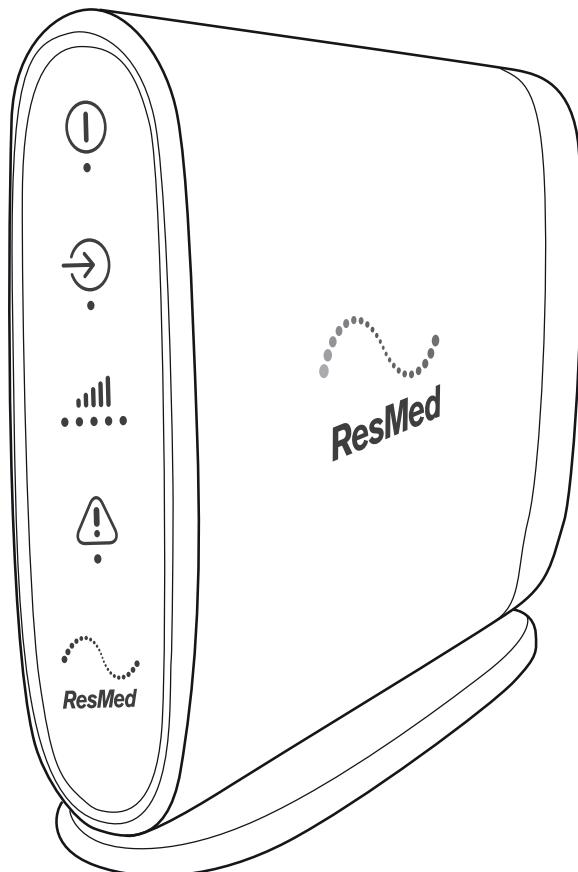

User guide  
(RCM1)


English | Français | Deutsch | Italiano | Nederlands | Svenska | Suomi | Dansk | Norsk | Español  
Português | Polski | Česky | Български | Hrvatski | Eesti | Ελληνικά | Magyar | Íslenska | Latviski  
Lietuvių k. | Română | Русский | Slovenčina | Slovenščina | Türkçe | عربى | فارسى | اردو

(A)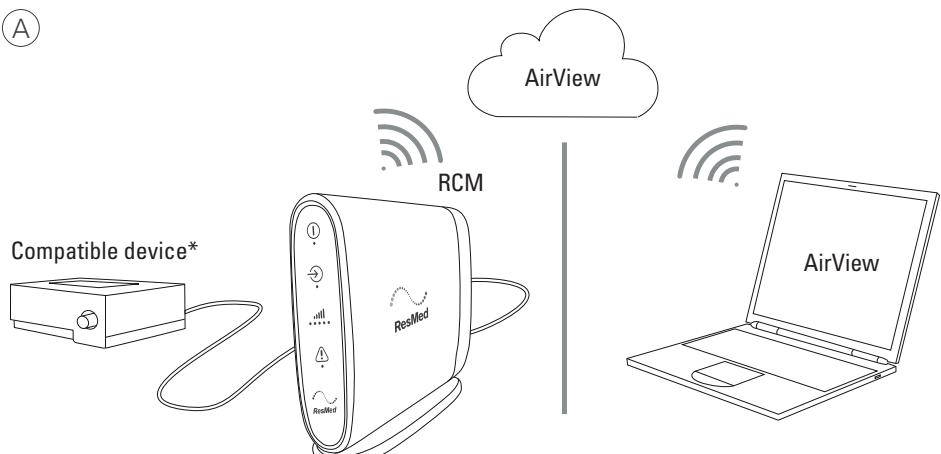


\*Dispositif Compatible / Kompatibles Gerät / Apparecchio compatibile / Compatibel apparaat / Kompatibel utrustning / Yhteensopiva laite / Kompatibelert apparat / Kompatibel enhet / Dispositivo compatible / Equipamento compatível / Zgodny urządzeniami / Kompatibilní zařízeními / Съвместимо устройство / Kompatibilan proizvod / Ühilduv seade / Συμβατή συσκευή / Kompatibilis készülék / Samhæft tæki / Saderiga ierīce / Suderinamas prietaisas / Dispozitiv compatibil / Соответствующее устройство / Kompatibilné zariadenie / Zdržliví pripomočki / Uyumlul cihaz / به اینگ تیوانس / تاکن توام / ممتغاه سازگار / الجهاز المترافق


(B)

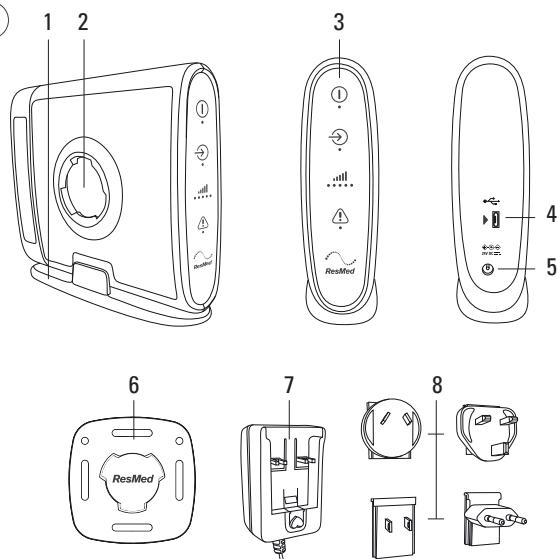


(C)

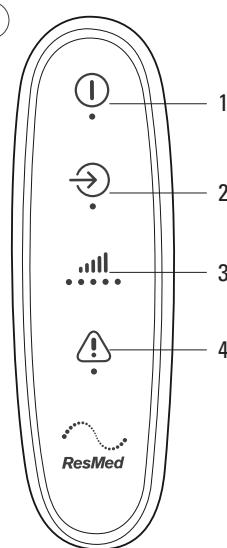(D)

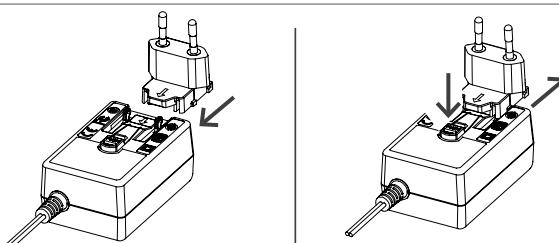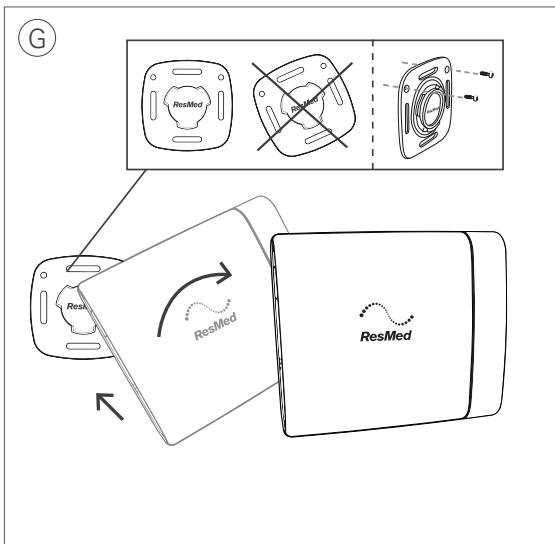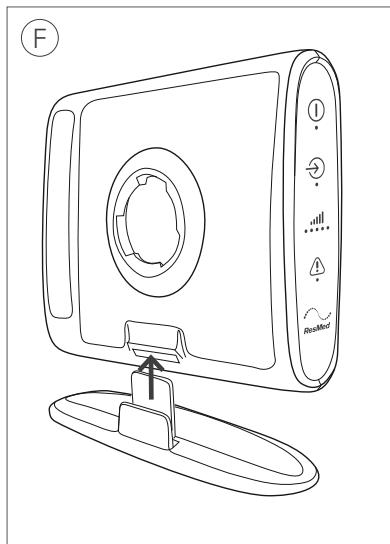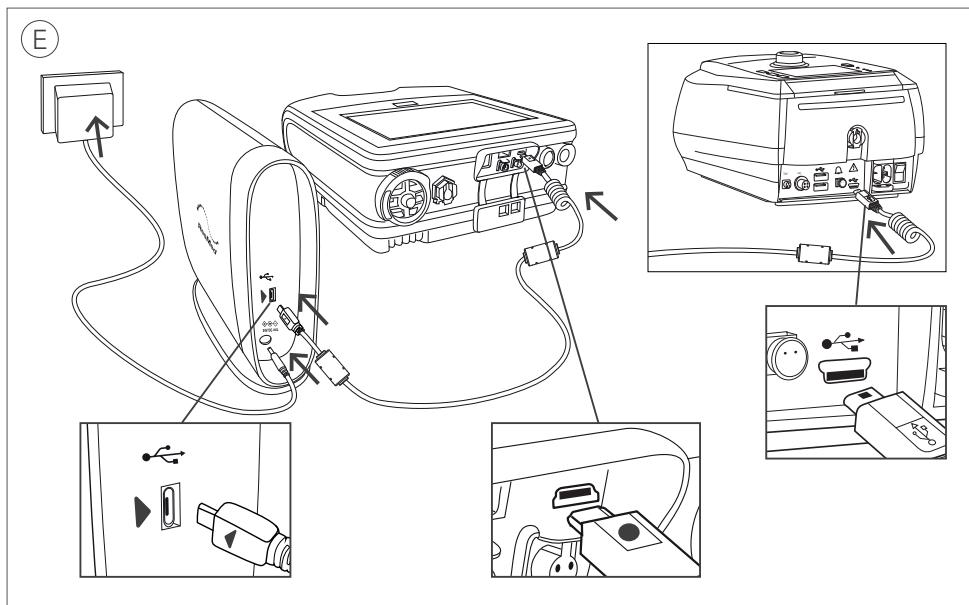
## Introduction

The ResMed Connectivity Module Hospital (RCMH) provides connection between compatible ResMed ventilation devices and hospital Electronic Medical Record (EMR) systems.

The RCMH sends therapy and device data recorded in the ventilation device to the EMR system on demand when requested by the EMR system.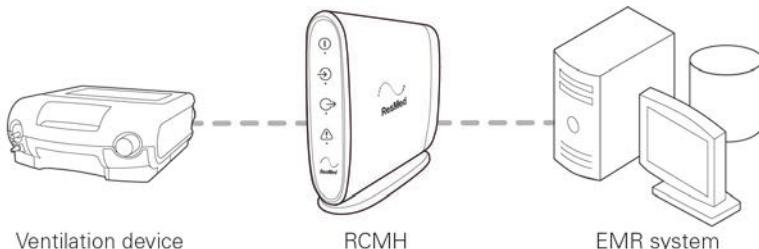

### **⚠ WARNING**

Read this entire guide before using the device.

### **⚠ CAUTION**

In the US, Federal law restricts this device to sale by or on the order of a physician.

## Compatible devices

For a full list of compatible devices for the RCMH, see the Data management/Device Compatibility List on [www.resmed.com/downloads/devices/RCMH](http://www.resmed.com/downloads/devices/RCMH). If you do not have internet access, please contact your ResMed representative.

## Intended use

The RCMH is indicated for use in the collection of therapy data from compatible ResMed ventilation devices to hospital EMR systems. It is intended for use in the hospital environment and by professional users only. The RCMH will not control any clinical device, nor provide interpretation of data.

## General warnings and cautions

The following are general warnings and cautions. Specific warnings, cautions and notes appear with the relevant instructions in the guide.

### **⚠ WARNING**

- Only use the power supply unit and plug blade attachments provided with RCMH.
- Beware of electrocution. Do not immerse RCMH or any of its components in water. Always unplug RCMH before cleaning and make sure that all parts are dry before plugging it back in.
- Do not open or modify the device. There are no user serviceable parts inside. Repairs and servicing should only be performed by an authorized ResMed service agent.
- Any change or modification to the product is not expressly approved by ResMed and could void the user's authority to operate the device.

- The RCMH device has not been tested or certified for use in the vicinity of X-ray, CT or MRI equipment. Do not bring the RCMH device within 13 ft (4 m) of X-ray or CT equipment. Never bring the RCMH device into an MR environment.
- It is the responsibility of the end user to validate the compatibility and use of information transmitted from the ResMed RCMH to the EMR hospital system.
- Connect to RCMH only items that are specified as part of or compatible with the device system. Additional equipment connected to medical electrical equipment must comply with the respective IEC or ISO standards (eg, IEC 60950 for data processing equipment). Furthermore, all configurations shall comply with the requirements for medical electrical systems (refer to IEC 60601-1). Anybody connecting additional equipment to RCMH configures a medical system and is therefore responsible for ensuring that the system complies with the requirements for medical electrical systems. Also be aware that local laws may take priority over the above mentioned requirements. If in doubt, please contact your ResMed representative.
- The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.
- The use of cables other than those specified for the device is not recommended. They may result in increased emissions or decreased immunity of the device.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12" to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.


## CAUTION

Do not use RCMH outdoors.

**Note:** For any serious incidents that occur in relation to this device, these should be reported to ResMed and the competent authority in your country.

**Note:** For assistance and reporting of issues associated with the RCM device, contact your Health Care Provider or authorised ResMed representative.

## RCMH at a glance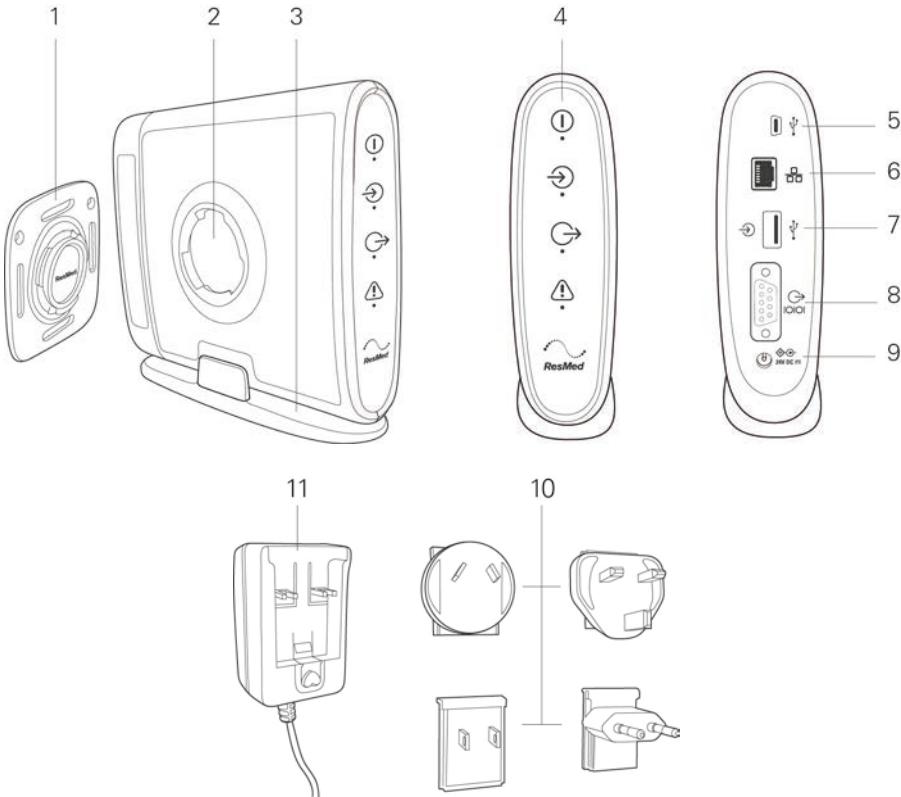


1. Wall mount
2. Wall mounting socket
3. Stand
4. Indicator panel
5. USB device port\*
6. Ethernet port\*

\*For service use only.

7. USB host port
8. RS232 port
9. Power inlet
10. Power inlet
11. Power inlet

## Indicators


The RCMH provides indication of the current operating state. When the Power, Vent. Input and Output indicators illuminate, the RCMH is operational.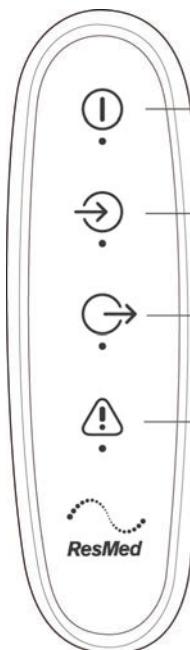

| Indicator                   | Status   |
|-----------------------------|--|
| <b>1 Power – Green</b>      | On: The power is on.<br>Off: The power is off.   |
| <b>2 Vent. Input – Blue</b> | On: Connected to the ventilation device.<br>Off: Disconnected from the ventilation device.<br>Blinking: Incompatible ventilation device software detected. |
| <b>3 Output – Blue</b>      | On: Connected to the EMR system and receiving data requests.<br>Off: No connection to the EMR system detected/no EMR data requests being received.         |
| <b>4 Error – Yellow</b>     | On: An error has occurred.<br>Off: No error  |

**Note:** All indicators will dim when the ambient light is low (at night) and will return to full brightness when the ambient light is higher (during the day).

## Assembling the PSU

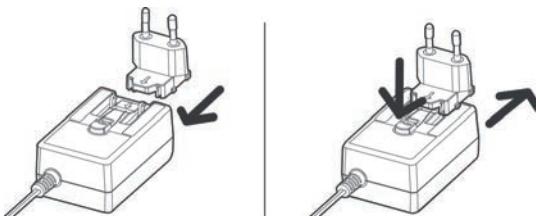
1. Insert the plug blade attachment suitable for your region into the PSU.
2. To remove the plug blade attachment from the PSU, press the button under the arrow and slide it out.

### **⚠ CAUTION**

- Do not leave the plug blade attachment in a power outlet alone.
- Do not plug the PSU upside down into a power outlet. Ensure that the power cord extends downward.

## Using the stand/wall mount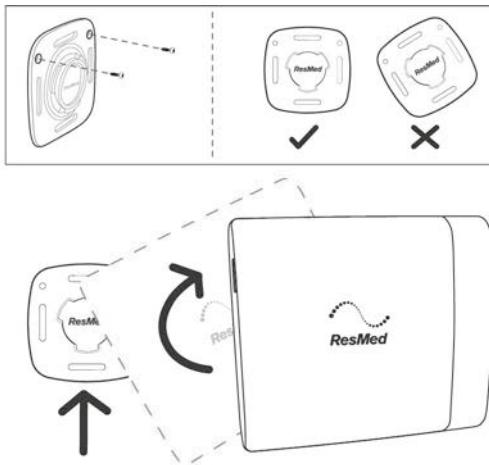

### Stand

1. Place the stand on a stable level surface.
2. Insert the longer edge of the stand into RCMH, ensuring that it clicks into place.
3. To remove the stand, release the clip on the stand.

### Wall mount


Use appropriate fittings to attach the wall mount. For example, use the round holes or the slots of the wall mount for screws or cable ties (not provided).

#### **WARNING**

Ensure that the wall mount is securely fixed in place.

1. Hold RCMH at an angle and push it onto the wall mount. Adjust the angle to fit RCMH in place.
2. Turn RCMH clockwise until it clicks into place.
3. To remove RCMH from the wall mount, turn it counterclockwise.

## Setting up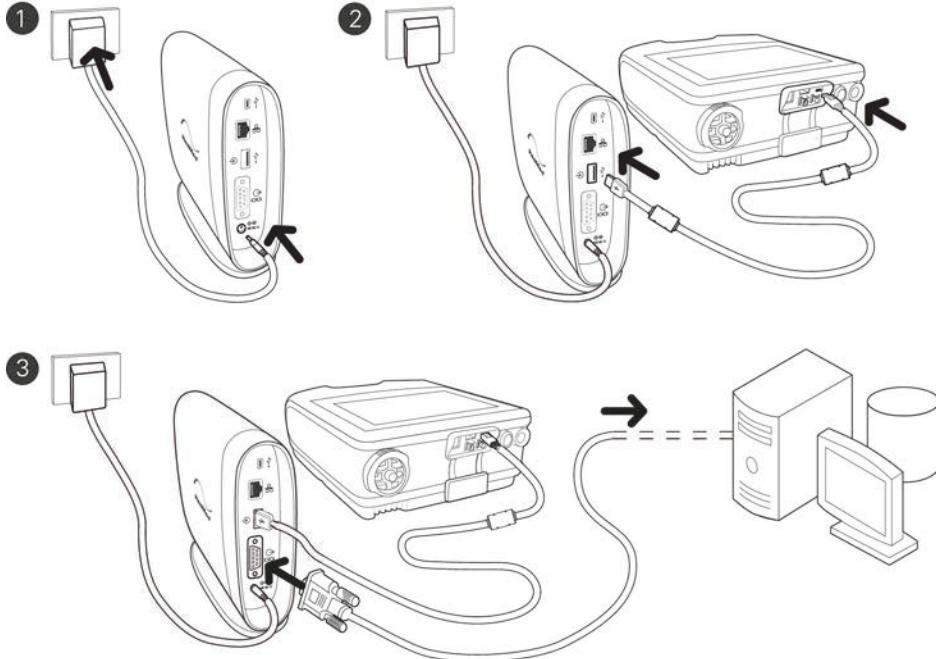

RCMH can only be connected to one ventilation device at a time.

Ensure that the RCMH is secured with the stand or wall mount. Refer to the Using the stand/wall mount section.

1. Connect the RCMH to the power using the PSU. Ensure that the **Power** indicator illuminates.
2. Connect the RCMH to a powered-on ventilation device by plugging in one end of the USB cable to the USB host port  and the other end to the USB port at the rear of the ventilation device. Ensure that the **Vent. Input** indicator illuminates.
3. Connect the RCMH to the EMR system via the RS232 port  **IOIOI**. The **Output** indicator will illuminate when RCMH receives data requests from the EMR system. EMR system configuration may be required to initiate data requests. Refer to the EMR system guide for EMR system configuration details.

For further assistance, contact your care provider or ResMed representative.

## Turning off the RCMH

To stop the RCMH, unplug the power cord from the power outlet.

## Cleaning and maintenance

The exterior of RCMH and the PSU can be cleaned with a damp cloth and an approved mild cleaning solution.

The following cleaners and disinfectants are compatible for use when cleaning external surfaces of RCMH:

- Actichlor™ Plus\*
- Isopropyl alcohol\*
- Bleach (1:10)\* (may also be known as 'diluted hypochlorite')
- Cavicide™\*\*
- Mikrozid™\*\*

\*Suitable for weekly use (260 cleaning cycles over design life)

\*\*Suitable for monthly cleaning only (60 cleaning cycles over design life)

Always follow the manufacturer's recommended cleaning instructions.

**Note:** In the event that you notice any unexpected changes in the device or signs of degradation that affect performance, discontinue use and contact your healthcare provider.

## ⚠ WARNING

Ensure that the RCMH and PSU are dry before reconnecting to the power outlet and ventilation device.

## Troubleshooting

| Problem/possible cause                         | Solution  |
|--|---|
| The Power indicator does not illuminate.       | <p>Check that the PSU is connected correctly to the power outlet and the rear of RCMH.</p> <p>Check that the plug blade attachment is inserted into the PSU correctly.</p> <p>Check that the PSU is the one provided with RCMH.</p>                             |
| The Vent. Input indicator does not illuminate. | <p>Check that the ventilation device is turned on.</p> <p>Check that the USB cable is connected correctly to the rear of RCMH and the ventilation device.</p> <p>Turn the RCMH off for at least 5 seconds, then on again.</p>                                   |
| The Vent. Input indicator is blinking.         | Upgrade the ventilation device software. Refer to the Data management/Device Compatibility List on <a href="http://www.resmed.com/downloads/devices/RCMH">www.resmed.com/downloads/devices/RCMH</a> .   |
| The Output indicator does not illuminate.      | <p>Change the connection from the RS232 port of the RCMH.</p> <p>Ensure the installation instructions for the EMR system have been followed, and the EMR system is sending data requests to RCMH. For more details, refer to the specific EMR system guide.</p> |
| The Error indicator is on.                     | Turn the RCMH off for at least 5 seconds, then on again, to see if this removes the error. If the Error indicator displays again, contact your care provider or ResMed representative.  |

If the problem cannot be solved, contact your care provider or ResMed representative.


## Technical specifications

| Dimensions (H x W x L)             | RCMH only: 134 mm x 44 mm x 150 mm<br>The stand will add 3 mm to the height and 6 mm to the width.  |        |                      |
|------------------------------------|---|--------|----------------------|
| Weight                             | RCMH only: 320 g<br>The stand will add 30 g and the wall mount will add 10 g.   |        |                      |
| Power supply unit (PSU)            | AC 100–240 V, 0.35–0.70 A, 50–60 Hz<br>DC 24 V, 1.25 A<br>Cable length 1.8 m<br>Class II, suitable for continuous operation<br>Maximum power consumption: <7W   |        |                      |
| Housing construction               | Flame retardant engineering thermoplastic and silicone  |        |                      |
| Environmental conditions           |   |        |                      |
| Operating temperature:             | +0°C to +40°C   |        |                      |
| Operating humidity:                | 10%–95% non-condensing  |        |                      |
| Operating altitude:                | Sea level to 3000 m   |        |                      |
| Storage and transport temperature: | -25°C to +70°C  |        |                      |
| Storage and transport humidity:    | 10%–95% non-condensing  |        |                      |
| Electromagnetic compatibility      | Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2:2014, for residential, commercial and light industry environments. It is recommended that mobile communication devices are kept at least 30 cm away from the device.<br>Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on <a href="http://www.resmed.com/downloads/devices">www.resmed.com/downloads/devices</a> . |        |                      |
| IEC 60601-1 classification         | Class II, continuous operation, suitable for use with oxygen, IP21 (Protected against finger sized objects and against vertically dripping water)   |        |                      |
| Design life                        | 5 years<br>The RCMH and PSU do not contain any serviceable parts.   |        |                      |
| Available data                     | For a list of available data for use by the EMR system, please contact your ResMed representative.  |        |                      |
| RS-232 port                        | The RCMH has a RS-232 digital data interface accessed via a female 9-pin D-sub connector on the RCMH rear panel. The RS-232 interface features a standard DCE pinout enabling direct connection to DTE device serial ports via a straight (one-to-one) cable.<br>The RS-232 interface has pin assignment and description per the table below.<br>The interface provides 1.5kV of isolation from connected ResMed ventilator DC levels.  |        |                      |
| Pin no.                            | Signal name   | I/O    | Description          |
| 1                                  | Reference OV  | -      | Reference signal OV  |
| 2                                  | TXD   | Output | Transmit data output |
| 3                                  | RXD   | Input  | Received data input  |
| 4                                  | N/C   | -      | Not connected        |
| 5                                  | Reference OV  | -      | Reference signal OV  |
| 6                                  | Reference OV  | -      | Reference signal OV  |
| 7                                  | CTS   | Input  | Not used             |
| 8                                  | RTS   | Output | Not used             |
| 9                                  | Reference OV  | -      | Reference signal OV  |

Note: I/O direction below is with respect to the RCMH.

---

The position of the signal pins within the connector on the RCMH rear panel are as shown in the diagram below.

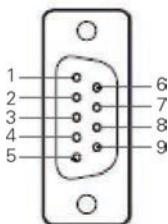The RCMH RS-232 data interface serial protocol is configured as follows:

|               |              |
|---------------|--------------|
| Baud rate:    | 115.2 kbit/s |
| Data Bits:    | 8            |
| Parity:       | None         |
| Stop Bits:    | 1            |
| Flow Control: | None         |

---

**Note:** The manufacturer reserves the right to change these specifications without notice.

## Symbols

The following symbols may appear on the product or packaging:

- Follow instructions for use. Manufacturer. European Authorized Representative.
- Batch code. Catalog number. Serial number. Direct current.
- Humidity limitation. Temperature limitation. Prescription only (In the US, Federal law restricts these devices to sale by or on the order of a physician). European RoHS.
- Protection against insertion of fingers and against vertically dripping water. Keep dry.
- Fragile, handle with care. RCM certification. Canadian Standards Associations.
- USB connector. Power indicator. Vent. Input (Ventilator Input) indicator. Ethernet port. Output indicator. RS232 port. MR unsafe. PSE mark.
- Polarity of DC power connector. Indicates a warning or caution. Medical device. Importer.

See symbols glossary at [ResMed.com/symbols](http://ResMed.com/symbols).

### Environmental information

This device should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to [www.resmed.com/environment](http://www.resmed.com/environment).

## Limited warranty

ResMed Pty Ltd (hereafter 'ResMed') warrants that your ResMed product shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This Limited Warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organization that has not been expressly authorized by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; and d) any damage caused by water being spilled on or into an electronic device.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty replaces all other expressed or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have resulted from the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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© 2023 ResMed. 278651/2 2023-04
See ResMed.com for other ResMed locations worldwide. AirView, Astral and Stellar are trademarks and/or registered trademarks of the ResMed family of companies. For patent and other intellectual property information, see ResMed.com/ip.