# ResMed

# Stellar™ series

## DC/DC Converter 24V/90W

User Guide

English | Deutsch | Français | Italiano | Español | Português | Nederlands | Svenska | Dansk | Norsk | Suomi | 简体中文 | 繁體中文 | 日本語

### ENGLISH

The DC/DC Converter 24V/90W allows you to operate the Stellar device from a 12V or 24V DC power source in a car, boat, or other vehicle equipped with a suitable battery. Consult your equipment supplier for more information about this accessory.

The H4i in heating mode can only be used when the device is connected to mains supply. The Stellar internal battery will not charge when powered from a DC power source.

### ⚠️ WARNINGS

- Read instructions before use.
- Only use the converter when connecting to a 12V or 24V DC supply.
- Do not connect the Stellar device to a 12V or 24V DC power source while the power source is being charged from AC mains.
- Do not attempt to modify the converter. There are no user-serviceable parts inside the converter.

### ⚠️ CAUTIONS

- The converter must only be used with Stellar devices to ensure proper performance of the device.
- The converter is intended for use on 12V or 24V negative ground systems (where the negative terminal of the battery is grounded). Using it on a

12V or 24V positive ground system may damage the Stellar device and the vehicle's electrical system.

If you are unsure whether you have a negative ground system, consult the car, boat or other vehicle manual.

- Make sure the converter is not covered during use to avoid overheating the device.
- The converter is not intended for use on an aircraft.

### Using the converter via a cigarette lighter

Refer to the illustration 1.

1. Before connecting the converter, ensure the switch **(A)** is off.
2. Position the converter on a level surface, with the ResMed logo facing up.
3. Plug the connector end **(B)** of the converter into the DC input socket **(C)** of the Stellar device.
4. Remove the cigarette lighter from the vehicle socket.
5. Plug the cigarette lighter adaptor plug **(D)** on the converter into the vehicle cigarette lighter socket.
6. The light **(E)** on the converter will glow green when the connection is made and the switch is on. The Stellar device should then power on.  
The light will flash continuously if the connection is not made, or if the device is in standby mode or power

save mode when the connection is made. If the battery is low, the light will turn amber. If the light does not operate, check that the switch on the cigarette lighter adaptor plug is on.

#### Notes:

- Some vehicles require the ignition key to be turned to the "accessories" position to provide power to the cigarette lighter socket.
- If you connect the Stellar device to a powered converter, the converter may power down. To resume power, turn the converter off and on again via the switch on the cigarette lighter adaptor plug.
- If the battery AND car, boat or other vehicle are not in good condition, there may not be enough reserve left in the battery to start the vehicle after use. The Stellar device will operate using the internal battery. Remove the converter.
- ResMed does not recommend frequent use of a standard vehicle battery to power the Stellar device. A standard vehicle battery is not optimized for frequent heavy discharges. Contact ResMed Technical Services for further information and recommendations.
- If the converter cuts power to the Stellar device when starting your car, boat or other vehicle, disconnect the converter from the vehicle, wait a few seconds, and then reconnect it.
- Do not leave the converter switched on and connected to the power source when the Stellar device is not in use, as this may discharge the battery over time. Ensure the DC converter is switched off when not in use.
- You must have an electrical system in good condition. Have it checked by an auto electrician if in doubt.
- Contact ResMed or a representative if you need help.

## Connecting a converter to a battery via battery clips

Refer to illustration 2.

An adapter cable is required if you need to connect directly to the battery terminals (e.g. if the battery is out of the vehicle).

For more information, see the battery guide available on [www.resmed.com](http://www.resmed.com) on the Products page under Service & Support.

## Technical specifications

Length (connector to connector): 1.6 m

Fuse rating: F 20A L, 250 V (fast blow, low breaking capacity)

Input: 12 V/24 V --- 13A/6.5A

Output: 24 V --- 3.75A

Typical power consumption: 70 W

Maximum power consumption: 110 W

### Environmental conditions:

Operating temperature: +5°C to +35°C;

Operating humidity: 10 to 95% non-condensing;

Operating altitude: Sea level to 2,591 m;

Air pressure range 1013 hPa to 738 hPa;

Storage and transport temperature: -20°C to +60°C;

Storage and transport humidity: 10 to 95% non-condensing.

### Electromagnetic compatibility:

Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for residential, commercial and light industry environments.

Ensure that wireless communications devices (such as mobile phones, wireless home network devices, cordless telephones and their base stations, and walkie-talkies) are kept at least 3.2 ft (1 m) away from the device.

Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on [www.resmed.com](http://www.resmed.com), on the Products page under Service and Support.

## Symbols which may appear on the device

Caution; Class II equipment; IP21 Protection against insertion of fingers and against vertically dripping water; Direct Current; Manufacturer; EC REP European Authorized Representative; RoHS European RoHS; SN Serial number; China pollution control; Fuse;

Environmental Information The DC/DC converter should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment. If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to [www.resmed.com/environment](http://www.resmed.com/environment).

## Limited warranty

ResMed warrants that your ResMed DC/DC Converter 24V/90W shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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©2018 ResMed. 248725/2 2018-03
See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide. For patent information, see [www.resmed.com/ip](http://www.resmed.com/ip). Stellar, H5i and ClimateLine are trademarks of ResMed Pty Ltd and Stellar and ClimateLine are registered in U.S. Patent and Trademark Office.
**ResMed Pty Ltd**