







































Icon Guide

English

Icon	Description
Diagnostic test status¹	
	Patient undergoing home sleep test.
	Diagnostic data available for physician to review.
	Diagnostic data reviewed by the physician; electronically-signed report is available.
	Interpretation has been completed and awaiting prescription for therapy. ²
	Prescription completed and can be on several stages (Ready to send, Results sent, Patient accepted, or Patient returned). ²
Monitoring type	
	Referred diagnostic patient awaiting setup ²
	Patient not wirelessly monitored
	Data downloaded via SD card
	Patient wirelessly monitored
	Remote Assist only ²
Dashboard notification	
	Patient's prescription changed
	Notes added to a patient file
	Diagnostic report ²
	Prescription for therapy report ²
Patient compliance status	
	Compliance conditions met
	Compliance conditions not met

Icon	Description	
Daily compliance status		
Icon legend:		Top—AHI / Bottom—Leak - Below threshold (green) - Above threshold (yellow) - Data not supported (white) Overall colour—Usage - Above threshold (green) - Below threshold (red)
Usage above threshold	Usage below threshold	
		
		High leak
		Leak data not supported
		High AHI
		AHI data not supported
		High AHI, leak data not supported
		High leak, AHI data not supported
		High AHI, high leak
		AHI, leak data not supported
		Device not used
		Monitoring—no data received yet
		Not monitoring—no data available
		Device has fault—no data available

¹ If a patient has not moved to the next status after 7 days, the icon will change into red. When the next step is completed, the icon will change back to green.

² Feature not available in all regions.



Icon	Description
myAir status³	
	Patient registered in myAir
	Patient invited to myAir
	Patient not registered in myAir
	Patient's myAir status is unknown
Care Check-In status⁴	
	Patient status available with positive responses
	Patient status available with negative responses
	Patient submitted a neutral or positive response
	Patient submitted a negative response
Ventilation patient status⁵	
	Value within threshold—notification not required
	Value above threshold—notification conditions met
	Value below threshold—notification conditions met
	Not monitoring—no notification
	No data available

^{3, 4, 5} Feature not available in all regions.

ResMed Corp
9001 Spectrum Center Boulevard
San Diego CA 92123 USA

0123

ResMed.com

DISTRIBUTED BY
1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia
[EC|REP] ResMed SAS Parc Technologique de Lyon 292 Allée Jacques
Monod 69791 Saint Priest Cedex France

See www.resmed.com for other ResMed locations worldwide.
For patent and other intellectual information, see www.resmed.com/ip
AirView and myAir are trademarks and/or registered trademarks of the
ResMed family of companies.
228786/9 2021-06