

Software release notes

English

ResScan Essentials 1.2

Product description

ResScan Essentials is designed to be used as a data viewer that enables clinicians and healthcare professionals to download, review, report, and share retrospective therapy information recorded by compatible Resmed therapy devices.

What's new

- Added support for AirStart™ 10 APAP and AirCurve™ 10 CS-A TJ devices in select regions.
- Improved visibility of patient lists with a wider name column and shorter rows. Downloads now display in order from most recent to oldest to help you find data faster.
- Added more summary settings, including **Rise Enable**, **Rise Time**, **Trigger** and **Cycle** settings for supported devices.
- Reports now use consistent default names, and the application checks for duplicate report names to prevent overwrite.
- IT administrators can now change the network ports used by ResScan Essentials.
 1. Open the installation folder.
For example, `C:\Program Files\ResMed\ResScan Essentials\resources\bin\electron.manifest.json`.
 2. Open the `electron.manifest.json` file.
Administrator access is required to edit the file.
 3. Set the `electronPort` value.
For example, `electronPort:9000`.
 4. Set the `aspCoreBackendPort` value.
For example, `aspCoreBackendPort:9001`.
 5. Save the file.
- Added language support for Japanese, Korean, Chinese (Simplified), Chinese (Traditional), Greek, Slovenian, Latvian, Hungarian, Bulgarian, Croatian, French (Canada), Portuguese (Brazil), Spanish (LATAM) and US English.
- Fixed several bugs and improved overall stability.

Recommended system requirements

- Microsoft Windows® 10 or Windows 11 operating system
- **Network:** 10/100/1000 Mbps
- **Storage:** 700 MB needed for installation, with additional storage required for data
- **USB:** USB A port (or USB A adapter)
- Google Chrome® or Microsoft Edge® browser
- A portable document reader, such as Adobe Acrobat Reader® software

Compatible devices

For a complete list of compatible devices, see the Data management/Device compatibility list on the Resmed website.

Support materials

- ResScan Essentials Clinical Guide
- ResScan Essentials GDPR Guide
- Data management/Device compatibility list

Supported languages

Region	Language	
Americas	• French (Canada)	• Spanish (LATAM)
	• Portuguese (Brazil)	• US English
APAC	• Chinese (Simplified)	• Japanese
	• Chinese (Traditional)	• Korean
Europe	• Croatian	• Italian
	• Bulgarian	• Latvian
	• Danish	• Norwegian
	• Dutch	• Portuguese (Portugal)
	• Finnish	• Romanian
	• French (France)	• Slovenian
	• German	• Spanish (Spain)
	• Greek	• Swedish
	• Hungarian	• UK English

Software distribution

You can download ResScan Essentials from the Resmed website.

Product code

23201(Web)

Still having problems?

Contact support in your region. For more information, see [Resmed.com/en-us/country-selector/](https://resmed.com/en-us/country-selector/).



Made in USA

Manufacturer: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA

ResScan Essentials, AirStart and AirCurve are trademarks of the Resmed family of companies. For patent and other information, see www.resmed.com/ip. Adobe Acrobat Reader is a registered trademark of Adobe Inc. Chrome is a trademark of Google LLC. Microsoft, Windows and Microsoft Edge are trademarks of Microsoft Corporation. All other trademarks are the property of their respective owners. 238171/1 2026-05