
Software release notes

AirMini™ by ResMed App 1.6.0

Product description

AirMini by ResMed is a mobile application used by patients and clinicians. Patients can operate their prescribed ResMed AirMini device, view collected data and change comfort settings. Clinicians can also operate the device and change the patient's prescription settings.

What's new

- We've added regulatory labels to the AirMini App on the device label page to comply with Medical Device Regulation (MDR) standards.

Deferred issues

- **Problem:** If a user navigates to **More** and taps on **Send feedback**, the app freezes after the email window appears.
Solution: Users can close and reopen the AirMini app to restore functionality.
- **Problem:** If a user navigates to the **AirMini App User Guide** or **AirMini Machine User Guide**, the dialogue box that confirms a secure connection does not appear.
Solution: At present, this problem is unresolved.
- **Problem:** AirMini app data did not upload to AirView™ for a few Android™ users in mid-March 2021.
Solution: At present, this problem is unresolved.
- **Problem:** After users install the AirMini app and try to pair with their machine via Bluetooth™, the user may select **Scan code** or **Enter a four-digit code** to verify their Bluetooth connection. The **Scan code** option crashes the app.
Solution: Users should **Enter a four-digit code** to verify their Bluetooth connection.
- **Problem:** After Android OS 11 users install the AirMini app and pair with their machine via Bluetooth, users are prompted to grant access to contacts and call history.
Solution: Users should not grant access; this data is not required by the AirMini app.

Supported operating systems

- Android operating system version 4.4 and newer
- iOS 10.2 and newer

Compatible machines

- The AirMini by ResMed app is compatible with the AirMini machine.

Supported languages

- English (US)
- French (France)
- German (Germany)
- Czech
- Dutch
- Finnish
- Italian
- Japanese

- Korean
- Norwegian
- Polish
- Portuguese (Brazil)
- Spanish (LATAM)
- Swedish
- Indonesian

Still having problems?

Contact support:

AirMiniAppSupport@resmed.com



ResMed Ltd

1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

See ResMed.com for other ResMed locations worldwide. Android is a trademark of Google Inc. iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license. Bluetooth is a registered trademark of Bluetooth SIG, Inc. AirMini and AirView are trademarks and/or registered trademarks of the ResMed family of companies. For patent and other intellectual information, see www.resmed.com/ip.

© 2022 ResMed Ltd. 108294/1 2022-11