**ResMed**

**Lumis™**

HFT

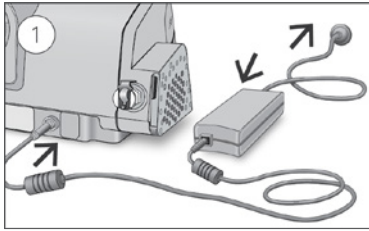
Quick start guide


English | Deutsch | Français | Italiano | Nederlands | Español | Português

Read your entire Lumis™ HFT User Guide before use.


## Setup

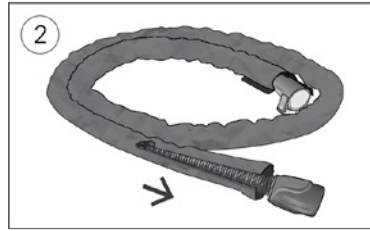With the device on a stable level surface below head height, connect one end of the power cord into the power supply unit and the other end into the power outlet.

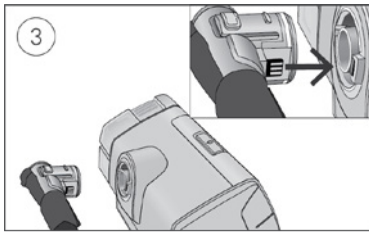
Fit the tubing wrap around the air tubing.


Connect the air tubing firmly to the air outlet located on the rear of the device.

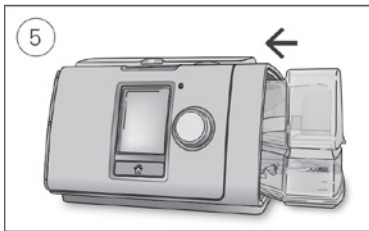Open the humidifier and fill it with water up to the maximum water level mark.

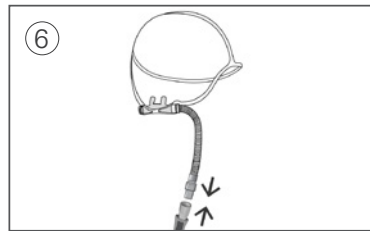Close the humidifier and insert it into the side of the device.

Connect the free end of the air tubing firmly onto the high-flow nasal cannula.

**Note:** It is recommended to place the device on a heat resistant pad to avoid possible heat damage to surfaces.

# Connecting/disconnecting supplemental oxygen

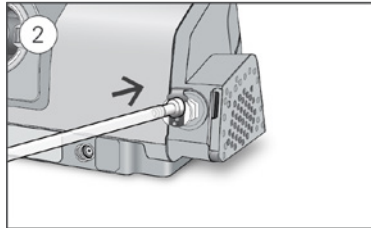
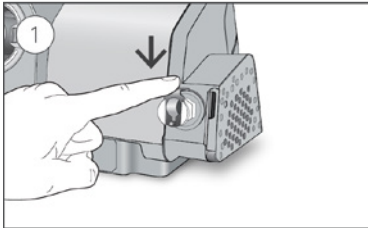
Your Lumis HFT device is designed to be compatible with levels of supplemental oxygen up to 15 L/min.

Always make sure that the device is turned on and airflow generated before the oxygen supply is turned on. Always turn the oxygen supply off before the device, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.



**Note:** Do not connect supplemental oxygen input directly to a medical gas pipeline system, oxygen bottle or other high pressure oxygen source. A flow regulator must be fitted to ensure the delivered oxygen remains at or below 15 L/min.

For advice on connecting to an oxygen source, contact your homecare provider.

## Connecting

## Disconnecting

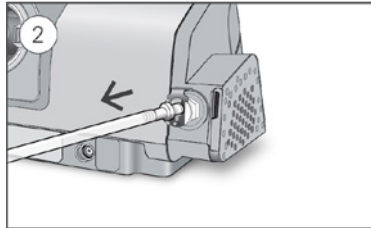
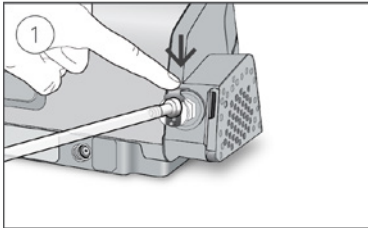
Refer to the Lumis HFT User Guide for warnings and cautions and details on how to perform a functional check.

# Setting therapy

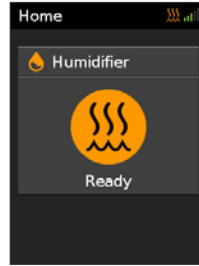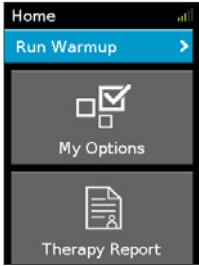
Note: Your clinician should have applied your prescribed settings to your device.

## Warmup



1. Select Run Warmup.
2. Warmup will take between 10-20 mins. Therapy can be started at any time during Warmup.
3. When Warmup is complete, Ready will be displayed.

## Start therapy

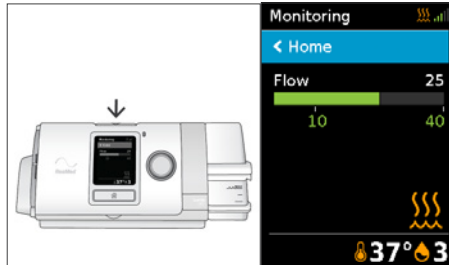4. Fit your high-flow nasal cannula.
5. Press the Start/Stop button to start therapy.
6. If you have supplemental oxygen, ensure source is now turned ON.

## Stop therapy

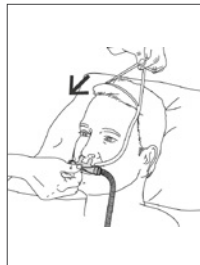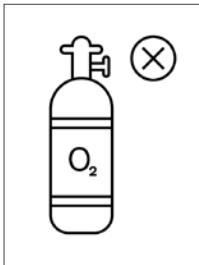1. Ensure oxygen source is turned OFF.
2. Remove your High Flow Nasal Cannula.
3. Press the Start/Stop button and then press the dial to confirm Yes to stop therapy.
4. Drying mode will commence.

**Warming mode** pre-heats the water before starting therapy so that the air is not cold or dry at the start of therapy.

**Drying mode** runs for 90 mins and dries the air tubing and cannula so that it is ready for use the next time. Ensure that the air tubing and cannula remain connected to the device and air flow unimpeded during the Drying mode.

## Caring for your device

Refer to the Lumis HFT User Guide for a detailed cleaning and maintenance information.

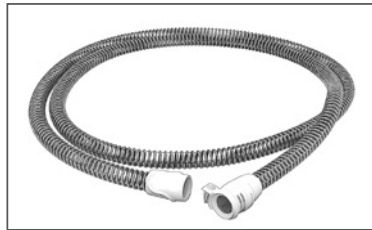
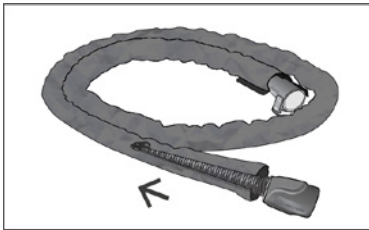
### Cleaning

You should clean the device weekly as described. Refer to the nasal cannula user guide for detailed instructions on cleaning your nasal cannula.

1. Wash the humidifier and air tubing in warm water using mild detergent.
2. Wash the tubing wrap in cold to warm water using a mild detergent. Rinse well and drip dry.
3. Rinse the humidifier and air tubing thoroughly and allow to dry out of direct sunlight and/or heat.
4. Wipe the exterior of the device with a dry cloth.

### Checking – tubing and tubing wrap

Remove the tubing wrap and check the air tubing is clean. Replace it if there are any holes, tears or cracks.

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© 2021 ResMed. 288389/2 2021-06
See ResMed.com for other ResMed locations worldwide. Lumis is a trademark and/or registered trademark of the ResMed family of companies. For patent and other intellectual property information, see [www.resmed.com/ip](http://www.resmed.com/ip).
**ResMed Pty Ltd**