


S8™ Auto 25


Quick Setup Guide for Clinicians

English

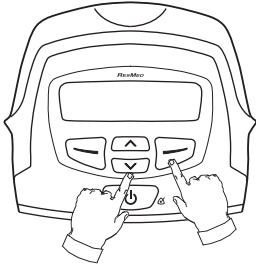

To access the Clinical Menu:


Hold down and simultaneously for at least three seconds.

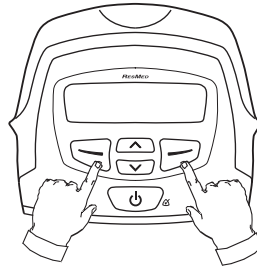

To access the Review menu:

Hold down and simultaneously for at least three seconds.

Choose Exit to return to the Patient menu.

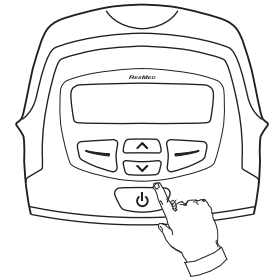


To access the Mask Fit function:

Hold down for at least three seconds.


Press to return to the Patient menu.

Treatment Screens

After starting therapy an introductory treatment screen will display. Press or to switch between views.

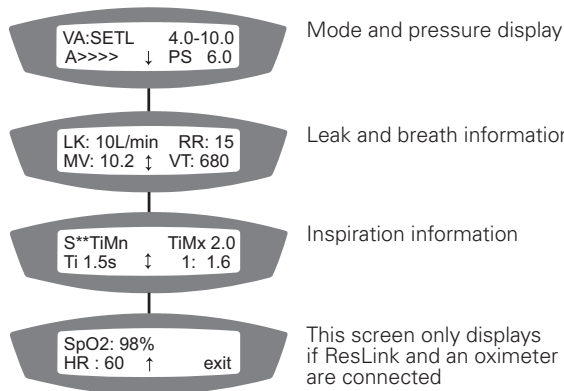


Patient Menu

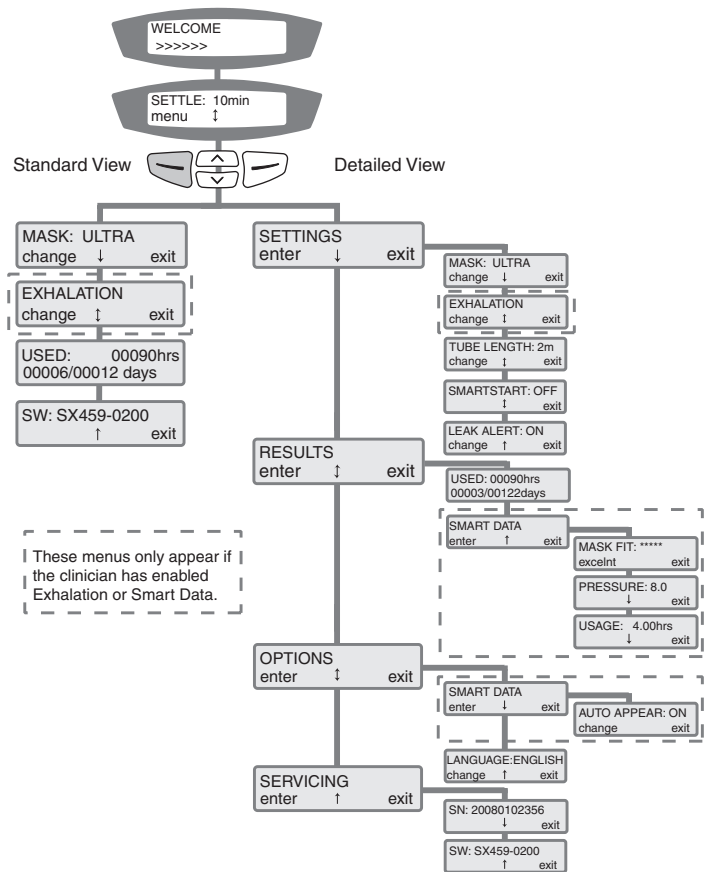


Exhalation (VAuto mode)

You can adjust the Exhalation rate so that the pressure drop is at a level that is most comfortable for the patient.

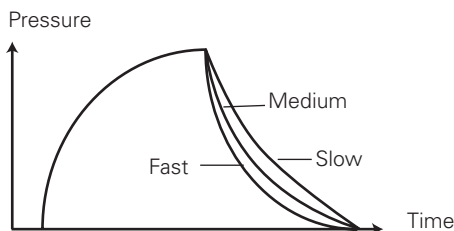

Clinical Menu

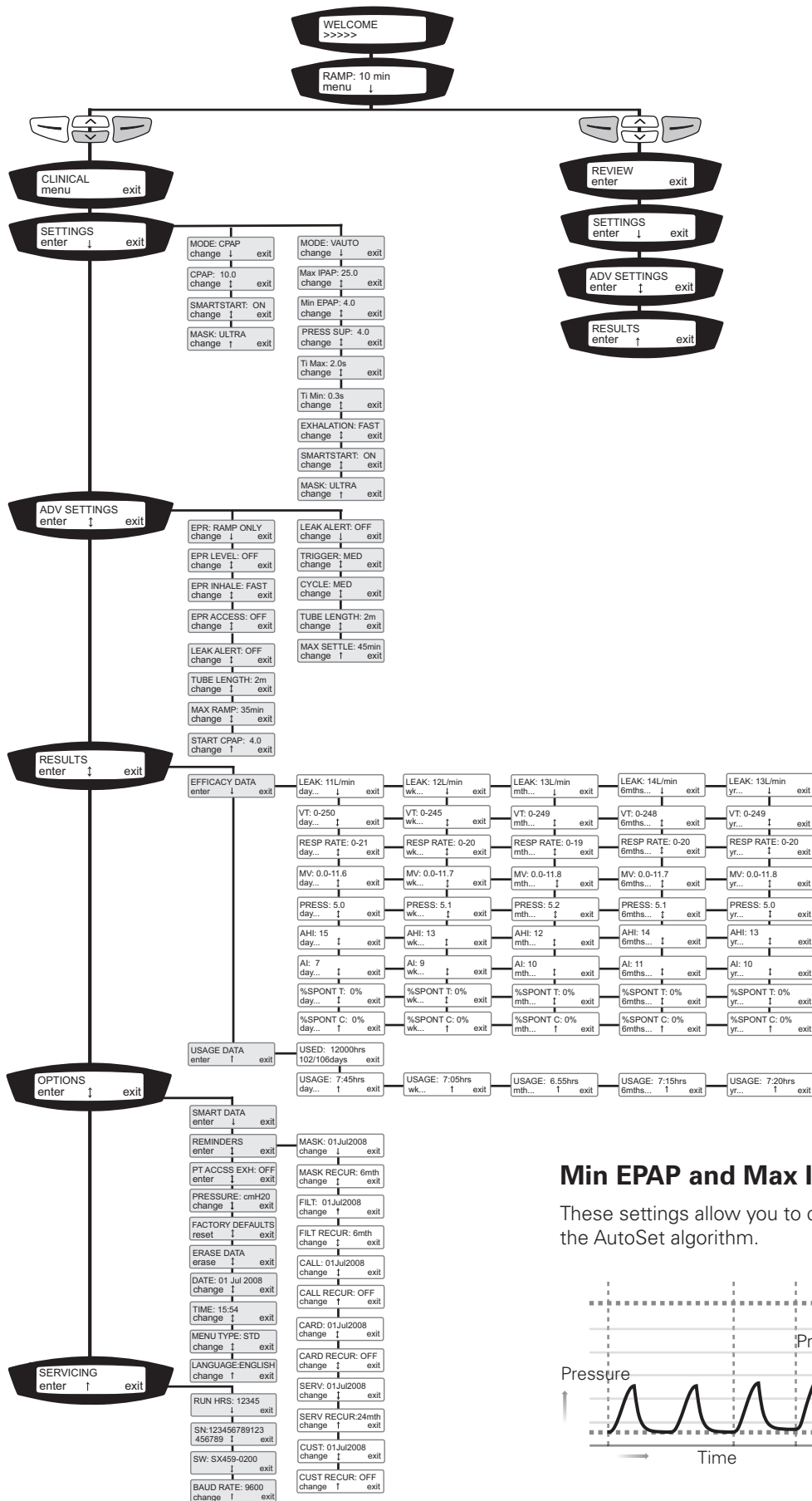

Min EPAP and Max IPAP (VAuto mode)

These settings allow you to constrain the pressure range of the AutoSet algorithm.

