



AcuCare™ HFNC
High flow nasal cannula



Speed, stability and success in HFOT

The AcuCare HFNC is fast and intuitive to fit, and is engineered to stay in place once fitted, providing speed and stability for built-in success in high flow oxygen therapy (HFOT).

Fits in seconds

Drawing on technology used in ResMed's premium patient interfaces, the AcuCare HFNC is built with the same innovative dual-strap headgear to allow fitting in seconds. The headgear's stretch fabric lets you move the straps apart to loosen, and bring them together to tighten. It's that simple.

Engineered to stay in place


Not only is the AcuCare HFNC fast to fit, but it's also designed to stay in place once fitted, thanks to its innovative headgear-to-frame combination. This allows you to quickly stabilise your patient, so you can spend more time on care where it's needed most.

Built-in success

When you select a ResMed interface, you're choosing technology that draws on over 25 years of experience in noninvasive solutions. So you can be confident in the quality of therapy you're providing to your patients.

Key features

- The innovative dual-strap headgear sits snugly at the crown of the head.
- The frame stabilises the cannula on the face, and suits a wide range of individual facial profiles.
- From this stable platform, the prongs provide a contoured fit in the nares, ensuring patient comfort even at high flow rates.

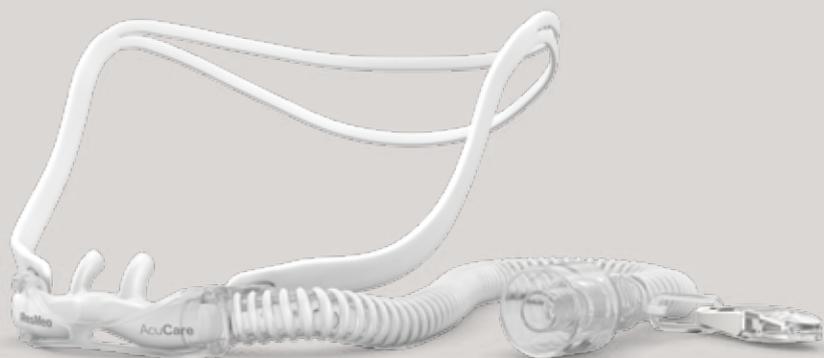

Sizing and fitting guide for AcuCare HFNC



- 1
- Use the sizing template on the packaging to determine the correct cannula size for your patient.
 - Inspect the nasal cannula before use, and replace it if any parts are damaged.



- 2
- Hold the prongs away from the nose.
 - Hold the bottom strap and position it at the back of the head.



- 3
- The top strap will be positioned on the top of the head.
 - Check that the prongs are in the nares, but not sealing them.
 - Move the straps apart to loosen, and bring them together to tighten.



- 4
- Fix the clip on the end of the short tube to your patient's garment.



- 5
- Connect the cannula to the heated humidification system tubing.

Product codes

AcuCare HFNC: Small (20 pk)	23003
AcuCare HFNC: Medium (20 pk)	23004
AcuCare HFNC: Large (20 pk)	23005

Note: For more details, refer to your nasal cannula user guide.