**ResMed**

**AirFit™**

**F20**

NON MAGNETIC

Full face mask

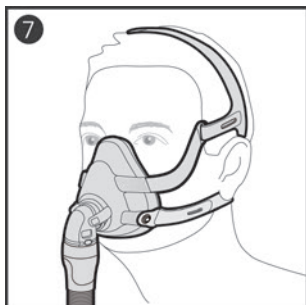
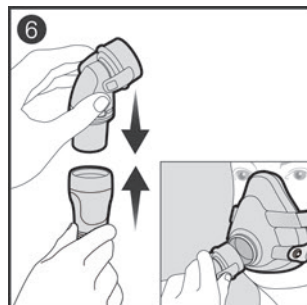
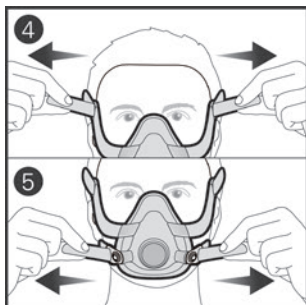
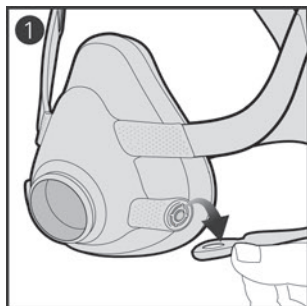
QuietAir



User guide


English | Français | Español | Português

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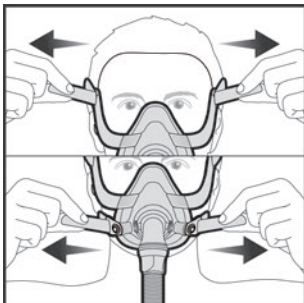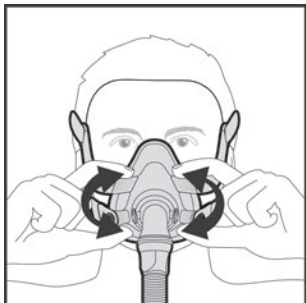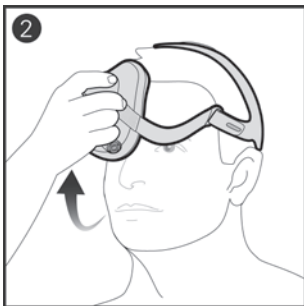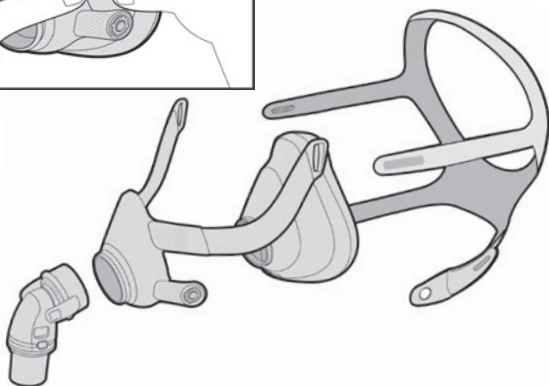
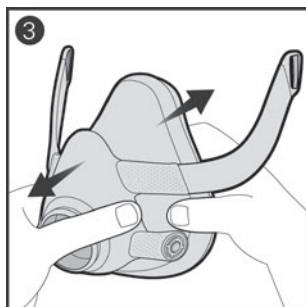
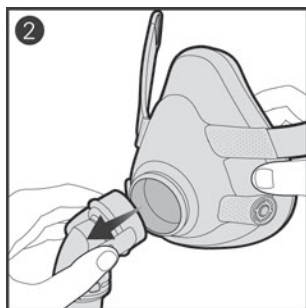
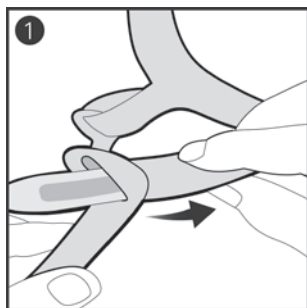
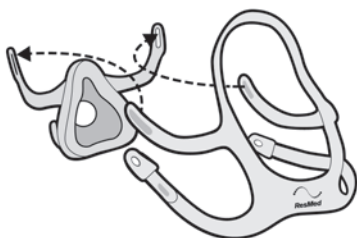
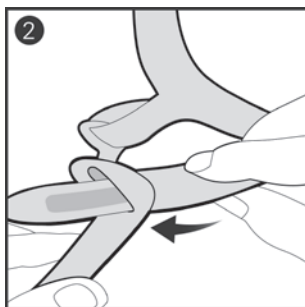
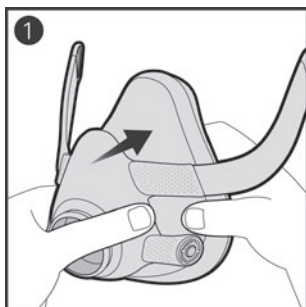
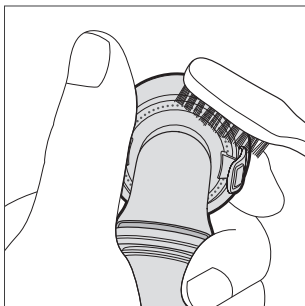
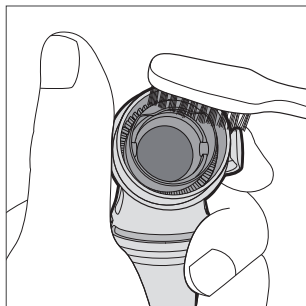
Thank you for choosing the AirFit™ F20 Non Magnetic full face mask.

### Using this guide

Please read the entire guide before use. When following instructions, refer to the images at the front of the guide.

### Intended use

The AirFit F20 Non Magnetic mask has two product variants:

- The AirFit F20 Non Magnetic variant is intended for single-patient reuse in the home environment
- The AirFit F20 Non Magnetic SLM (Sleep Lab Mask) variant is intended for multi-patient reuse in the hospital/institutional environment.

This mask is intended for patients weighing more than 66 lb (30 kg) who have been prescribed non-invasive CPAP or bi-level positive airway pressure (PAP) therapy. The Sleep Lab Mask is the only variant that is validated and intended for multi-patient reprocessing and must be reprocessed if reused between patients.

### QuietAir™ elbow

The QuietAir elbow is intended for single-patient reuse in both home and hospital/institutional environments.

## GENERAL WARNINGS

- The mask must be used under qualified supervision for users who are unable to remove the mask by themselves. The mask may not be suitable for those predisposed to aspiration.
- The mask must be fitted with the supplied elbow (containing the valve and vent assembly) to ensure safe and functional usage unless otherwise specified. Do not use the mask if the valve or vent assembly is damaged or missing.
- The mask contains safety features in the elbow, which are the valve and vent, to enable normal breathing and exhaust exhaled breath. Occlusion of the valve and vent needs to be prevented to avoid having an adverse effect on the safety and quality of the therapy. Regularly inspect the valve and vent to ensure they are kept clean, undamaged and clear of blockages. The mask should

not be worn if the valve is damaged as it will not be able to perform its safety function. The elbow should be replaced if the valve is damaged, distorted or torn. The vent holes and valve should be kept clear.

- The mask should only be used with CPAP or bilevel devices recommended by a physician or respiratory therapist.
- Avoid connecting flexible PVC products (eg, PVC tubing) directly to any part of the mask. Flexible PVC contains elements that can be damaging to the materials of the mask, and may cause the components to crack or break.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.  
Explanation: CPAP and bilevel devices are intended to be used with special masks (or connectors) which have venting to allow continuous flow of air out of the mask. When the device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask holes. When the device is turned off, the mask valve opens to atmosphere allowing fresh air to be breathed. However, a higher level of exhaled air may be rebreathed when the device is off. This applies to most full face masks for use with CPAP and bilevel devices.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame. Only use oxygen in well ventilated rooms.
- At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate. This warning applies to most types of CPAP or bilevel devices.
- The technical specifications of the mask are provided for your clinician to check that they are compatible with the CPAP or bilevel device. If used outside specification or if used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the CPAP or bilevel device function.

- Discontinue using this mask if you have ANY adverse reaction to the use of the mask, and consult your physician or sleep therapist.
- Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.
- The F20 line of full face CPAP masks are not intended to be used simultaneously with nebulizer medications that are in the air path of the mask/tube.
- As with all masks, some rebreathing may occur at low CPAP pressures.
- Refer to your CPAP or bilevel device manual for details on settings and operational information.
- Remove all packaging before using the mask.

## Using your mask

When using your mask with ResMed CPAP or bilevel devices that have mask setting options, refer to the Technical specifications section in this user guide for the correct setting.

For a full list of compatible devices for this mask, see the Mask/Device Compatibility List on [www.resmed.com/downloads/masks](http://www.resmed.com/downloads/masks). If you do not have internet access, please contact your ResMed representative.

Use a standard conical connector if pressure readings and/or supplemental oxygen are required.

## Fitting

1. Lift and pull both mask clips away from the frame.
2. Ensure that the ResMed logo on the headgear is facing outwards and is upright. With both lower headgear straps released, hold the mask against your face and pull the headgear over your head.
3. Bring the lower straps below your ears, and attach the mask clip to the frame.
4. Undo the fastening tabs on the upper headgear straps. Pull the straps evenly until the mask is stable and positioned as shown in the illustration. Reattach the fastening tabs.
5. Undo the fastening tabs on the lower headgear straps. Pull the straps evenly until the mask is stable and sits comfortably on the chin. Reattach the fastening tabs.

6. Connect the device air tubing to the elbow. Attach the elbow to the mask by pressing the side buttons and pushing the elbow into the mask, ensuring it clicks in on both sides.
7. Your mask and headgear should be positioned as shown in the illustration.

## Adjustment

If necessary, slightly adjust the position of the mask for the most comfortable fit. Ensure that the cushion is not creased and the headgear is not twisted.

1. Turn on your device so that it is blowing air.

### **Adjustment tips:**

- ✓ With air pressure applied, pull the mask away from your face to allow the cushion to inflate and reposition onto your face.
- ✓ To resolve any leaks at the upper part of the mask, adjust the upper headgear straps. For the lower part, adjust the lower headgear straps.
- ✓ Adjust only enough for a comfortable seal. Do not overtighten as this may cause discomfort.

## Removal

1. Lift and pull both mask clips away from the frame.
2. Pull the mask away from your face and back over your head.

## Disassembly

If the mask is connected to your device, disconnect the device air tubing from the elbow.

1. Undo the fastening tabs on the upper headgear straps. Pull the straps out of the frame.

**Tip:** Keep the mask clips attached to the lower headgear straps to easily distinguish the upper and lower straps when reassembling.

2. Remove the elbow from the mask by pressing the side buttons and pulling the elbow away.
3. Hold the side of the frame between the upper and lower arms. Gently pull the cushion away from the frame.

## Reassembly

1. Attach the cushion to the frame by aligning the circular openings and pushing together until retained.
2. With the ResMed logo on the headgear facing outside and upright, thread the upper headgear straps into the frame slots from the inside. Fold them over to secure.

## Cleaning your mask at home

It is important to follow the steps below to get the best performance out of your mask.

### **WARNING**

- As part of good hygiene, always follow cleaning instructions and use a mild liquid detergent. Some cleaning products may damage the mask, its parts and their function, or leave harmful residual vapours that could be inhaled if not rinsed thoroughly.
- Regularly clean your mask and its components to maintain the quality of your mask and to prevent the growth of germs that can adversely affect your health.
- Visible criteria for product inspection: If any visible deterioration of a system component is apparent (cracking, discoloration, tears, etc.), the component should be discarded and replaced.

### Daily/After each use:

1. Disassemble the mask according to the disassembly instructions.
2. Rinse the frame, elbow and cushion under running water. Clean with a soft brush until dirt is removed.
3. Soak the components in warm water (approximately 86°F/30°C) with a mild liquid detergent for up to ten minutes.
4. Shake the components vigorously in the water for at least one minute.
5. Brush the moving parts of the elbow and around the vent holes.
6. Brush the areas of the frame where the arms connect, and inside and outside the frame where the elbow connects.
7. Rinse the components under running water.
8. Leave the components to air dry out of direct sunlight. Make sure to squeeze the arms of the frame to ensure that excess water is removed.

## Weekly:

1. Disassemble the mask. The mask clips can remain attached to the headgear during cleaning.
2. Handwash the headgear in warm water (approximately 86°F/30°C) with mild liquid detergent.
3. Rinse the headgear under running water. Inspect to ensure the headgear is clean and detergent free. Wash and rinse again, if necessary.
4. Squeeze the headgear to remove excess water.
5. Leave the headgear to air dry out of direct sunlight.

## Reprocessing the mask between patients

Reprocess this mask when using between patients. Cleaning and disinfection instructions are available on [www.resmed.com/downloads/masks](http://www.resmed.com/downloads/masks). If you do not have internet access, please contact your ResMed representative.

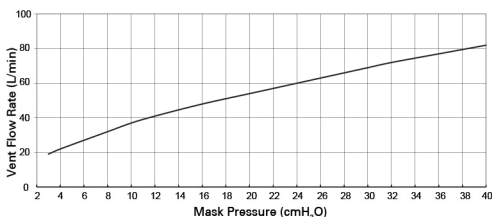
## Troubleshooting

| Problem/possible cause                                                                             | Solution                                                                                                                                                                                                                            |
|----------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Mask is uncomfortable.</b><br>Headgear straps are too tight.                                    | The cushion membrane is designed to inflate against your face to provide a comfortable seal with low headgear tension. Adjust straps evenly. Ensure that the headgear straps are not too tight and that the cushion is not creased. |
| Mask could be the wrong size.                                                                      | Talk to your clinician to have your face size checked. Note that sizing across different masks is not always the same.                                                                                                              |
| <b>Mask is too noisy.</b><br>Elbow incorrectly installed or incorrect assembly of the mask system. | Remove the elbow from your mask, then reassemble according to the instructions. Check that the mask is correctly assembled according to the instructions.                                                                           |
| Vent is dirty.                                                                                     | Use a soft bristle brush to clean the vent.                                                                                                                                                                                         |

| Problem/possible cause                                                                                  | Solution                                                                                                                                                                                                                                            |
|---------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Mask leaks around the face.<br>Cushion membrane is creased. Mask is incorrectly positioned or adjusted. | Refit your mask according to the instructions. Ensure that you position the cushion correctly on your face before pulling headgear over your head. Do not slide the mask down your face during fitting as this may lead to creasing of the cushion. |
| Mask could be the wrong size                                                                            | Talk to your clinician to have your face size checked. Note that sizing across different masks is not always the same.                                                                                                                              |

## Technical specifications

|                            |                                                                                                                                         |
|----------------------------|-----------------------------------------------------------------------------------------------------------------------------------------|
| <b>Pressure-flow curve</b> | The mask contains passive venting to protect against rebreathing. As a result of manufacturing variations, the vent flow rate may vary. |
|----------------------------|-----------------------------------------------------------------------------------------------------------------------------------------|

| Pressure (cm H <sub>2</sub> O) | Flow (L/min) |
|--------------------------------|--------------|
| 3                              | 19           |
| 13                             | 42           |
| 22                             | 57           |
| 31                             | 70           |
| 40                             | 82           |

|                          |                            |
|--------------------------|----------------------------|
| <b>Therapy pressure:</b> | 3 to 40 cmH <sub>2</sub> O |
|--------------------------|----------------------------|

### Resistance with Anti Asphyxia Valve (AAV) closed to atmosphere:

Drop in pressure measured (nominal)

|             |                        |
|-------------|------------------------|
| at 50 L/min | 0.2 cmH <sub>2</sub> O |
|-------------|------------------------|

|              |                        |
|--------------|------------------------|
| at 100 L/min | 0.6 cmH <sub>2</sub> O |
|--------------|------------------------|

### Resistance with Anti Asphyxia Valve (AAV) open to atmosphere:

|                          |                        |
|--------------------------|------------------------|
| Inspiration at 50 L/min: | 0.6 cmH <sub>2</sub> O |
|--------------------------|------------------------|

|                         |                        |
|-------------------------|------------------------|
| Expiration at 50 L/min: | 0.8 cmH <sub>2</sub> O |
|-------------------------|------------------------|

|                                                               |                       |
|---------------------------------------------------------------|-----------------------|
| <b>Anti Asphyxia Valve (AAV) open-to-atmosphere pressure:</b> | ≤3 cmH <sub>2</sub> O |
|---------------------------------------------------------------|-----------------------|

|                                                                 |                       |
|-----------------------------------------------------------------|-----------------------|
| <b>Anti Asphyxia Valve (AAV) closed-to-atmosphere pressure:</b> | ≤3 cmH <sub>2</sub> O |
|-----------------------------------------------------------------|-----------------------|

**Sound:** DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871:1996 and ISO 3744:2010. The A-weighted sound power level and the A-weighted sound pressure level of the mask at a distance of 1 m, with uncertainty of 3 dBA, are shown

|                                      |        |
|--------------------------------------|--------|
| Power level (with Standard elbow)    | 30 dBA |
| Power level (with QuietAir elbow)    | 21 dBA |
| Pressure level (with Standard elbow) | 22 dBA |
| Pressure level (with QuietAir elbow) | 13 dBA |

---

|                         |                                                                                                                                    |
|-------------------------|------------------------------------------------------------------------------------------------------------------------------------|
| <b>Gross dimensions</b> | Mask fully assembled with elbow assembly (no headgear)<br>6.1" (H) x 6.3" (W) x 5.8" (D)<br>(154 mm (H) x 159 mm (W) x 147 mm (D)) |
|-------------------------|------------------------------------------------------------------------------------------------------------------------------------|

---

|                                 |                                                                                                                                                                                                                                      |
|---------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Environmental conditions</b> | Operating temperature: 41°F to 104°F (5°C to 40°C)<br>Operating humidity: 15% to 95% non-condensing<br>Storage and transport temperature: -4°F to 140°F (-20°C to +60°C)<br>Storage and transport humidity: up to 95% non-condensing |
|---------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

---

|                     |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                |
|---------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Service life</b> | The service life of the AirFit F20 Non Magnetic mask system is dependent on the intensity of usage, maintenance, and environmental conditions to which the mask is used or stored. As this mask system and its components are modular in nature, it is recommended that the user maintain and inspect it on a regular basis, and replace the mask system or any components if deemed necessary or according to the 'visual criteria for product inspection' in the 'Cleaning your mask at home' section of this guide. Refer to the 'Mask components' section of this guide for information of how to order replacement parts. |
|---------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

---

|                             |                                                                                                                                                               |
|-----------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Mask setting options</b> | For AirSense™, AirCurve™ or S9™: Select 'Full Face'.<br>For other devices: Select 'MIR FULL' (if available), otherwise select 'FULL FACE' as the mask option. |
|-----------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------|

---

Device Setting

Full Face

---

## Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

## Disposal

This mask and packaging do not contain any hazardous substances and may be disposed of with your normal household refuse.

## Symbols

The following symbols may appear on your product or packaging.

Full face mask

Device setting - Full Face

Size - small

Size - medium

Size - large

QuietAir vent

### LATEX?

NOT MADE  
WITH NATURAL  
RUBBER LATEX

Not made with natural rubber latex

See symbols glossary at [ResMed.com/symbols](https://www.resmed.com/symbols)

## Limited warranty

ResMed Pty Ltd (hereafter 'ResMed') warrants that your ResMed mask system (including mask frame, cushion, headgear and tubing) will be free from defects in material and workmanship from the date of purchase for 90 days, or in the case of disposable masks and disposable mask components for 7 days. This warranty is only available to the initial consumer. It is not transferable. During the warranty period, if the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components. This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organization that has not been expressly authorized by ResMed to perform such repairs; and c) any damage or contamination due to cigarette, pipe, cigar or other smoke. Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty replaces all other expressed or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have resulted from the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

Visit [ResMed.com](https://www.resmed.com) for the latest information on ResMed's Limited Warranty.

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© 2023 ResMed. 638428/1 2023-10
See [ResMed.com](https://www.ResMed.com) for other ResMed locations worldwide. AirFit, AirSense, AirCurve, QuietAir and S9 are trademarks and/or registered trademarks of the ResMed family of companies. For patent and other intellectual property information, see [ResMed.com/ip](https://www.ResMed.com/ip).