

AirFit™ F30



Full face mask

Fitting guide for AirFit F30



- Twist and pull both magnetic clips away from the frame.
- Place the ultra-compact cushion under your nose and ensure it sits comfortably on your face. With the ResMed logo on the headgear facing up, pull the headgear over your head.



- Bring the lower headgear straps under your ears and attach the magnetic headgear clips to the frame.



- Undo the fastening tabs on the upper headgear straps and pull evenly.
- Repeat with the lower headgear straps.



- If the headgear is too close to the ear, adjust the headgear strap on the crown of the head.



- Connect the air tubing from your device to the elbow.



- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
- Start the device.

Refer to the user guide for further instructions.



Additional options

- For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.
- Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.



AirFit™ F30
Full face mask



Simplicity wrapped in high performance

Introducing ResMed AirFit™ F30 – our ultra-compact mask that delivers full-face functionality in a design your patients can appreciate. Smaller, quieter and less obtrusive than a traditional mask, its under-the-nose design can help to make therapy less challenging by increasing visual and physical freedom. Issues with discomfort and facial markings on the nasal bridge may now be things of the past.



ResMed.com/AirFitF30

1 ResMed external 7 days clinical study of existing 21 ResMed and non-ResMed patients, conducted between 16/04/2018 - 05/05/2018. Data on file; ID A4356449.

AirFit and QuietAir are trademarks of ResMed Ltd. Specifications may change without notice. © 2019 ResMed Ltd. 1011198/1 2019-01

ResMed.com/AirFitF30

Minimal-contact redefined

The under-the-nose design of the AirFit F30 aims to eliminate pressure, discomfort and facial markings on the nasal bridge. The quick-release elbow is easy to connect and disconnect if patients need to get up during the night, and the 360° rotation and tilting at the elbow helps to reduce tube drag, whilst offering greater freedom of movement during sleep.


More peaceful sleep

The AirFit F30 features the same innovative QuietAir™ vent as the AirFit F20, gently diffusing exhaled air to minimise noise, helping to facilitate a more peaceful sleep for patients and their bed partner.


Streamlined operations

You can achieve fast and efficient fitting with just one frame, one headgear and two cushion sizes. This streamlined setup simplifies stock management and logistics. Designed for an accurate first-time fit it can help to reduce call backs and refits.

AirFit F30 highlights

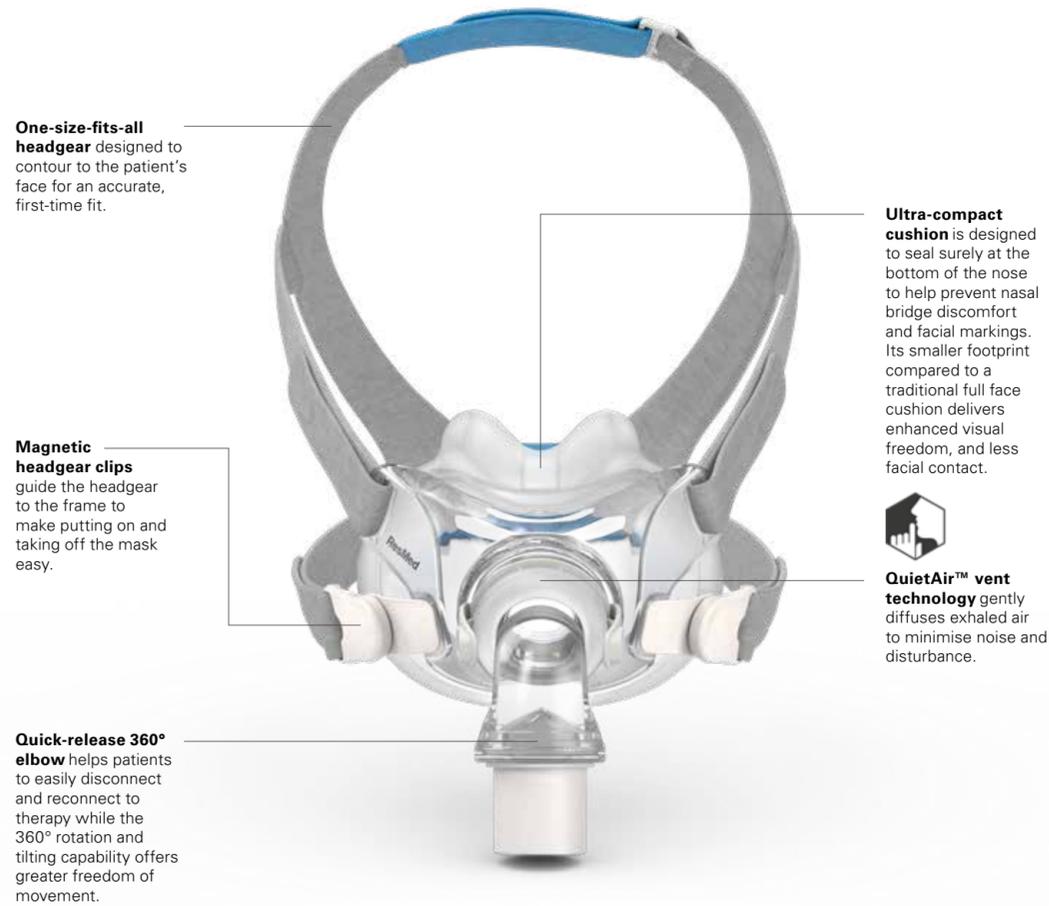

Few parts for simplified stock management

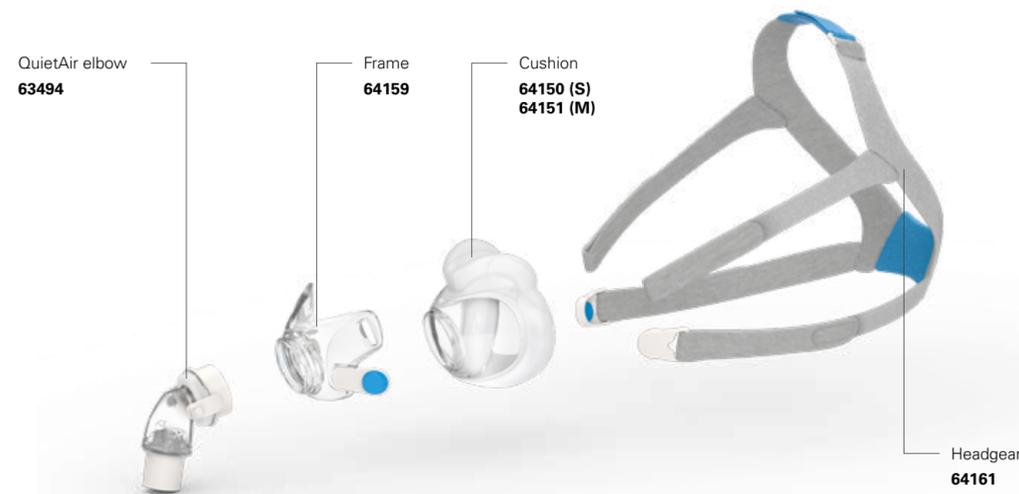

Septum bridge membrane ensures an intuitive fit and provides a stable seal¹ under the nose and around the mouth through the night.

Frosted silicone surface provides a soft cushion texture around the sensitive nasal region for a comfortable¹ seal and easy fit.

Polished silicone finish designed to seal around the mouth and jawline for a smooth and stable fit.



Sizing tips


AirFit F30 has two interchangeable cushion sizes - small and medium.

The modular design of the AirFit F30 means each size cushion easily clicks into the same mask frame, so you can swap between them with ease for more flexibility.

We recommend that you refer to the mask fitting template to gauge which size is the most suitable for each of your patients.

Product codes

AirFit F30 mask system	ANZ
AirFit F30: Small	64126
AirFit F30: Medium	64127

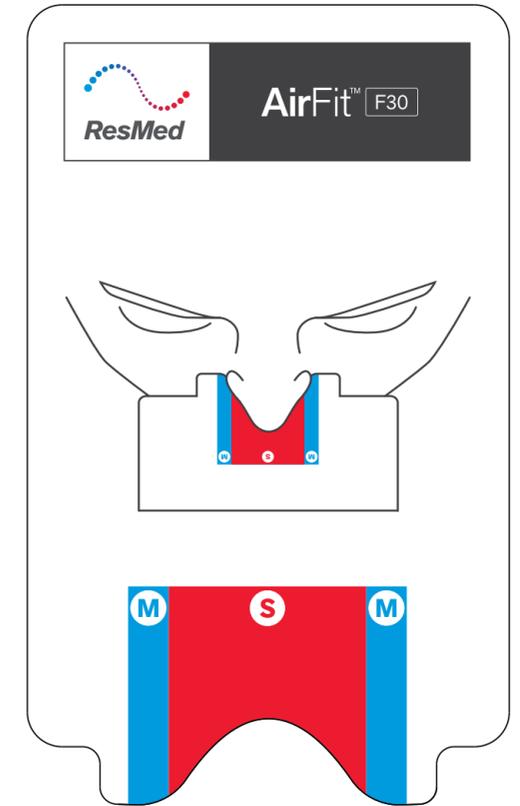

Actual size