



AirFit™ N20 N20
Nasal masks CLASSIC



The compact and classic nasal masks designed to easily fit everyone

Every patient's face is different, but our new AirFit N20 and AirFit N20 Classic nasal masks make it quick and easy to fit a broad range of patients.

Patients who like a clear line of sight will love the compact version while those who prefer a classic mask will appreciate the freedom and flexibility of the new adaptive frame.



Designed to fit

Every AirFit mask features the InfinitySeal™ silicone cushion which delivers a robust seal regardless of facial shape or size.


Soft and gentle

Both the classic and compact masks are gentle on the skin. With a soft cushion and headgear, your patients will enjoy a more comfortable therapy experience.

Made to mix and match

The AirFit N20 and AirFit N20 Classic nasal masks use the same cushion for greater flexibility and efficiency. You have fewer products to manage; your patients have more options to choose from.

92% of patients said they preferred wearing the AirFit N20 over the current market-leading nasal mask.¹

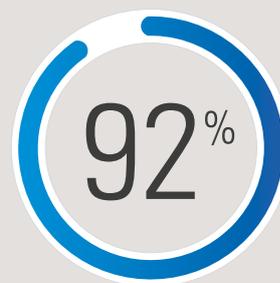

AirFit™ N20




Visual freedom

Let patients see and do more in bed with an under-the-eye frame that combines all-round stability and less facial contact.

Extra movement

A short, flexible tube interconnects the elbow to the machine tube, allowing patients to move more easily without encountering tube drag or leakage.

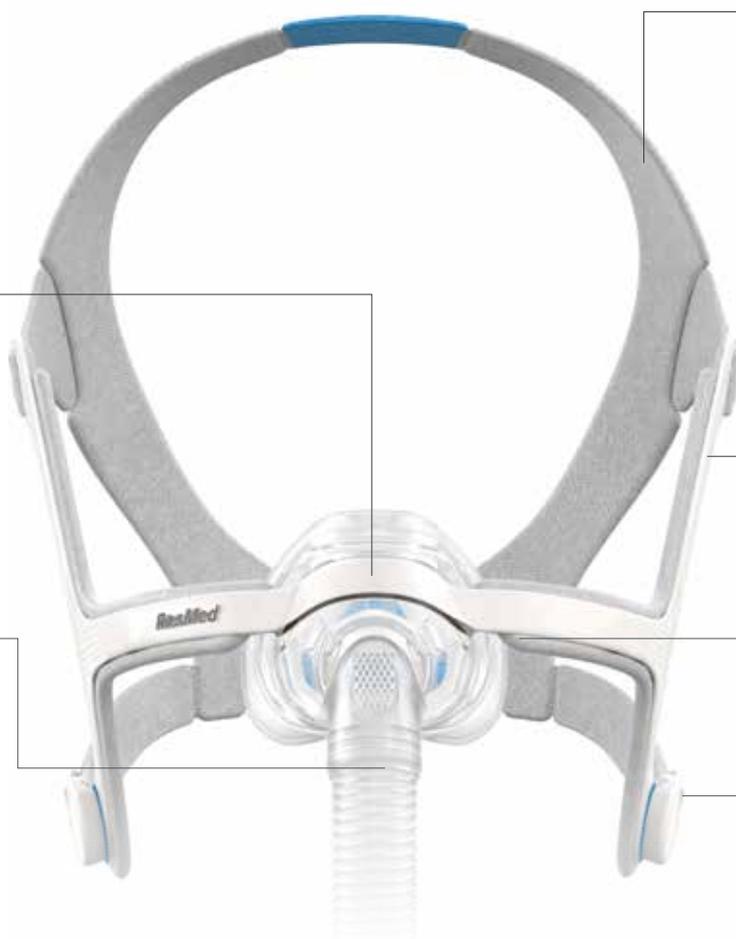

Soft frame

Padding has been seamlessly incorporated into the face-side of the frame to ensure comfort at every touchpoint.

Soft and

Specifically designed for softness, the padding helps ensure a secure fit into therapy and comfort.

Dependable

Set patients up for a reliable and comfortable fit thanks to the InFit cushion – designed for blow-outs, tolerance, and to seal well under various conditions.

Fast fitting time

Taking the mask on and off is convenient and easy, with magnetic clips that guide the headgear to the frame in seconds.


Elbow and short tube
63565

Frame
63566
63567 for Her

Cushion
63550 (S)
63551 (M)
63552 (L)

AirFit N20 headgear
63560 (S)
63561 (STD)
63562 (L)

AirFit N20 for Her headgear
63558

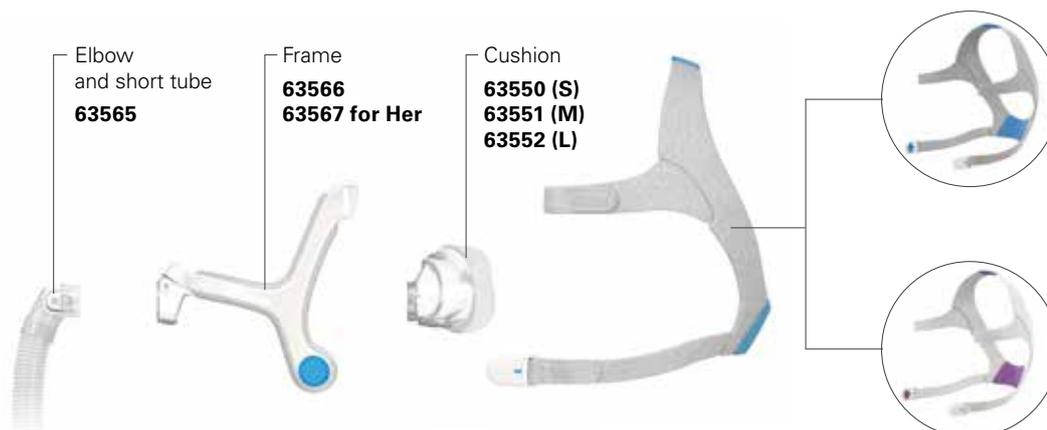

In an international study, AirFit N20 fit 99.4% of all patients.²



Soft and gentle

Designed for extra plush headgear so patients settle in comfortably quickly and comfortably.


Classic support

Forehead support from an adaptive frame provides stability and adjusts mask tension in response to movement.

Sealable seal

Comes up with an effective seal! The InfinitySeal silicone is designed to prevent late misfitting and variable pressures.

AirFit™ N20 CLASSIC

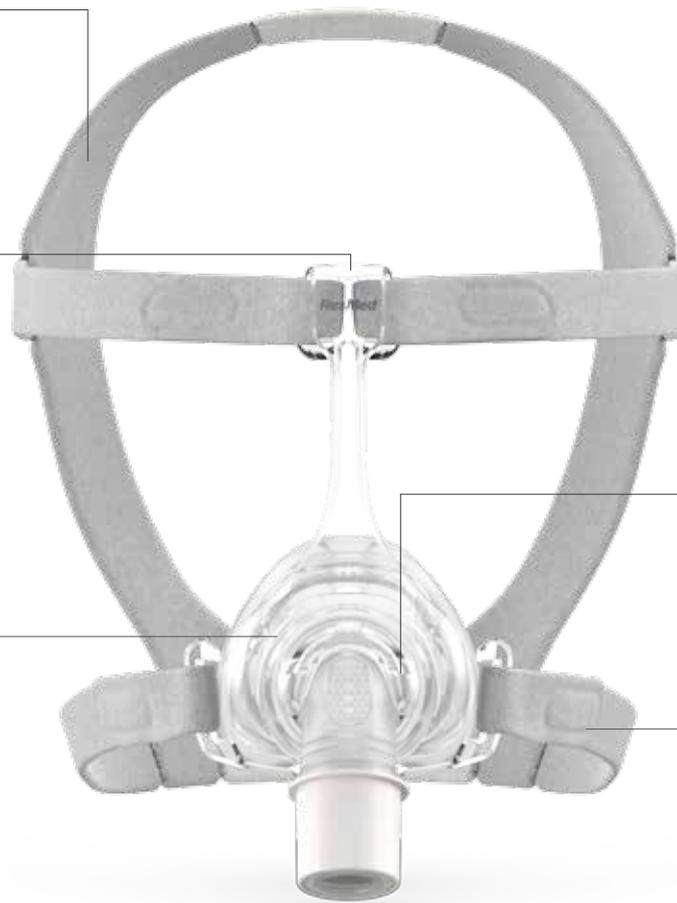


Quick to disconnect

The quick-release elbow is simple to connect and disconnect, so patients can move in and out of bed with ease.

Easy fitting

Classic velcro tabs for fitting and smooth slide-in hooks for fastening make everyday handling easy.

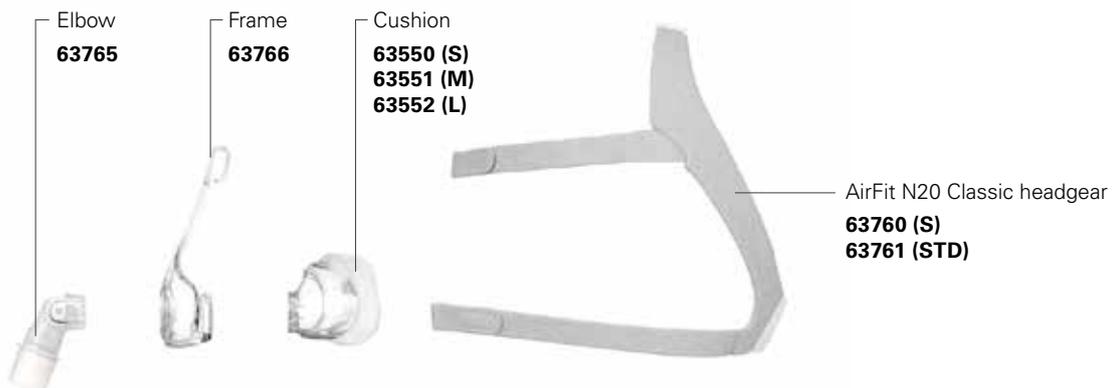




Sizing tips

AirFit N20 and AirFit N20 Classic masks have three interchangeable cushion sizes - small, medium and large.

The modular design of the AirFit N20 masks means each size cushion easily clicks into the same mask frame, so you can swap between them with ease for more flexibility.

We recommend that you refer to the mask fitting template to gauge which size is the most suitable for each of your patients.

You can download it on ResMed.com/AirFit20

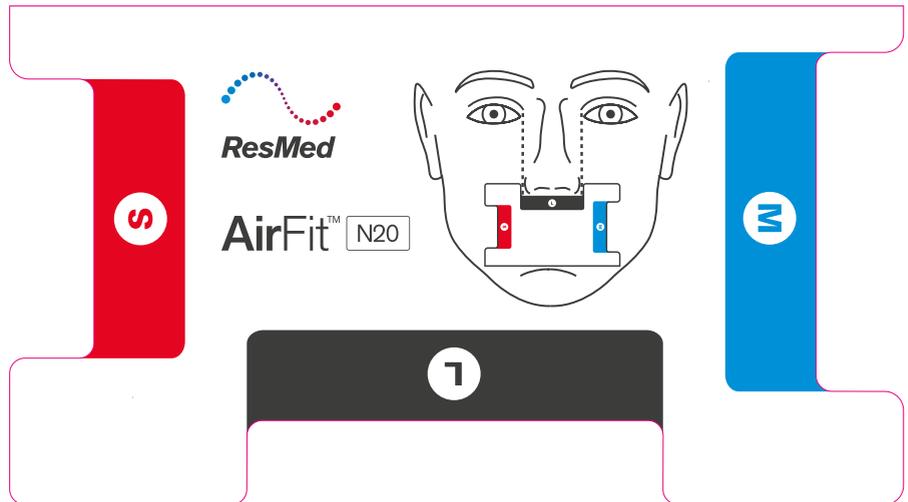

Product codes

AirFit N20

	Europe 1	Europe 2	Europe 3	Japan
AirFit N20 for Her: Small	63510	63513	63516	63531
AirFit N20: Small	63509	-	-	-
AirFit N20: Medium	63511	63514	63517	63532
AirFit N20: Large	63512	63515	63518	63533

AirFit N20 Classic


	Europe 1	Europe 2	Europe 3	Japan
AirFit N20: Small	63710	63713	63716	63731
AirFit N20: Medium	63711	63714	63717	63732
AirFit N20: Large	63712	63715	63718	63733

Europe 1: English (UK), Arabic, Dutch (Belgium, Netherlands), French, German and Italian (Switzerland)


Europe 2: English, Danish, Estonian, Finnish, Norwegian, Russian and Swedish

Europe 3: English, Bulgarian, Czech, Greek, Hungarian, Polish, Portuguese, Romanian, Spanish and Turkish

88% of patients ranked AirFit N20 higher in comfort over the market-leading nasal mask.²

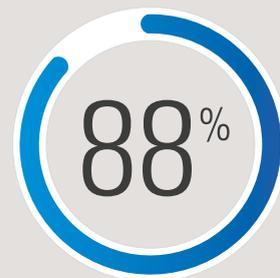

One cushion. A range of technological advances for comfort and stability

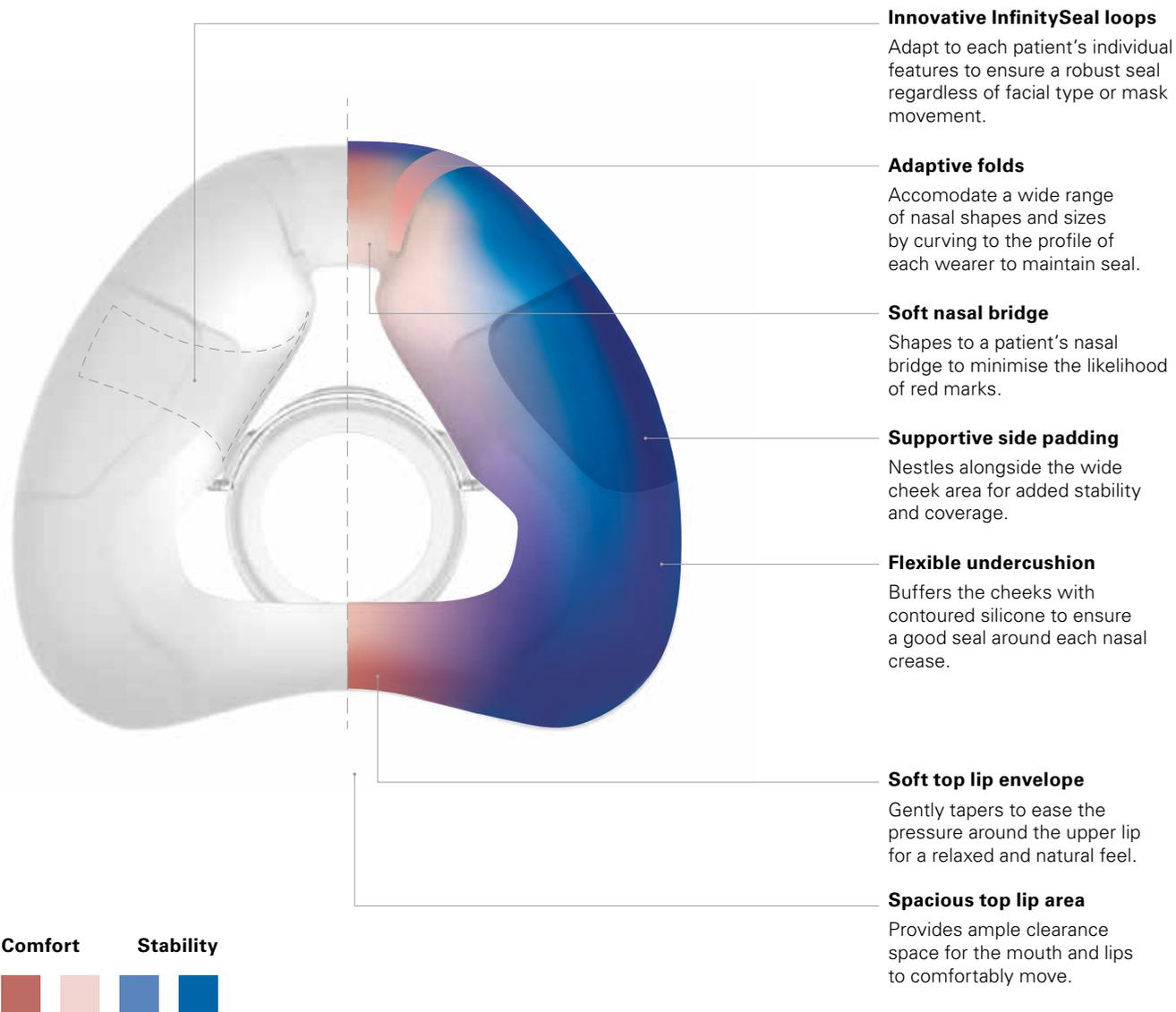



Fitting guide for AirFit N20 compact



- 1
- Start by ensuring the magnets on the lower straps are unclipped.
 - If they are clipped, detach them by twisting and pulling them away from the frame.



- 2
- Pull the headgear over the head making sure the ResMed logo is facing outwards.



- 3
- Position the lower straps so they both sit below the ears.
 - Bring each of the magnets up to meet their corresponding clips on the frame.



- 4
- Adjust the fastening tabs on the upper straps until the mask feels even and stable.
 - Reattach the fastening tabs.



- 5
- Repeat Step 4 with the lower straps.



- 6
- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
 - Start your device.

for AirFit N20 Classic



- 1
- Start with the lower straps unhooked from the frame.



- 2
- Ensure the mask logo is facing downwards.
 - Hold the mask against your face and pull the headgear over your head.



- 3
- Bring the lower straps below your ears, and hook the straps into the mask frame.



- 4
- Adjust the velcro tabs on the upper straps until the mask feels even and stable. Do not overtighten.
 - Reattach the velcro tabs.



- 5
- Repeat Step 4 with the lower straps.
 - Reattach the fastening tabs.



- 6
- Connect your device's air tube to the elbow, then click the elbow into your mask.
 - Start your device.

Refer to the user guide for further instructions.



Additional options

- A patient can position the lower part of the headgear over or under their hair to suit any hairstyle preference.

1 ResMed internal study of 24 existing ResMed patients, conducted between 23/11/2015 - 21/12/2015 comparing the market leading mask with AirFit N20 ; ID A3695085.
2 ResMed AirFit N20 internal global fitting study of 159 existing ResMed patients, conducted 12/11/2015 ; ID A3697629.