

ResMed

Actual Size:
148 mm (Width)
x 105 mm (Height)


Ruler is for scaling purposes.

How to use the fitting template:

1. Please ensure the template is printed to scale
2. Cut out along dotted line
3. Refer to diagram on template to determine appropriate cushion size


AirFit™ N30i

Nasal cradle mask

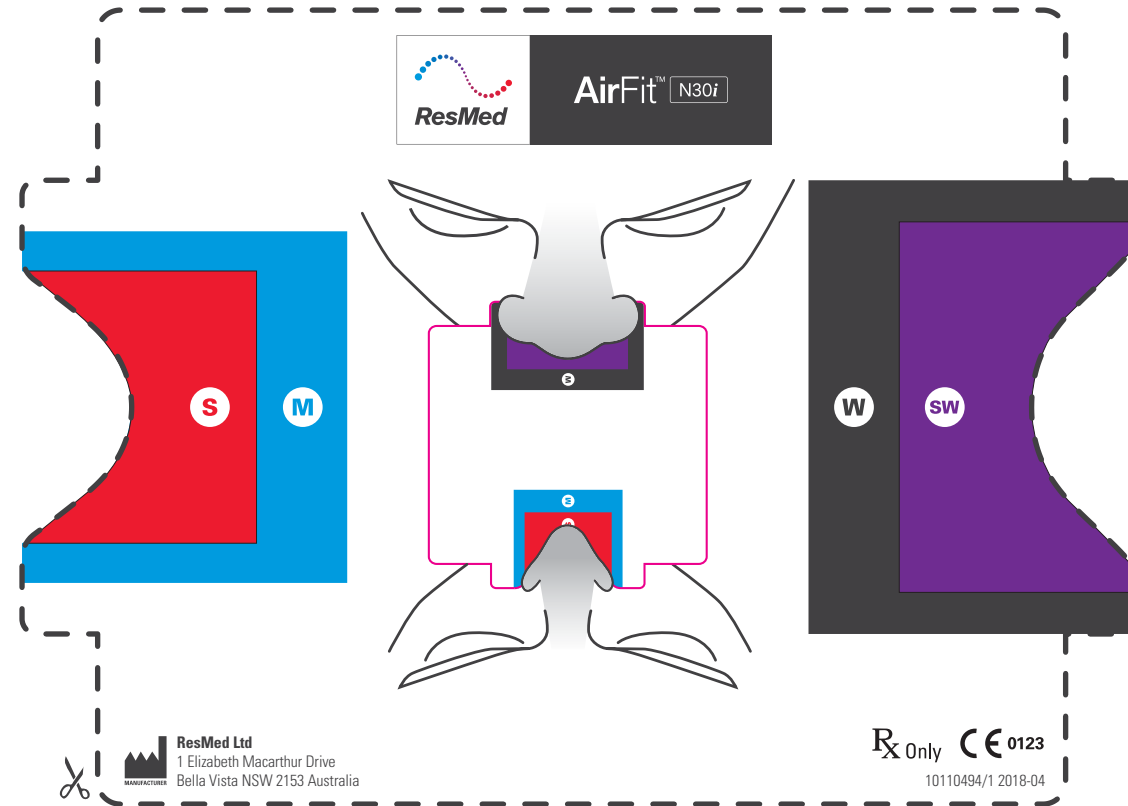

Sample size check

10 mm 20 mm 30 mm 40 mm 50 mm 60 mm 70 mm 80 mm 90 mm 100 mm 110 mm 120 mm 130 mm 140 mm 150 mm 160 mm 170 mm

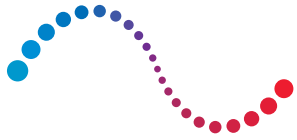




ResMed

AirFit™ N30i
Nasal mask

AirFit™ P30i
Nasal pillows mask

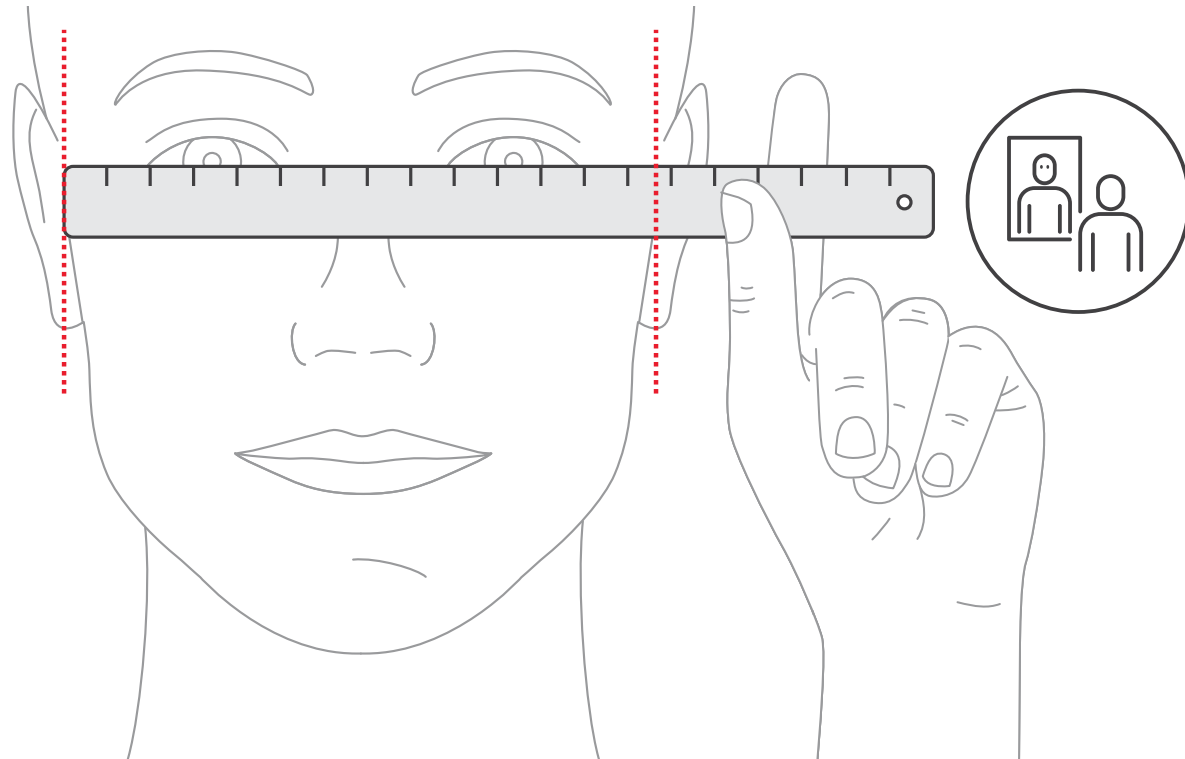Frame Fitting Instructions



 Small frame	 Standard frame
0-150mm	>150mm
0-6"	>6"