

AirTouch™ N20



Our softest nasal mask ever

When it comes to effective sleep therapy, comfort is key. That's why we created ResMed AirTouch™ N20, our first nasal mask to feature our patented UltraSoft™ memory foam cushion technology, which provides patients with a unique comfort experience. Like AirTouch F20, 9 out of 10 study participants who tried AirTouch N20 rated the memory foam cushion comfortable.^{1,2*†}



ResMed.com/AirTouchN20

UltraSoft memory foam adapts to the unique contours of each face, creating a personalized fit.



AirTouch N20 highlights

Plush headgear is specifically designed for extra softness and stability.

UltraSoft cushion is designed to seal across a wide range of therapy pressures² and prevent red marks around the nose area. A majority of nasal mask users who tried it in a recent study got an excellent seal and no mask leak while wearing it.^{1*}

Modular frame is designed so that the memory foam cushion can be swapped out easily with a ResMed AirFit™ N20 InfinitySeal™ silicone cushion.

Quick-release elbow makes it simple to disconnect from therapy without having to remove the mask and ensures parts don't get lost or stuck.

Magnetic clips guide the headgear to the frame quickly and easily. No wonder 9 out of 10 CPAP users found AirTouch N20 easy to use in a recent study.^{1*}



AirTouch N20 for Her
Offers a smaller size range and lavender mask accents.



AirTouch N20 components

Elbow with short tube



Frame

AirTouch N20
AirTouch N20 for Her




Cushion

63950 (S)
63951 (M)
63952 (L)



Headgear

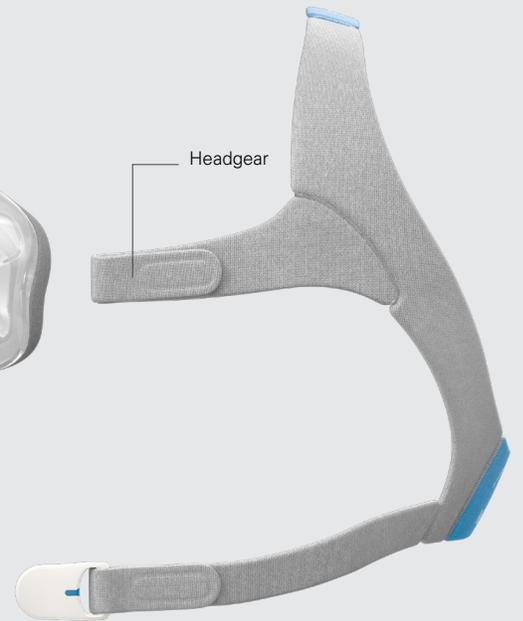

Frame system (includes frame, cushion and elbow with short tube)

63953 (S)
63955 (M)
63956 (L)
63954 for Her (S)



N20 headgear with headgear clips
63560 (S)
63561 (Std)
63562 (L)



N20 for Her headgear with headgear clips
63558 (S)


Product codes

Complete mask systems (Includes frame system + headgear)

Product name	Cushions included	Product code
AirTouch N20: Small	S	63903
AirTouch N20: Medium	M	63901
AirTouch N20: Large	L	63902
AirTouch N20 for Her	S	63900

Legend: **S**–Small **M**–Medium **L**–Large **Std**–Standard

AirTouch™ N20

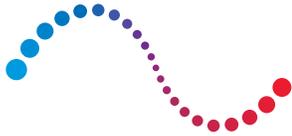

ResMed

AirTouch™ N20

Fitting guide for AirTouch N20



- Twist and pull both magnetic clips away from the frame.



- With both lower headgear straps released, hold the mask against the patient's face.
- Pull the headgear over the patient's head, ensuring that the ResMed logo on the headgear is facing up.



- Bring the lower straps below the patient's ears and attach the magnetic clip to the frame.



- Undo the fastening tabs on the upper headgear straps.
- Adjust the straps evenly until the mask sits comfortably just over the nose.



- Undo the fastening tabs on the lower headgear straps.
- Adjust the straps evenly until the mask is stable.
- Reattach the fastening tabs.



- Connect the air tube on the device to the short tube.
- Attach the elbow to the mask. First, press the elbow's side buttons, and then push the elbow into the mask, ensuring it clicks in on both sides.
- Start the device.



Refer to the user guide for further instructions and complete labeling information, including indications, contraindications, warnings and precautions.

* CPAP users rated mask criteria on a Likert Scale of 0-10. A 'comfortable mask' is equal to a score of 8 or higher. An 'excellent seal (no leak)' is equal to a score of 10. 'Easy to use' is defined as easy to fit, adjust and remove.

† CPAP users rated mask criteria on a Likert Scale of 1-5. Scores were 4 (Good) to 5 (Very good).

‡ AirTouch N20 is validated at therapy pressures up to 30 cmH₂O

1 ResMed guided external clinical study of 25 evaluable, existing CPAP mask users from Mar. 15 to Apr. 5, 2018, in Bella Vista, AU who trialed ResMed AirTouch N20 and ResMed AirFit™ N20 at home for 7 nights each in place of their prescribed nasal mask. 2 ResMed guided external clinical study of 29 evaluable CPAP users from Sept. 11, 2017, to Feb. 19, 2018, in San Diego, CA, who trialed AirTouch F20 at home for 30 nights each in place of their prescribed mask. Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA (858) 836-5000 or (800) 424-0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit, AirTouch, InfinitySeal and UltraSoft are trademarks and/or registered trademarks of the ResMed family of companies. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2020 ResMed. 10111154/2 2020-02