

Mirage™ SoftGel

NASAL MASK

Fitting Guide



- Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



- Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.



- Reconnect the lower headgear clip ensuring it clicks into place.



- Adjust the upper straps evenly. Do not overtighten.



- Adjust the lower straps to a comfortable position. Do not overtighten.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.



- Adjust dial to correct any leak at nose.



- Adjust bottom straps to correct any leak at top lip.