

Fitting Guide



- Prior to fitting the mask, remove the elbow from the mask by pressing the side tabs.



- Connect the elbow to the device air tubing.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, loop the headgear into the lower hooks on the mask frame.



- Unfasten the Velcro®, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the Velcro.



- Repeat step 5 with the lower headgear straps.



- Attach the combined elbow assembly and air tubing to the mask by pressing the side tabs and pushing into mask frame.
- Turn on the therapy device.



- Adjust the back strap to fine-tune the fitting if necessary.
- Do not overtighten.
- The back strap can be positioned over or under the hair, with hair tied or untied.

Mirage™ FX for Her
NASAL MASK

www.resmedmiragefx.com