Mirage™ SoftGel

NASAL MASK







User Guide

Svenska



Mirage™ SoftGel




Fitting / Maskinpassning / Tilpasning / Tilpasning / Sovitusohje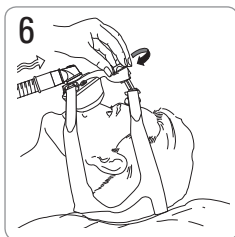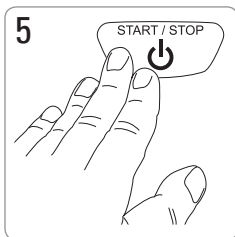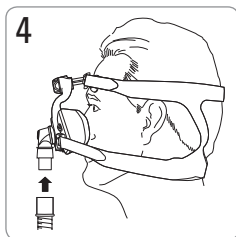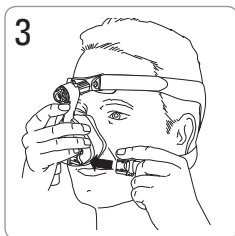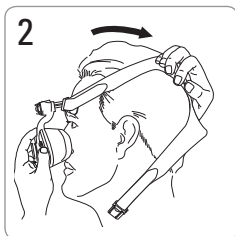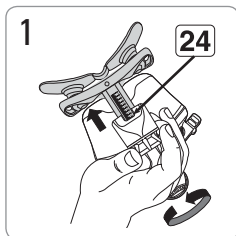

Removal / Avlägsnande / Aftagning / Fjerne masken / Irrottaminen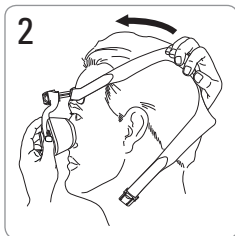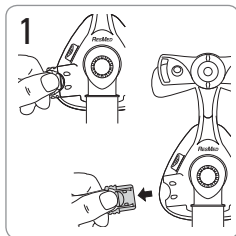

Mirage SoftGel

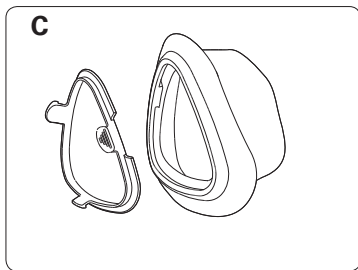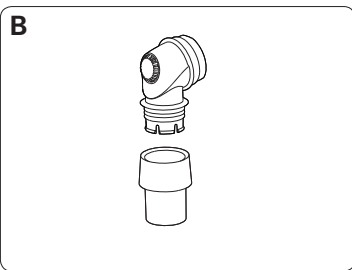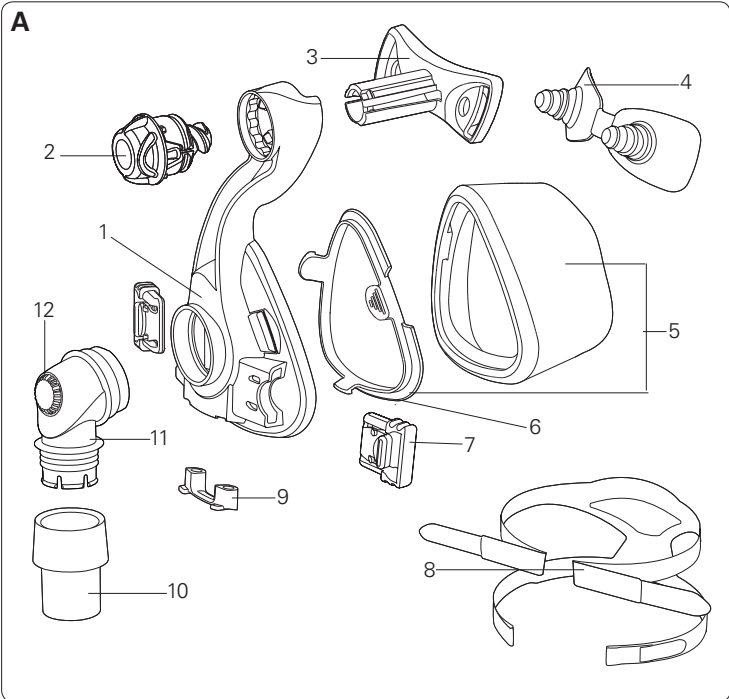Mask components / Maskkomponenter / Maskekomponenter / Maskens komponenter / Maskin osat

Item / Produkt / Vare / Artikkel / Osa	Description / Beskrivning / Beskrivelse / Beskrivelse / Nimitys	Part number / Artikelnummer / Delnummer / Artikkelnummer / Tuotenumero
1	Mask frame / Maskram / Maskeramme / Maskeramme / Maskin runko	60180 (St) 60193 (S)
2	Dial / Kontrollratt / Reguleringsknap / MicroFit-bryter / Säädin	61289
3	Forehead support / Pannstöd / Pandestøtte / Pannestøtte / Otsatuki	16393
4	Forehead support pad / Pannstödsdyna / Pandepude / Pute til pannestøtte / Otsatuen pehmike	60123 (1) 60124 (10)
5	Soft Gel Cushion & clip / Mjukdel & clips / Næsepude og låsering / Cushion og klips / Pehmike ja kiinnike	61631 (S) 61632 (M) 61633 (L) 61634 (LW)
6	Cushion clip / Mjukdelsclips / Låsering / Cushionklips / Pehmikkeen kiinnike	60176 (St) 60197 (S)
7	Headgear clip / Clips för huvudbandet / Hovedbåndscips / Klips til hodestropp / Pääremmien kiinnike	16569 (2) 16734 (10)
8	Headgear / Huvudband / Hovedbånd / Hodestropper / Pääremmit	16118 (S) 16117 (M) 16119 (L)
9	Ports cap / Portlock / Porthætte / Porthette / Liitäntöjen korkki	16570 (2) 16571 (10)
10	Swivel / Swivel / Drejestuds / Svivel / Pyörivä liitin	16565
11	Elbow / Knärör / Vinkelstykke / Kneledd / Kulmakappale	16387
12	Vent / Ventil / Lufthuller / Luftventil / Ilma-aukko	-

A	Complete system / Komplet system / Komplet maske / Komplet maskesystem / Maski täydellisenä	61636 (S) 61637 (M) 61638 (L) 61639 (LW)
B	Elbow assembly / Knärörsmontering / Vinkelsamling / Kneleddsett / Kulmakappalesarja	16399
Also available / Övrigt / Desuden fås / Leveres også / Saatavissa myös		
C	Activa LT Cushion & clip / Mjukdel & clips / Næsepude og låsering / Cushion og klips / Pehmike ja kiinnike	60198 (S) 60177 (M) 60178 (L) 60179 (LW)

S Small / Small / Small / Liten / Pieni










M Medium / Medium / Medium / Medium / Keskikokoinen

L Large / Large / Large / Stor / Suuri













LW Large Wide / Large Wide / Large Wide / Stor Vid / Suuri, leveä

St Standard / Standard / Standard / Standard / Vakio

Disassembly / Demontering / Demontering / Demontering / Purkaminen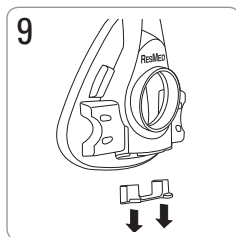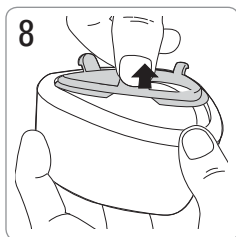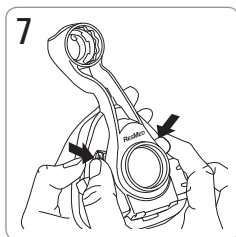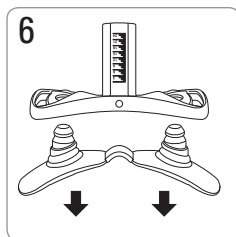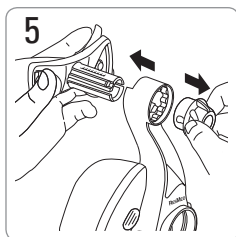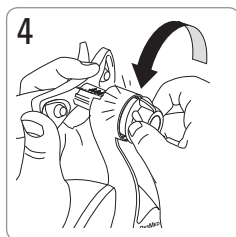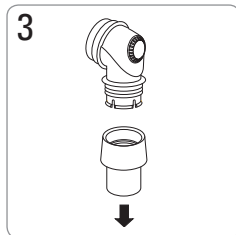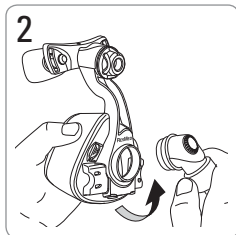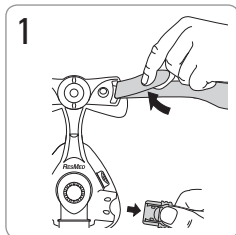

Reassembly / Montering / Genmontering / Montering / Kokoaminen

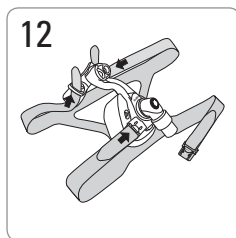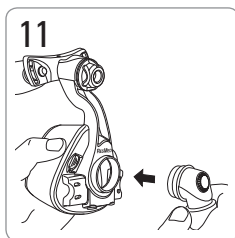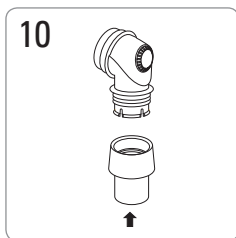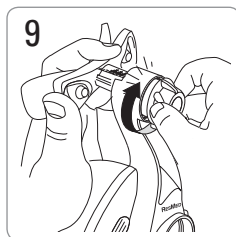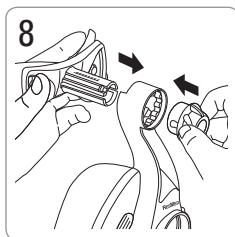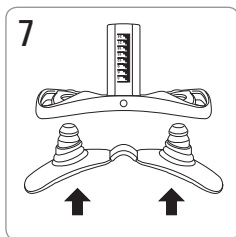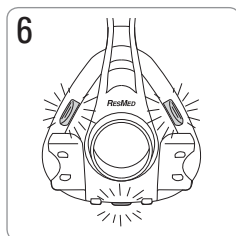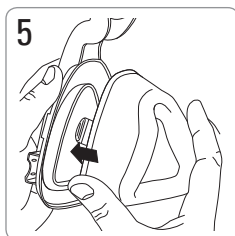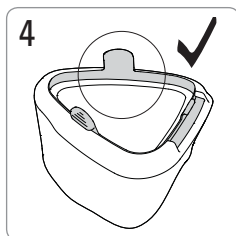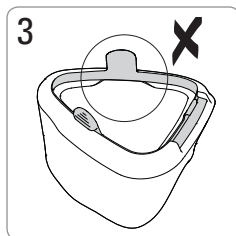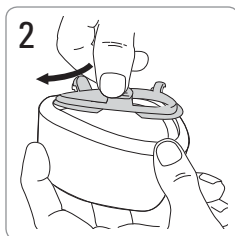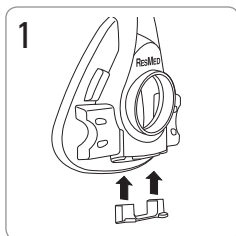Mirage™ SoftGel

NÄSMASK

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
English • Svenska • Dansk • Norsk • Suomi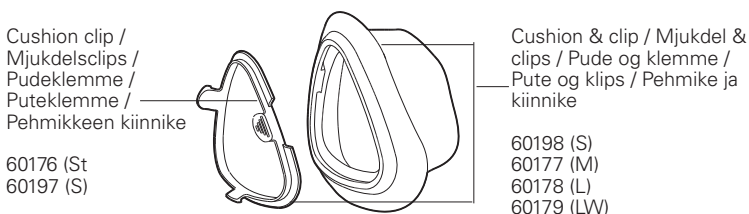

English

Please read this information in conjunction with your Mirage™ SoftGel User Guide when changing your Mirage SoftGel cushion to the Mirage Activa LT cushion.

Intended use

The Mirage Activa LT channels airflow noninvasively to a patient from a positive airway pressure device such as a continuous positive airway pressure (CPAP) or bilevel system.

The Mirage Activa LT is:

- to be used by adult patients (> 30 kg) for whom positive airway pressure has been prescribed
- intended for single-patient re-use in the home environment and multipatient re-use in the hospital/institutional environment.

Troubleshooting

Problem/possible cause	Solution
------------------------	----------

Mask is uncomfortable

Cushion incorrectly inflated	With the air turned off, adjust the straps so that you can insert two fingers between your cheek and the lower headgear strap.
------------------------------	--

Technical specifications

Dead space information

The dead space of the mask varies according to cushion size. It is 185 mL for the Medium size.

Resistance

Drop in pressure measured (nominal)
at 50 L/min: 0.1 cm H₂O
at 100 L/min: 0.7 cm H₂O

Sound

DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 30 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 22 dBA, with uncertainty 3 dBA.

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Mask setting options

Select 'Activa' (if available) or 'Standard' as the mask option when using the Mirage Activa LT with ResMed devices that have mask setting options.

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© 2011 ResMed.
Mirage and Activa are trademarks of ResMed Pty Ltd and are registered in U.S. Patent and Trademark Office.
ResMed Pty Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK