

Fitting guide for Quattro Air NV



- Remove the elbow from the mask by pressing the side buttons and pulling the elbow away. Then unclip the emergency release pinch clip (not using the cord) to release the lower headgear straps.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head. Ensure that the label on the back of the headgear faces outwards when the headgear is put on.



- Undo the fastening tabs on the upper headgear straps. Pull the straps evenly until the mask is stable and positioned as shown in the illustrations. Reattach the fastening tabs.



- Bring the lower straps below the ears. Loop the headgear onto the frame and clip the quick-release together.



- Undo the fastening tabs on the lower headgear straps. Pull the straps evenly until the mask sits firmly on the chin and reattach the fastening tabs.



- Start ventilation and then connect the air tubing from the device to the elbow. Attach the elbow to the mask by pressing the side buttons and pushing the elbow into the mask.



- Readjust the forehead straps so that the mask fits comfortably and an optimal seal is achieved. Overtightening may cause leaks.



- The mask and headgear should be positioned as shown.



Removing the mask using the quick-release cord:

- Pull the quick-release cord down. The quick-release clip separates from the mask.
- Pull the mask away from the face and back over the head.

Optimal cushion positioning



Align the bottom of the cushion with the crease of the chin and the top of the cushion with the corner of the eye.



Quattro Air NV sits on the nose at, or below, eye level.



Delivering NIV ... the lighter way

A busy healthcare environment requires a mask that delivers high-quality treatment with minimal fuss. Likewise, a patient on long-term, noninvasive ventilation (NIV) requires a mask that's comfortable and easy-to-use. ResMed's Quattro™ Air NV does it all – in a lightweight mask designed to meet a range of needs in the hospital and the home.

Comfort

The SoftEdge™ headgear is made of premium rolled-edge fabric that's gentle on the skin. Its lightweight design may also help minimise skin breakdown to enhance patient comfort over long-term use.

Efficiency

Quattro Air NV has been designed to make initiating NIV easy and effective. Its Spring Air™ cushion is designed to provide a secure seal throughout the mask.

Simplicity


With only five parts, the mask can be assembled in minimal time. And thanks to its quick-release clip with cord, it can be removed quickly and easily in an emergency.

Note: For complete user instructions, refer to the Quattro Air NV user guide.


Taking care of business

Quattro Air NV has been designed to deliver effective noninvasive ventilation to your patients, while maximising time and cost savings for you. It's made up of just five parts, so it's quick and easy to assemble and disassemble. Its modularity makes it straightforward to clean and reprocess for safe, multi-patient use. And because it shares common components with Quattro Air, inventory management is streamlined and cost-efficient. So with Quattro Air NV taking care of your business needs, you can spend more time taking care of what's most important—your patients.

Quattro Air NV features and benefits

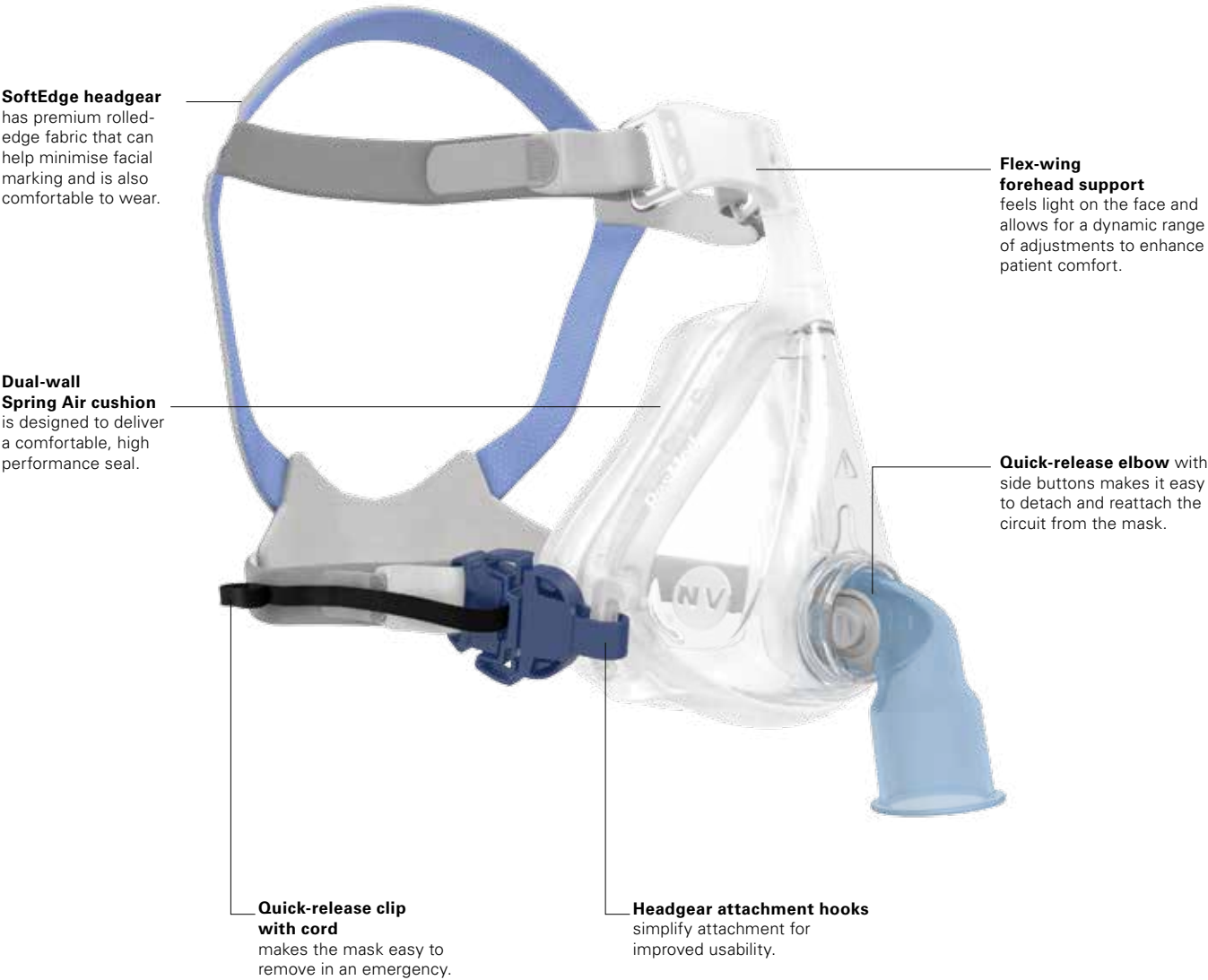

Quattro Air NV components

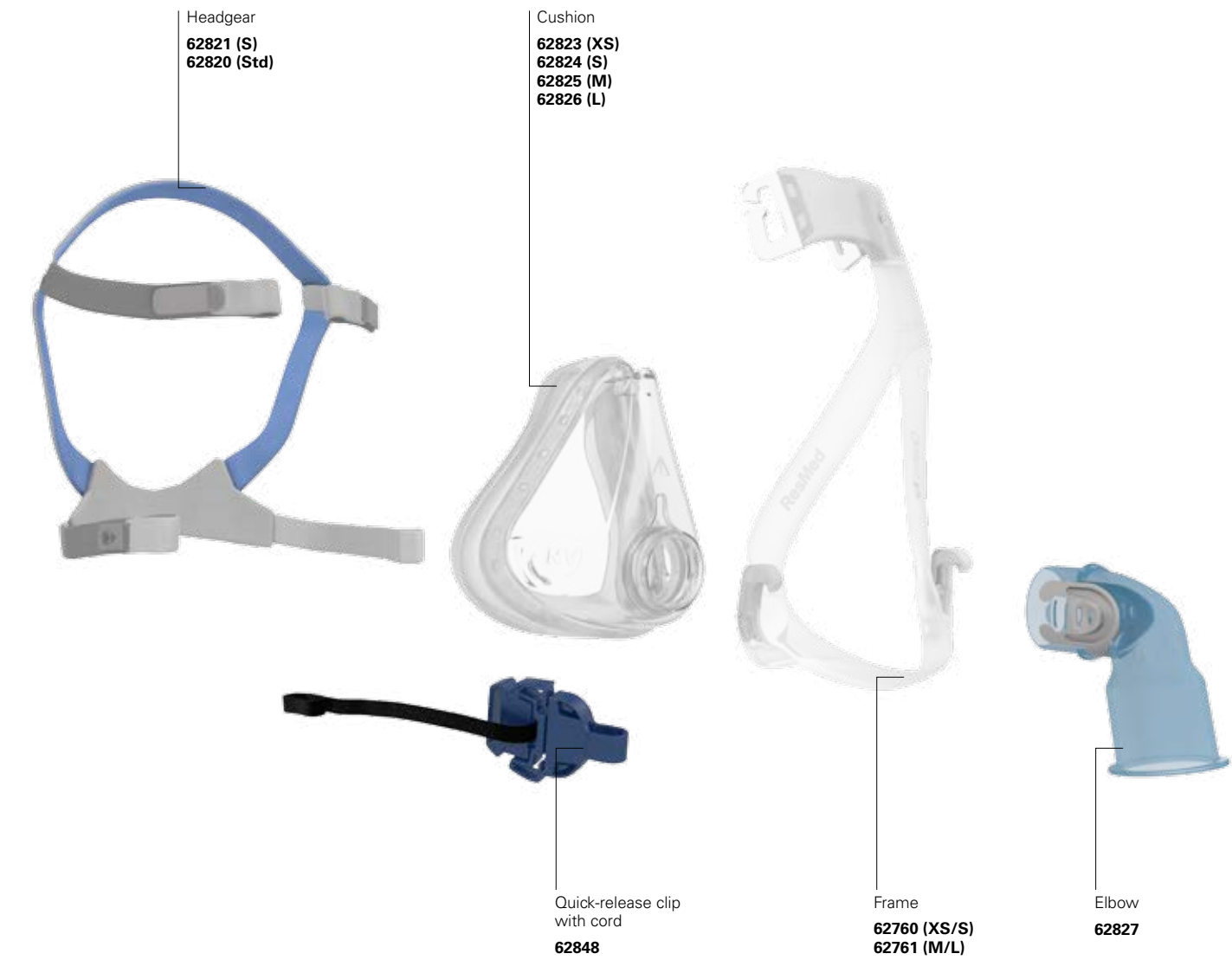

The XS mask system comes with the Small sized headgear
The S, M, & L mask systems come with the Standard size headgear
Note: For more details, refer to the Quattro Air NV user guide.

Product codes	
(Complete mask system)	
	Europe
Quattro Air NV mask system XS	62782
Quattro Air NV mask system S	62783
Quattro Air NV mask system M	62784
Quattro Air NV mask system L	62785

Quattro™ Air NV
Non-vented full face mask

Quattro Air NV sizing tips

Use the fitting template below as a guide to size your patients for Quattro Air NV. Ensure the patient's mouth is relaxed and closed when sizing.

