

Swift™ LT

NASAL PILLOWS SYSTEM

Fitting Guide



- Holding the pillows at your nose, pull the headgear over your head.
- Headgear should sit midway between the ears and eyes.



- Adjust the top straps by pulling them evenly until comfortable.
- Do not overtighten.



- Repeat Step 2 with the back straps.
- Do not overtighten.



- Using the click-fit adjuster, position the nasal pillows at the most suitable angle.
- Connect the Swift LT short tube to the device air tubing.
- Turn on the device.



- The Swift LT is now fitted and ready for use.



Optional

- For alternate tubing positions, use the tube retainer.



- To attach, push it into the top headgear buckle.
- Clip in either the short tube or device air tubing, depending on flexibility needs.
- The tubing can be positioned to either side of your face or centered.
- If the pillows pull on your nose, adjust the tubing at the tube retainer.

 For details, refer to the *Swift LT Nasal Pillows System User Guide*.