



Noxturnal™ / Activate a licence

After installation, Noxturnal can be used with unlimited functionality for 60 days without licence activation. After this initial period, the software will revert to standard functionality, which allows the use of the Nox T3™ system for ambulatory polygraphy. All other features need the activation of their respective licences (see below).

Once activated, a Noxturnal licence is specific to one computer. Licences cannot be ported from one computer to another nor can they be shared within a network.

If multiple licences need to be installed on the same computer, each licence must be activated individually.

Noxturnal Licences

PN	Licence
7073473	Noxturnal A1 PSG File Format Licence Enables reading / analysis / reporting of Nox A1™ recordings (incl. video playback). Enables configuration and download of ambulatory Nox A1 recordings.
7073474	Noxturnal A1 Online Acquisition Licence Enables online acquisition from the Nox A1 without video. Enables reading / analysis / reporting of Nox A1 recordings (incl. video playback). Enables configuration and download of ambulatory Nox A1 recordings.
7073475	Noxturnal Video Online Acquisition Licence Enables online video acquisition (to be used with Noxturnal A1 Online Acquisition Licence).
7073477	Noxturnal EBM PG File Format Reading Licence Enables reading of polygraphy recordings in Embla format (Embletta & Embletta Gold).
7073973	Noxturnal EBM PSG File Format Reading Licence Enables reading of polysomnography recordings in Embla format (S4500, Titanium, etc.).
7073478	Noxturnal EDF File Format Reading Licence Enables import and reading of recordings in EDF format.

Licence Activation

You must have administrator rights on the computer to activate a Noxturnal licence.

After ordering a Noxturnal licence, you will receive a card containing a licence key.
To activate the licence:


1. Go to <http://license.noxmedical.com>. Enter your email address. It is also possible to complete additional fields (e.g. the name of your organisation). Click on **Continue**.



2. Enter the licence key and click on **Submit**.



3. Enter the PC identifier. To find the identifier, open Noxturnal and look under **Help > About Application > License > Activate feature**. *The PC identifier is a unique number given by Noxturnal to tie a Noxturnal licence to a specific computer. Make sure you are entering the PC identifier of the computer on which you want to activate the licence.* Click on **Submit**.

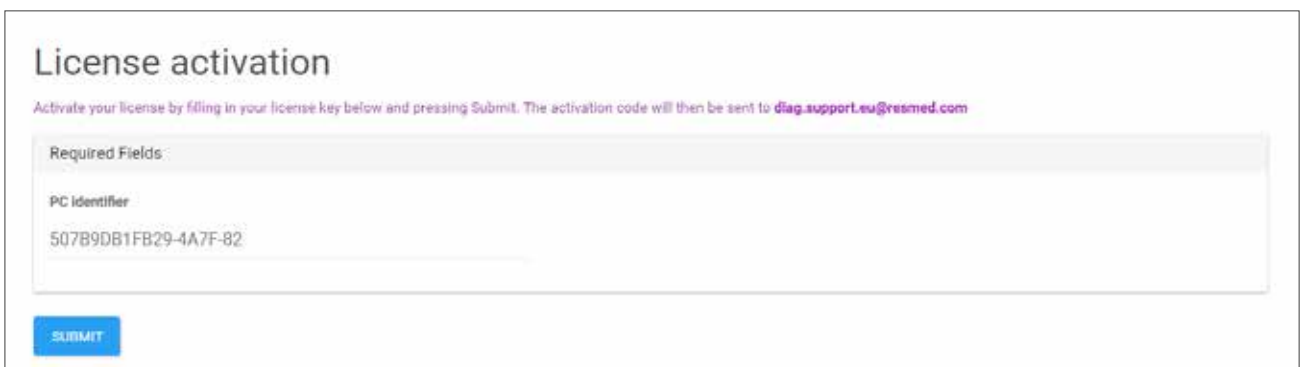

- After submitting your activation request, you will receive an activation code by email.

Dear diag.support.eu@resmed.com

You have activated your license by entering the following info:

License key

3618-EC03-782F-4239-8508-02C5-99AD-9944

PC identifier

507B9DB1FB29-3A4E-7D

and have therefore enabled the following:

Video Online Acquisition

Enables Noxturnal to do video acquisition during online recordings.

499B-1662-AF3E-B104-EC16-4086-C7C0-8E75

Activation code

For any additional help contact support@noxmedical.com

- Open Noxturnal as an administrator. To do this, right click on the **Noxturnal icon** > **Run it as an administrator**.
- Go into **Help > About Application > License > Activate feature** and enter the activation code. Click on **OK**.

Activate feature

Visit license.noxmedical.com to activate additional features. You will need to input the PC identifier below and a License Key from your local distributor.

PC identifier
507B9DB1FB29-4A7F-82 Copy

Activation Code
499B-1662-AF3E-B104-EC16-4086-C7C0-8E75

OK Cancel

- Verify that the licence has been correctly activated.

License Information

Standard License
Additional features are available for activation

Features : Activate feature

Feature	Activated	Expires
Nox A1 PSG File Format License	Yes	
Nox A1 Online Acquisition License	Yes	
Video Online Acquisition License	Yes	
EBM PG File Format Reading License	Yes	
EBM PSG File Format Reading License	Yes	
EDF File Format Reading License	No	
Command Line Application License	No	

Close

