



Mask/Device Compatibility List

English

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

Note: Not all products are available in all regions.

Full Face Masks

Device	AcuCare F1-4	AirFit™ F30i ¹	AirFit™ F30	AirTouch™ F20 / AirTouch™ F20 for Her / AirFit™ F20 / AirFit™ F20 for Her	AirFit™ F10 / AirFit™ F10 for Her	Mirage™	Mirage Series 2	Ultra Mirage™	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her / Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Hospital Full Face
AirMini™	X	X	✓	✓	X	X	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	N/V	Full Face	Full Face	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	N/V	Full Face	Full Face	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 CS Pacewave	N/V	Full Face	Full Face	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 CS-A Pacewave	N/V	N/V	Full Face	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
Lumis™	N/V	N/V	Full Face ³	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
S9 AutoSet™ / S9 Elite™	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
S9 Escape™	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AutoSet CS™ 2	N/V	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE ¹	FULL FACE
AutoSet CS™ (S9)	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AutoSet CS™ (S9 PaceWave)	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AutoSet CS-A™ (S9 PaceWave)	N/V	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
C-Series Tango™	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.							
S7™ Lightweight	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.							
S7 Elite™ / S7 AutoSet Spirit	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S7 AutoSet Respond™	N/V	N/V	N/V	N/V	MIRAGE	STANDARD	ULTRA	MIRAGE	MIRAGE	MIRAGE	MIR FULL	MIR FULL
S8™ Auto 25	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Spirit™	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Vantage™	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™	N/V	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.							
S8 Escape™ II	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.								
S8 Elite™ / S8 Elite™ II	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Spirit™ II	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.							

Device	AcuCare F1-4	AirFit™ F30i ¹	AirFit™ F30	AirTouch™ F20 / AirTouch™ F20 for Her AirFit™ F20 / AirFit™ F20 for Her	AirFit™ F10 / AirFit™ F10 for Her	Mirage™	Mirage Series 2	Ultra Mirage™	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Hospital Full Face
VPAP™ III / VPAP™ III ST	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ IV / VPAP™ IV ST	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Adapt SV	N/V	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE ¹	FULL FACE
VPAP™ Adapt (S9)	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Adapt (S9 PaceWave)	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Malibu	N/V	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ Auto 25	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
Auto 25 (S9)	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ S / VPAP™ ST	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9)	N/V	Full Face	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST-A (S9)	N/V	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	N/V	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
S9 VPAP™ Tx	N/V	Full Face ²	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Tx	N/V	N/V	N/V	N/V	FULL FACE	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
Stellar™	N/V	N/V	N/V	FULL FACE	FULL FACE	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	✓	N/V	N/V	N/V	✓	N/V	N/V	✓	✓	N/V	✓	✓
VS III™	N/V	N/V	N/V	N/V	✓	N/V	N/V	✓	✓	N/V	✓	✓
Astral™	FULL FACE	N/V	N/V	FULL FACE	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

² Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

³ Treatment efficacy may be impacted in areas such as accuracy of control and AutoEPAP, particularly at higher pressures. Caution: Ventilation and alarms effectiveness should be verified, including after any change of; ventilation and alarm setting, circuit configuration, or co-therapy (e.g. nebulisation, oxygen flow)

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Masks 1

Device	AirFit™ N30 for AirMini	AirFit™ N30i ¹ / AirFit™ N30	AirFit™ N20 Classic	AirFit™ N20 / AirFit™ N20 for Her	AirTouch™ N20 / AirTouch™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage™ FX for Her	Mirage Activa™ LT	Mirage™ SoftGel	Hospital Nasal
AirMini™	✓	X	✓	✓	✓	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
AirCurve™ 10 CS Pacewave	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
AirCurve™ 10 CS-A Pacewave	X	N/V	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
Lumis™	X	N/V	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
S9 AutoSet™ / S9 Elite™	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
S9 Escape™	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
AutoSet CS™ 2	X	N/V	N/V	N/V	N/V	N/V	N/V	VISTA ¹	ACTIVA	ACTIVA	VISTA
AutoSet CS™ (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal

Device	AirFit™ N30 for AirMini	AirFit™ N30 ¹ / AirFit™ N30	AirFit™ N20 Classic	AirFit™ N20 / AirFit™ N20 for Her	AirTouch™ N20 / AirTouch™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage FX™ for Her	Mirage Activa™ LT	Mirage™ SoftGel	Hospital Nasal
AutoSet CS™ (S9 PaceWave)	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
AutoSet CS-A™ (S9 PaceWave)	X	N/V	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
C-Series Tango™	X	N/V	N/V	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.			
S7™ Lightweight	X	N/V	N/V	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.			
S7 Elite™ / S7 AutoSet Spirit	X	N/V	N/V	N/V	N/V	N/V	N/V	STANDARD	STANDARD	STANDARD	STANDARD
S7 AutoSet Respond™	X	N/V	N/V	N/V	N/V	N/V	N/V	STANDARD	STANDARD	STANDARD	STANDARD
S8™ Auto 25	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 AutoSet Spirit™	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 AutoSet Vantage™	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 Escape™	X	N/V	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.					
S8 Escape™ II	X	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.							
S8 Elite™ / S8 Elite™ II	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 AutoSet Spirit™ II	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 Compact™	X	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.					
VPAP™ III / VPAP™ III ST	X	N/V	N/V	N/V	N/V	MIRAGE	MIRAGE	STANDARD	STANDARD	STANDARD	N/V
VPAP™ III ST-A	X	N/V	N/V	N/V	N/V	MIRAGE	MIRAGE	STANDARD	STANDARD	STANDARD	N/V
VPAP™ IV / VPAP™ IV ST	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
VPAP™ Adapt SV	X	N/V	N/V	N/V	N/V	N/V	N/V	VISTA ¹	ACTIVA	ACTIVA	VISTA
VPAP™ Adapt (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
VPAP™ Adapt (S9 PaceWave)	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
VPAP™ Malibu	X	N/V	N/V	N/V	N/V	SWIFT	SWIFT	VISTA ¹	N/V	N/V	MERIDIAN
VPAP™ Auto 25	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
Auto 25 (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
VPAP™ S / VPAP™ ST	X	N/V	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
VPAP™ S (S9) / VPAP™ ST (S9)	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
VPAP™ ST-A (S9)	X	N/V	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	X	N/V	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
S9 VPAP™ Tx	X	Pillows ²	N/V	Pillows	N/V	Pillows	Pillows	Nasal	Nasal	Nasal	Nasal
VPAP™ Tx	X	N/V	N/V	N/V	N/V	PILLOWS	PILLOWS	NASAL	NASAL	NASAL	NASAL
Stellar™	X	N/V	PILLOWS	PILLOWS	PILLOWS	PILLOWS ¹	PILLOWS ¹	NASAL	NASAL	NASAL	NASAL
VS Ultra™ / VS Integra™	X	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	X	N/V	N/V	N/V	N/V	N/V	N/V	N/V	✓	N/V	✓
Astral™	X	N/V	Pillows	Pillows	Pillows	N/V	N/V	Nasal	N/V	Nasal	Nasal

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

² Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Masks 2

Device	Pixi™	Mirage Kidsta™	Mirage Micro™	Mirage Vista™	Ultra Mirage II	Mirage	Mirage Activa™	Modular	Papillon™	Meridian
AirMini™	X	X	X	X	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
AirCurve™ 10 CS Pacewave	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
AirCurve™ 10 CS-A Pacewave	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
Lumis™	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
S9 AutoSet™ / S9 Elite™	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
S9 Escape™	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
AutoSet CS™ 2	N/V	N/V	VISTA¹	VISTA	ULTRA	N/V	ACTIVA	N/V	PAPILLON	VISTA
AutoSet CS™ (S9)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
AutoSet CS™ (S9 PaceWave) / AutoSet CS-A™ (S9 PaceWave)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
C-Series Tango™	N/V	N/V	N/V		No setting required. Consult your clinician before changing to a different mask type.					N/V
S7™ Lightweight	N/V	N/V	N/V		No setting required. Consult your clinician before changing to a different mask type.					N/V
S7 Elite™ / S7 AutoSet Spirit	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	STANDARD	N/V
S7 AutoSet Respond™	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	STANDARD	N/V
S8™ Auto 25	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD
S8 AutoSet Spirit™	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD
S8 AutoSet Vantage™	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD
S8 Escape™	N/V	N/V			Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.					
S8 Escape™ II	N/V	N/V			Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.					
S8 Elite™ / S8 Elite™ II	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD
S8 AutoSet Spirit™ II	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD
S8 Compact™	N/V	N/V			No setting required. Consult your clinician before changing to a different mask type.					
VPAP™ III / VPAP™ III ST	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	N/V	N/V
VPAP™ III ST-A	N/V	STANDARD	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	N/V	N/V
VPAP™ IV / VPAP™ IV ST	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD
VPAP™ Adapt SV	N/V	N/V	VISTA¹	VISTA	ULTRA	N/V	ACTIVA	N/V	N/V	VISTA
VPAP™ Adapt (S9)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
VPAP™ Adapt (S9 PaceWave)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
VPAP™ Malibu	N/V	N/V	MERIDIAN	VISTA	ULTRA	N/V	N/V	N/V	N/V	MERIDIAN
VPAP™ Auto 25	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD
Auto 25 (S9)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	STANDARD
VPAP™ S / VPAP™ ST	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD
VPAP™ S (S9) / VPAP™ ST (S9) / VPAP™ ST-A (S9)	Pediatric	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	Pediatric	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
S9 VPAP™ Tx	Pediatric	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	N/V	N/V	Nasal
VPAP™ Tx	N/V	NASAL	NASAL	NASAL	NASAL-ULTRA	N/V	NASAL	N/V	N/V	NASAL

Device	Pixi™	Mirage Kidsta™	Mirage Micro™	Mirage Vista™	Ultra Mirage II	Mirage	Mirage Activa™	Modular	Papillon™	Meridian
Stellar™	Pediatric	NASAL	NASAL	NASAL	ULTRA	N/V	NASAL	N/V	N/V	N/V
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	✓	✓	✓	✓	N/V	✓	N/V	N/V	N/V
Astral™	Pediatric	Nasal	Nasal	Nasal	Nasal Ultra	N/V	N/V	N/V	N/V	N/V

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

Nasal Pillows

Device	AirFit™ P30i ¹	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Mirage Swift™	Mirage Swift™ II	Swift™ FX / Swift™ FX for Her / Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT	Swift™ LT for Her / Swift™ LT-F
AirMini™	X	✓	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Pillows	X	Pillows ²	N/V	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Pillows	X	Pillows ²	N/V	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 CS Pacewave	Pillows	X	Pillows ²	N/V	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 CS-A Pacewave	N/V	X	Pillows ²	N/V	Pillows	Pillows	Pillows	Pillows
Lumis™	N/V	X	Pillows ²	N/V	Pillows	Pillows	Pillows	Pillows
S9 AutoSet™ / S9 Elite™	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
S9 Escape™	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Pillows
AutoSet CS™ 2	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
AutoSet CS™ (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
AutoSet CS™ (S9 PaceWave)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
AutoSet CS-A™ (S9 PaceWave)	N/V	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	N/V	X		No setting required. Consult your clinician before changing to a different mask type.				
S7™ Lightweight	N/V	X		No setting required. Consult your clinician before changing to a different mask type.				
S7 Elite™ / S7 AutoSet Spirit	N/V	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S7 AutoSet Respond™	N/V	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8™ Auto 25	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 AutoSet Spirit™	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 AutoSet Vantage™	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	N/V	X		Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.				
S8 Escape™ II	N/V	X		Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.				
S8 Elite™ / S8 Elite™ II	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 AutoSet Spirit™ II	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	N/V	X		No setting required. Consult your clinician before changing to a different mask type.				
VPAP™ III / VPAP™ III ST	N/V	X	MIRAGE ²	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	N/V	X	MIRAGE ²	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ IV / VPAP™ IV ST	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Adapt SV	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ Adapt (S9 PaceWave)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows

Device	AirFit™ P30i ¹	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Mirage Swift™	Mirage Swift™ II	Swift™ FX / Swift™ FX for Her / Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT	Swift™ LT for Her / Swift™ LT-F
VPAP™ Malibu	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Auto 25	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
Auto 25 (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ S / VPAP™ ST	N/V	X	SWIFT ²	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ ST-A (S9)	N/V	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	N/V	X	Pillows ²	Pillows	Pillows	Pillows	Pillows	Pillows
S9 VPAP™ Tx	Pillows ³	X	Pillows ²	N/V	Pillows	Pillows	Pillows	Pillows
VPAP™ Tx	N/V	X	PILLOWS ²	PILLOWS	PILLOWS	PILLOWS	PILLOWS	PILLOWS
Stellar™	N/V	X	PILLOWS ^{1, 2}	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹	PILLOWS ¹
VS Ultra™ / VS Integra™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V
Astral™	N/V	X	N/V	Pillows	Pillows	Pillows	Pillows	Pillows

¹ SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

² Accuracy of tidal volume/minute ventilation may reduce using the AirFit P10 mask in bilevel modes, particularly at higher pressure supports.

³ Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible