


Mirage Quattro Mask Fitting Template

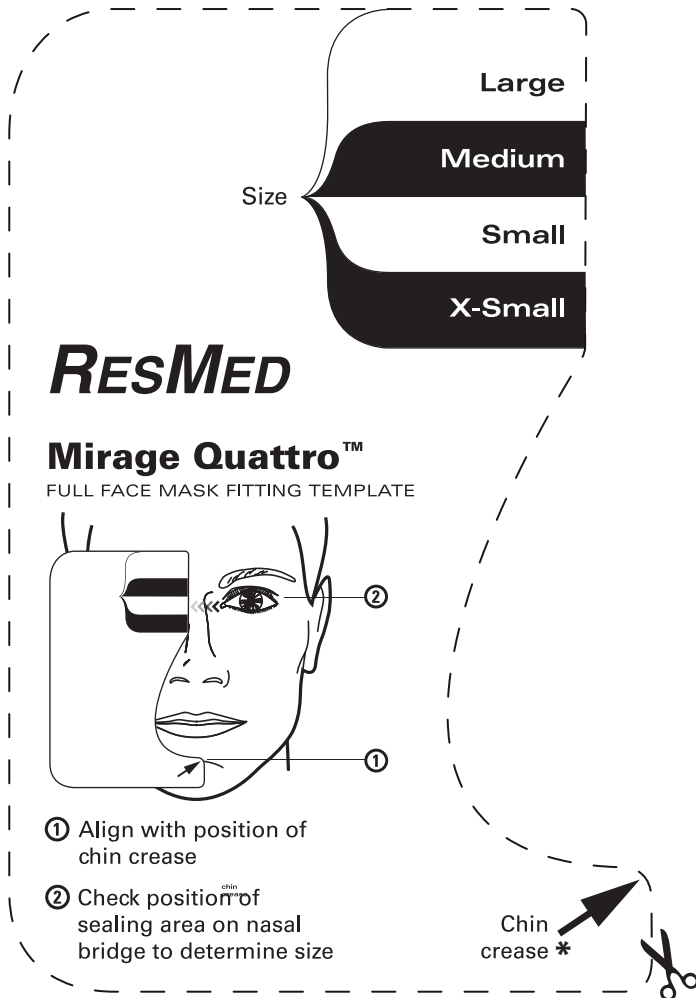


How to use the fitting template:

1. Cut out template along the dotted line
2. Follow the directions on the template to determine appropriate cushion size

*** If chin crease falls between sizes, choose smaller size.**

- ① Align with position of chin crease
- ② Check position of sealing area on nasal bridge to determine size

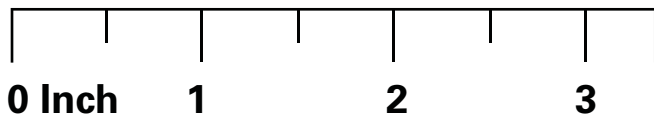

Actual Size: 3 ³/₈ inch (Width) X 5 ¹/₈ inch (Height)

Ruler is for scaling purposes. To ensure this print is accurate, match a ruler with the ruler above.

Note: Mask fitting templates should be used to aide patients in the fitting of CPAP masks. Altering the size or dimensions of the original online PDF in any way may compromise the quality and accuracy of this template. The use of this template is not intended to replace the advice of a medical professional, sleep technician or HME dealer.