

Designed especially **for women**



Quattro™ FX for Her is a light, unobtrusive full face mask, providing users with an effective alternative to conventional full face therapy.

Benefits for you ...

It's as easy as: **Size. Swing. Seal**

Quattro FX for Her gives you the tools to achieve the best fit for your patients—first time, every time. The custom-made sizing tool and **Swing Seal** fitting technique simplify the sizing, fitting and sealing process ... saving you time, money and additional patient visits.

Time and cost savings

The suspension-like combination of the Spring Air™ cushion and Spring frame means forehead support is unnecessary. Eliminating an additional point of adjustment, fitting Quattro FX for Her is fast and straightforward.

Therapy acceptance

Quattro FX for Her is designed to provide performance, comfort and a feeling of openness, helping users accept and continue therapy ... at titration and beyond.

... and your patients

Quality and performance retained

New technologies in the Spring Air cushion and Spring frame offer the stability and angular adjustment provided by conventional forehead support features. Quattro FX for Her offers the quality and performance needed, in a less obtrusive full face mask.

Freedom to be

Fewer points of contact with the face reduce the chances of skin irritation; a clear line of sight helps break down the physical and emotional barriers associated with full face therapy.

A good night's sleep

With headgear hair management options, Quattro FX for Her fits in with her personal style ... hair can be worn up or down, and over or under the headgear straps.



Quattro™ FX for Her
FULL FACE MASK

Less mask. **More you**



Technology highlight

Size. Swing. Seal *NEW*

Sizing, fitting and sealing Quattro FX for Her is fast and straightforward.

The custom-made **sizing tool** provides an easy and accurate sizing process.

Once sized correctly, fitting the mask is easy too, with the **Swing Seal** technique:

- 1 Firstly position the cushion at an angle on the nasal bridge ...
- 2 ... then **swing** the lower half of the mask down against the face to **seal**.

Quattro FX for Her: fits right, seals securely ... first time.

Clear line of sight

and a feeling of openness created by a full face mask that starts mid-way on the nasal bridge; a smaller frame also makes side-sleeping more comfortable

Built-in vent design

disperses air gently and quietly

Flexible, durable Spring frame

is strong enough to stabilise the mask on the face, and soft enough to accommodate user movement during sleep

360° elbow rotation

positions the tubing for ease of use

New, tension-reducing headgear design

features a contoured crown strap that sits snug at the top of the head and avoids the sensitive area around the back of the neck

Caters for her personal style

Whether the user has long or short hair, wears it tied up or untied, the Quattro FX for Her headgear is comfortable and convenient ... and it's finished in a more feminine colour scheme, just for her



Dual-wall, Spring Air cushion provides soft, even pressure distribution

Set-and-forget headgear clips are easy to grip for attaching and detaching, and retain desired headgear settings

Swivel offers a convenient way to connect and disconnect air tubing



Contoured cushion membrane around sensitive nasal bridge area rolls and inflates to gently meet the nose rather than stretch over it

Product Codes

| | Europe 1 | Europe 2 | Europe 3 |
|--------|--------------|--------------|--------------|
| Small | 62518 | 62520 | 62522 |
| Medium | 62519 | 62521 | 62523 |


Europe 1: English (UK), French, German and Italian (Switzerland)

Europe 2: English, Danish, Finnish, Norwegian and Swedish

Europe 3: English, Czech, Dutch (Belgium), French (Belgium), German (Belgium), Greek, Hungarian, Italian, Polish, Portuguese, Russian, Slovak, Spanish and Turkish

ResMed Ltd
1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. **EC REP** ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide. Quattro and Spring Air Technology are trademarks of ResMed Ltd and registered in U.S. Patent and Trademark Office. © 2011 ResMed Ltd. Specifications may change without notice. 1014871/2 2011-12

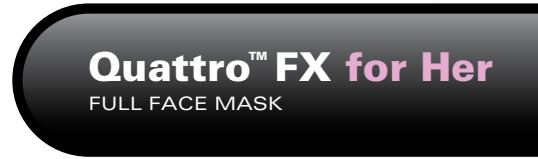

www.quattrofx.com