


Fitting Guide



- Hold the pillows at your nose, ensuring the ResMed logo is at the top. Pull one strap around and behind your ear.

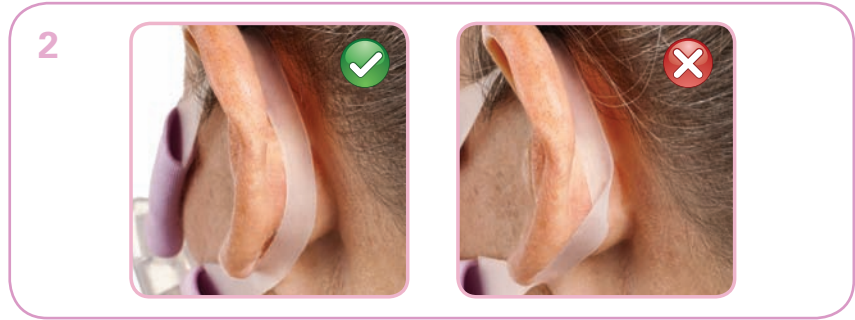

- Ensure that the strap lies flat and comfortably across the ear. Take care to avoid twisting the strap.



- Repeat the same process with the other strap.



- Adjust the mask's fit by pulling both of the upper straps at the same time, along the notches until you get a comfortable fit.



- Repeat the process on the mask's lower straps. Do not over tighten.



- Check that the pillows are upright at the entrance to the nose, then connect the free end of the short tube assembly to the air tubing of your device. Now it's ready for use.

Swift™ FX Bella
NASAL PILLOWS SYSTEM